smartmix X2

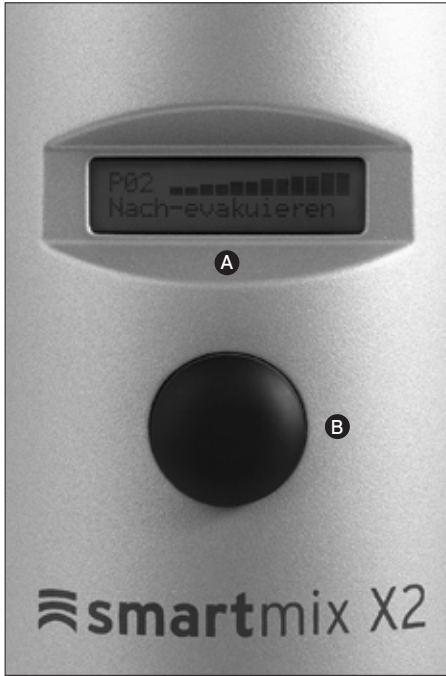
Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Modo de empleo

Deutsch	04 - 09
English	10 - 15
Française	16 - 23
Italiano	24 - 29
Español	30 - 35

SMARTMIX X2

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- A** LC-Display
- B** Multifunctional turning knob
- C** Preliminary filter

DESCRIPTION OF THE UNIT

The Smartmix X2 vacuum mixer is for mixing all types of plaster, investment material and silicone paste. This unit is constructed robustly. A twin-chamber vacuum pump creates a high vacuum in a very short time. As the vacuum sucks the mixing beaker onto the unit, no additional catches are required.

The multifunctional turning and pushing knob allows to operate and to program the Smartmix X2 very easily. The Smartmix X2 includes a high quality electronic control system which can be used to store up to 20 mixing programmes. Each mixing programme includes the following settings:

1. Mixing time

Stirring time is adjustable from 10 sec to 5 min.

2. Speed (150 - 550 r.p.m.)

The speed can be set in stages of 10 from 150 - 550 r.p.m.

3. Premixing

The premixing stage can be switched on or off. When switched on, an interval of 5 - 20 seconds (in stages of 5) can be programmed. To avoid

stirring up the powder, use a lower speed for pre-mixing. When the pre-mixing function is activated (time \geq 5 sec), a pre-mixing speed from 150 - 250 min^{-1} can be selected. If the premixing stage is activated, no hand mixing is required.

4. Prevacuum

This stage can also be switched on or off. When it is switched on, a constant vacuum is created in the mixing vessel without actually mixing. The intervals can be set between 5 and 60 seconds (progressively).

5. Interval time

The interval time denotes after how many seconds the direction of rotation of the mixing blade should be reversed. The shortest selectable interval time amounts to five seconds. Below five seconds, changing of the rotational direction is disabled.

6. Re-evacuation

Shiftable function. After the end of the stirring process, the mixture may rest under vacuum and be degassed. The time interval for re-evacuation can be selected from 5 - 30 seconds. Only after the end of the re-evacuation time a signal will sound to indicate completion of the mixing process.

7. Programme description

The name of a programme can be entered here. The name can have up to 12 characters.

All input is done using the multifunctional turning and pushing knob.

INSTALLATION

After unpacking the unit, please check it immediately for possible damage incurred during transport. Should the unit have been damaged during transport, please complain to your supplier immediately. Although this unit is supplied ready for use, always check the voltage setting (115/230 V) immediately above the power socket. If you are not certain which voltage is used in your country, please ask your local supplier of electrical appliances.

INSTALLING ON A BENCH

The stand is available as an accessory and consists of a firm, heavy base with a back. The Smartmix X2 is attached with its backside to the four screws at the backside plate of the pedestal. Turn in the securing screw to prevent unintended separation of the device from the pedestal. The unit is then ready for use.

FITTING TO A WALL

A wall drilling template is supplied with the unit for drilling holes in the wall quickly and exactly. Ensure that the holes are aligned exactly and horizontally (check with a spirit level if necessary). Once the holes have been drilled, press the rawplugs into them. Screw in the screws until approximately 5 - 6 mm protrude and hang the unit on them.

Caution: Special rawplugs must be used for sandwich-type plaster board walls!

1. Stick template onto required wall mounting position.
2. Mark the center of the 4 crossed lines using a tip end instrument.
3. Remove template and drill the holes. Now you can press the enclosed dowels into the holes. Screw the enclosed bolts into the dowels and leave a 0.20 - 0.24 inch space before they align with the dowels.
4. The bigger holes on the back (marked black on this template) will help you finding the screw-positions when hanging Smartmix X2 onto the wall. Check carefully if the device has engaged correctly on your wall before removing the weight from your arms when holding the device.

OPERATING THE UNIT

To start the Smartmix X2, first turn on the On/Off switch at the underside of the device. At the front, the device is equipped with a two-line LC display **(A)** and the multifunctional turning and pushing knob **(B)**. Using these controls, all functions of the Smartmix X2 can be reached. The device is ready for action immediately after power-up. When the Smartmix X2 is turned on for the first time, the language selection menu will appear. Select the desired language and confirm by pressing the button. After each subsequent start of the system, the stirring program most recently used will be indicated on the display.

Example:

Programme number			Programm description					
P01	Alpenrock							
Mixing	Time	0:30						

↑
Mixing time

LANGUAGE

The Smartmix X2 includes a multilingual menu (German, English, French, Italian and Spanish). If the language selected during installation should not be the preferred language, the language can be changed in program P21.

- _ Select program P21 by turning the turning-pushing knob.
- _ Keep the turning-pushing knob pressed for 1.5 seconds.
- _ The menu changes to Language Selection.
- _ Select the desired language by turning the turning-pushing knob.
- _ Confirm your selection by pressing the button. The unit is then ready for use.

MIXING

First select a suitable beaker size and prepare the materials to be mixed.

Note: The Smartmix stirring beakers have a marking for the maximum allowable filling height of the

material to be mixed. At high speeds (> 400 min⁻¹), the stirring beaker must not be filled up to that marking. In this case, reduce the filling quantity or use next larger beaker.

The material to be mixed does not require pre-mixing if the option «pre-mixing» has been selected (see chapter «Storing a stirring program»). Wipe the top edge of the mixing vessel clean and fit the lid. Shift from program P01 to program P20 by turning the multifunctional turning-pushing knob **(B)**.

Please note: The Smartmix X2 can be used even when no special stirring program has been entered yet. There are stirring parameters already deposited in the programs. Simply press the turning-pushing knob **(B)** for 1.5 seconds, select the desired stirring time using the turning knob and press again the turning-pushing knob **(B)** for 1.5 seconds to store. Now the device is ready for operation.

Start the mixing operation by briefly pressing the turning-pushing knob and move the stirring beaker up to the manifold from below. After only a few seconds, the suction holds the vessel in place. Once full vacuum has been attained, mixing commences automatically. The mixing time is counted backwards on the LC display. An alarm can be heard once mixing has been completed.

During mixing, the vacuum pump sucks the beaker onto the unit firmly. After completion of the mixing procedure, press again the turning-pushing knob to switch off the vacuum pump. Please hold tight the beaker during this step because immediately after pressure equalization it will fall off the device.

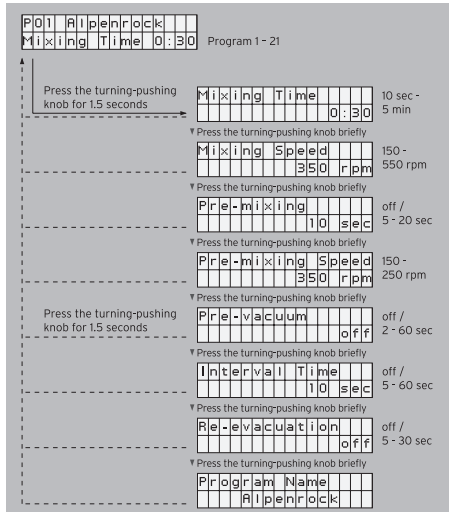
STORING MIXING PROGRAMMES

The user interface of the Smartmix X2 comprises two levels: The working level of the mixing programs 1 - 20 and the programming level for defining the stirring parameters for the individual program. Press the turning-pushing knob **(B)** for 1.5 sec to change between the two levels. Turn the turning-pushing knob to change the individual setting.

How to change between the two menu levels:

1. Change from the working level to the programming level: Press the turning-pushing knob for 1.5 sec to get into the programming level, where stirring parameters can be changed.
2. Change from the programming level to the working level: Any time and at any point you may return to the working level by pressing the turning-pushing knob for 1.5 seconds. The changes to the stirring parameters entered on the programming level will be stored immediately upon leaving the programming level.

MENU

PROGRAMMING

1. Turn the turning-pushing knob **(B)** to select the program to be modified.
2. Press the turning-pushing knob **(B)** for 1.5 seconds. Set the stirring time to the desired value (from 10 seconds to 5 minutes) by turning.
3. Confirm your input by briefly pressing the turning-pushing knob **(B)**. Brief pressing will cause the program to go to the next stirring parameter.
4. The next parameter is mixing speed. Select the desired speed by turning the turning-pushing knob and confirm by pressing briefly.
5. As the next parameter, pre-mixing may be switched off or on, entering the pre-mixing time (5 - 20 sec). Again, select by turning the

- turning-pushing knob **(B)** and confirm by pressing briefly. When a time for pre-mixing has been selected, the pre-mixing speed remains to be selected (150 - 250 min⁻¹) and confirmed by briefly pressing the turning-pushing knob.
6. Next there is the possibility of evacuating the material to be mixed prior to the mixing procedure proper without stirring. Pre-evacuation can be turned off or on, entering the pre-evacuation time (2 - 60 seconds). Adjust by turning the turning-pushing knob **(B)** and confirm by pressing briefly.
 7. The interval time is the parameter indicating after how many seconds the direction is to be changed during the mixing process.
 8. To let the material rest some time after mixing (e. g. for degassing of investment materials), the re-evacuation parameter can be set to a time from 5 - 30 seconds.
 9. Enter the program name into the last field. To this end, again turn the turning-pushing knob **(B)** until the desired letter is found. Confirm the selected letter by briefly pressing the turning-pushing knob **(B)**, and the cursor will automatically advance to the next position. Names comprising up to 12 letters can be stored. After entering the program name, exit the programming level by pressing the turning-pushing knob for 1.5 seconds. The program is stored, and the mixing process can be started. (Refer to the „Mixing“ section for further instructions).

VACUUM DISPLAY

During the stirring process, instead of the program designation on the display the level of the vacuum is indicated above the bar symbol.

REPLACING THE PRELIMINARY FILTER

The preliminary filter is inside the suction nozzle. The main filter inside the unit and the preliminary filter protect the vacuum pump against possible contamination. Regular inspection of the pre-filter is important. When the filter is soiled, it must be changed. For changing the filter, pull it off downwards and clip on the new pre-filter. Never operate the Smartmix X2 without a preliminary filter!

SERVICING

Like most AmannGirrbach equipment, the Smartmix X2 is designed to be maintenance-free and includes a maintenance-free vacuum pump. Simply keep the unit clean. The sealing surfaces of the suction nozzle and lid must be kept particularly clean to ensure that the vacuum is optimum. The seal can be kept supple by coating it periodically with a thin coat of Vaseline or similar grease.

Technical data:

Depth/Width/Height:	250 x 160 x 350 mm
Weight:	8,5 kg
Power supply:	115/230V - 50/60Hz
Fuse:	T 2,5A
Power uptake:	210 W
Mixing speed:	Up to 550 r.p.m.
Vakuum:	-800 mbar
Pump capacity:	15,8 l/min

Accessories and single components:

Art.Nr.:	
115730	Smartmix X2 stand
115620	Beaker with lid and blade S, 100 ccm
115630	Beaker with lid and blade S, 250 ccm
115640	Beaker with lid and blade S, 500 ccm
115650	Beaker with lid and blade S, 750 ccm
115660	Beaker with lid and blade S, 1,000 ccm
115621	Replacement breaker, 100 ccm
115631	Replacement breaker, 250 ccm
115641	Replacement breaker, 500 ccm
115651	Replacement breaker, 750 ccm
115661	Replacement breaker, 1,000 ccm
115701	Spare filters Smartmix X2 (5 pcs.)

Packing list:

Art.-Nr.	
115700	Smartmix X2 unit Beaker with stirring unit 500 ml, power cable, pre-filter, drilling stencil

WARNINGS AND SAFETY ASPECTS:

The Smartmix X2 was developed for use in dental laboratories. It must be installed and operated as described in the user's manual. The manufacturer is not liable for damage caused by using the unit for other purposes or handling it incorrectly.

In addition, if the intended application is not listed in the user's manual, the user is obliged to check personally in advance that the unit is suitable for this purpose.

- _ Do not clean the Smartmix X2 with liquids containing solvent**
- _ Keep the lid and mixing beaker clean - especially those parts which create the seal**
- _ Do not run the vacuum pump too long without connecting a mixing beaker**
- _ When releasing the vacuum, always hold the mixing beaker with your hand**
- _ Always operate the Smartmix X2 with a pre-filter!**

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Amann Girrbach AG
Amann Girrbach GmbH