# as400/as400 micro

**Modellherstellung**  
*Model fabrication*  
*Fabrication du modèle*  
*Sviluppo dei modelli*  
*Elaboración de modelos*

**Betriebsanleitung**  
**Operating instructions**  
**Mode de fonctionnement**  
**Istruzioni sul**  
**funzionamento**  
**Instrucciones de servicio**

**AMANNGIRRBACH**

Deutsch	04 - 05
English	06 - 07
Français	08 - 09
Italiano	10 - 11
Español	12 - 13

EU-Declaration of conformity	15
---------------------------------	----

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## OPERATING INSTRUCTIONS

Translation of the original operating instructions

- 1 On/Off Switch
- 2 Motor unit
- 3 Catch
- 4 Dust container
- 5 Suction hose
- 6 Microfilter for AS400micro

### DESCRIPTION OF UNIT

The AS400/AS400micro dust extractor is based on a concept which has been tried and tested for years and used successfully in many branches. The motor unit of the AS400/AS400micro is designed for extremely quiet running and includes a powerful "Turboflo" motor turbine which is guaranteed to produce exceptionally high suction. The plaster dust is extracted via a two-stage filter system. The first stage involves blasting the plaster dust into a paper dust bag. During the second stage, the removable fibrous filter traps all dust particles remaining in the outgoing air. Additionally, in the AS400micro variant 99.995% of the respirable dusts which may be produced in combination with sandjets and zirconium oxide processing systems are filtered out by an integrated microfilter module. The housing of the AS400/AS400 micro is made of sturdy steel plate and coated with long lasting, resistant epoxy resin.

### INSTALLATION

After unpacking the unit, please check it immediately for possible damage incurred during transport. Should the unit have been damaged during transport, please complain to your supplier immediately.

### AS400 VERSION

The AS400 aspirator is suitable for gypsum dusts.

### AS400 MICRO VERSION

The AS400micro version has an integral **microfilter module**. This high performance filter module retains up to 99.995% of harmful respirable dusts.

**IMPORTANT: When used with zirconium oxide milling devices and blasting cubicles, the AS400micro (with three-stage filter system!) provides optimal protection from zirconium oxide dusts and respirable dusts which conventional filter systems may release again into the workplace air. The AS400micro complies with the required guidelines for the use of aspirators used for zirconium oxide milling devices and blasting cubicles and thus provides optimal protection from harmful respirable dusts.**

### CONNECTING THE ELECTRICITY SUPPLY

Check the voltage. Check that the voltage quote on the rating plate on the back of the housing coincides with your electricity supply.

The dust extractor receives its power from the unit to which it is connected and is operated via the ON/OFF switch on this unit. The power cable of the AS400/AS400micro is fitted with an earthed plug and should be connected to the connecting cable supplied with it. The other end of the connecting cable should be plugged into the back of the unit. If a different type of unit is used, a connecting cable with the correct type of plug is required.

### CONNECTING THE HOSE

Before connecting the suction hose (1.25 m) supplied with the unit, the black plastic part on one end of the hose must be removed. Into the black plastic part (black plastic nut). The plastic nut on the end of the hose should then be wound onto the threaded coupling on the extractor.

### SERVICING/REPLACING THE FILTER

Open the catches on the side of the unit and lift the motor housing upwards. When changing and cleaning the filters, sufficient inhalation protection is mandatory.

#### 1) Replacing the paper filter bag

A paper filter bag should always be used in laboratories as it enables the extractor to be emptied very easily and neatly. It also enhances the suction. Ensure that the correct type of filter is fitted (No. 114953).

#### 2) Cleaning the fibrous filter

Lift the fibrous filter carefully and, preferably in open air, brush it with a household brush until it appears clean. This should be carried out when replacing the filter bag to ensure that the extractor performs optimally.

#### 3) Replacing the fibrous filter

When replacing the fibrous filter, check that the washer seals properly and is not damaged. Should it have been damaged, air may leak in between the motor housing and dust container and prevent the extractor performing optimally. Should this occur, replace the fibrous filter with a new one (No. 114954).

### PLEASE NOTE!

Like all electrical appliances, dust extractors must be operated carefully. This unit must be serviced regularly to ensure that it runs smoothly. Failure to service this unit regularly or replace worn components may

adversely affect its safety. In this case, the manufacturer assumes no liability whatsoever.

#### Technical Data:

D/W/H:	340x340x430mm
Weight:	10,3 kg
Capacity:	12 Liter
Power supply:	230V/50/60Hz
Power uptake:	1100 W
Air throughput:	43,5 l/sek.
Suction pressure:	2450 mm WS
dBA:	51

#### Accessories/single components:

Order No.	
114953	Paper dust bags, pack of 10
114955	Suction hose, 1.5 m
114954	Fibrous filter
114956	Connecting Cable
114957	Additional cotton filter
114952	Microfilter

#### Packing List AS400:

Order No.	
114950	<b>Dust extractor AS400</b>
114953	Paper dust bags, pack of 10
114955	Suction hose, 1.5 m
114954	Fibrous filter
	User manual

#### Packing List AS400micro:

Order No.	
114960	<b>Dust extractor AS400micro</b>
114953	Paper dust bags, pack of 10
114955	Suction hose, 1.5 m
114954	Fibrous filter
	User manual

## ! WARNINGS AND SAFETY ASPECTS:

The AS400/AS400micro was developed for use in dental laboratories. It must be installed and operated as described in the user's manual.

The manufacturer is not liable for damage caused by using the unit for other purposes or handling it incorrectly.

In addition, if the intended application is not listed in the user's manual, the user is obliged to check personally in advance that the unit is suitable for this purpose.

- **Keep the unit clean**
- **Empty the filter bag regularly**
- **Keep the fibrous filter clean**
- **When changing filter systems, sufficient inhalation protection is mandatory**
- **Keep the hoses free**
- **Replace worn parts immediately**
- **Protect the unit against moisture**
- **Always unplug the unit before cleaning it**
- **Do not suck up water or other liquids with this unit**

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Amann Girschbach AG
**Amann Girrbach AG**
**Amann Girrbach GmbH**