
Gebrauchsanweisung
Instructions for use
Mode d'emploi
Istruzioni per l'uso
Instrucciones de empleo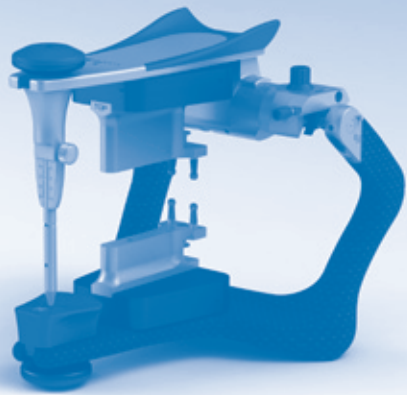

Manufacturer | Hersteller
 Distribution | Vertrieb

Amann Girrbach AG
 Herrschaftswiesen 1
 6842 Koblach | Austria
 Fon +43 5523 62333-0
 Fax +43 5523 55990
 austria@amanngirrbach.com

Distribution | Vertrieb D/A

Amann Girrbach GmbH
 Dürrenweg 40
 75177 Pforzheim | Germany
 Fon +49 7231 957-100
 Fax +49 7231 957-159
 germany@amanngirrbach.com
 www.amanngirrbach.com

www.hellblau.com 32101-FB 2011-11-29

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Original-Gebrauchsanweisung

PRODUKTBESCHREIBUNG

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INSTRUCTIONS FOR USE

Translation of the original instructions for use

PRODUCT DESCRIPTION

Artex iTero model adapter set for mounting a set of models, which were fabricated using iTero scan data, in the Artex Carbon. The upper model is mounted in the adapter marked with TOP (upper). The second (lower) adapter is used for the lower model. Artex iTero model adapters are for use in Artex Carbon articulators that are equipped with the interchangeable Splitex system.

MOUNTING IN THE ARTICULATOR

Mount the adapter set in the Artex articulator using the Splitex plates. Place the adapter marked with TOP in the upper arm of the articulator (cover picture).

MOUNTING THE MODELS ON THE ADAPTERS

Place the upper and lower models on the adapter pins via the retral drill holes in the model and slide the model on until it fits flush on the adapter (Figure 1).

REMOVING THE ADAPTER FROM THE ARTICULATOR

Models can be removed individually from the articulator together with the adapter.

DETACHING THE MODELS FROM THE ADAPTER

Press with the thumbs against the middle of the small ejector on the back of the adapter. This detaches the model from the adapter (Figure 2 and Figure 3).

DELIVERY INCLUDES

1 Model adapter set

GUARANTEE

Application recommendations, no matter whether verbally, in writing, or by means of practical instructions are based on our experience and tests and are thus only a recommendation. Our products are subject to continuous further development. We reserve the right to alter the products in terms of design or composition.

iTero is a Registered Trademark of Cadent, Inc.

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iTero è un marchio registrato di Cadent, Inc.
iTero es una marca registrada de Cadent, Inc.