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INSTRUCTIONS FOR USE ARTEX®MANDIBULAR SET-UP INDEX KEY

Translation of the original instructions for use

Fundament scale for average-value alignment of the edentulous mandible model at the symphysis and retromolar pads.

OPERATION/HANDLING

Close articulator centric. Set the anterior guidance pin out of contact. Evaluate the patient information and enter the same in the Mandibular Set-Up Index Key.

Adjusting the frontal height

Measure the distance of the symphysis pad to the lip closure line marked on the wax wall (18 mm in the example). **(Fig. 1)**

Sagittal alignment

The transverse crosspieces must be adjusted in a sagittal direction (with both slopes positioned dorsally) so that the upper edge of the crosspiece lies in the middle of the retromolar triangle (model analysis). Mandibular model connected with an elastic band to the Set-Up Index Key and placed in the upper portion of the articulator. **(Fig. 2)**

Turn the articulator upside down. Then place this on the moistened surface of the mandibular model and onto the cast plaster of the base plate (creamy). Close articulator. Remove the plaster by tapping with a hammer. **(Fig. 3)**

Maxillary model based on the wax impression made. Make a plaster cast as previously described. **(Fig. 4)**

Parts and optional accessories:

217700C	Set-Up Index Key mand. 126mm C-Version
216250C	Set-Up Index Key mand. Splitex 126mm C-version

INSTRUCTIONS FOR USE ARTEX®SET-UP TEMPLATES

Translation of the original instructions for use

Guide for set-up of full dentures according to the Spee/Wilson occlusion curve. The horizontally and vertically adjustable device with occlusal plane indicator supports the different templates.

OPERATION/HANDLING

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Maxillary and mandibular models are placed in the articulator. Set-up teeth 41 and 31. The heights of the incisal edges are taken using the bite fork (measure). Anterior teeth 31 and 41 are tilted protrusively, with the incisal edges oriented to the maxillary gingivobuccal fold! Adjust the base surface of the teeth to the middle ridge of the jaw.

Then remove the maxillary model. Insert the set-up template with the adjustment plate. Insert template frontally on the incisal edges of teeth 31 and 41, adjust to align the upper third of the retromolar triangle (model analysis). Attach template.

Adjust the maxillary form to teeth 33 and 43. The incisal edges of the teeth are in contact with the template. Take care to note the varying tilt of the individual teeth. **(Figs. 5 + 6)**

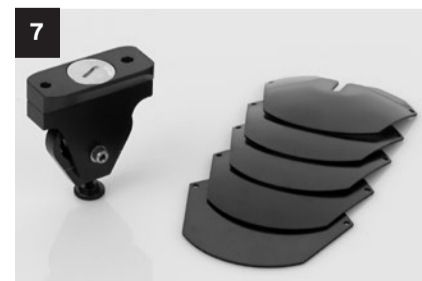
Adjustment of the lateral teeth.

Remove template. Select a bite plate to suit the type of teeth (see Table) and insert it into the bite fork. The bite plate is in contact with the frontal and incisal edges of teeth 33 and 43, and is positioned dorsally along the upper third of the retromolar triangle. Attach template. Set up the lateral teeth according to the model analysis or adjust aesthetically. All retromolar pads, aside from the lingual pads of teeth 34 and 44, are in contact with the template. **(Figs. 7 + 8)**

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