
Bedienungsanleitung
User manual
Mode d'emploi
Istruzioni d'uso
Modo de empleo

Deutsch 04 - 05
English 06 - 07
Française 08 - 09
Italiano 10 - 11
Español 12 - 13

DE

DE

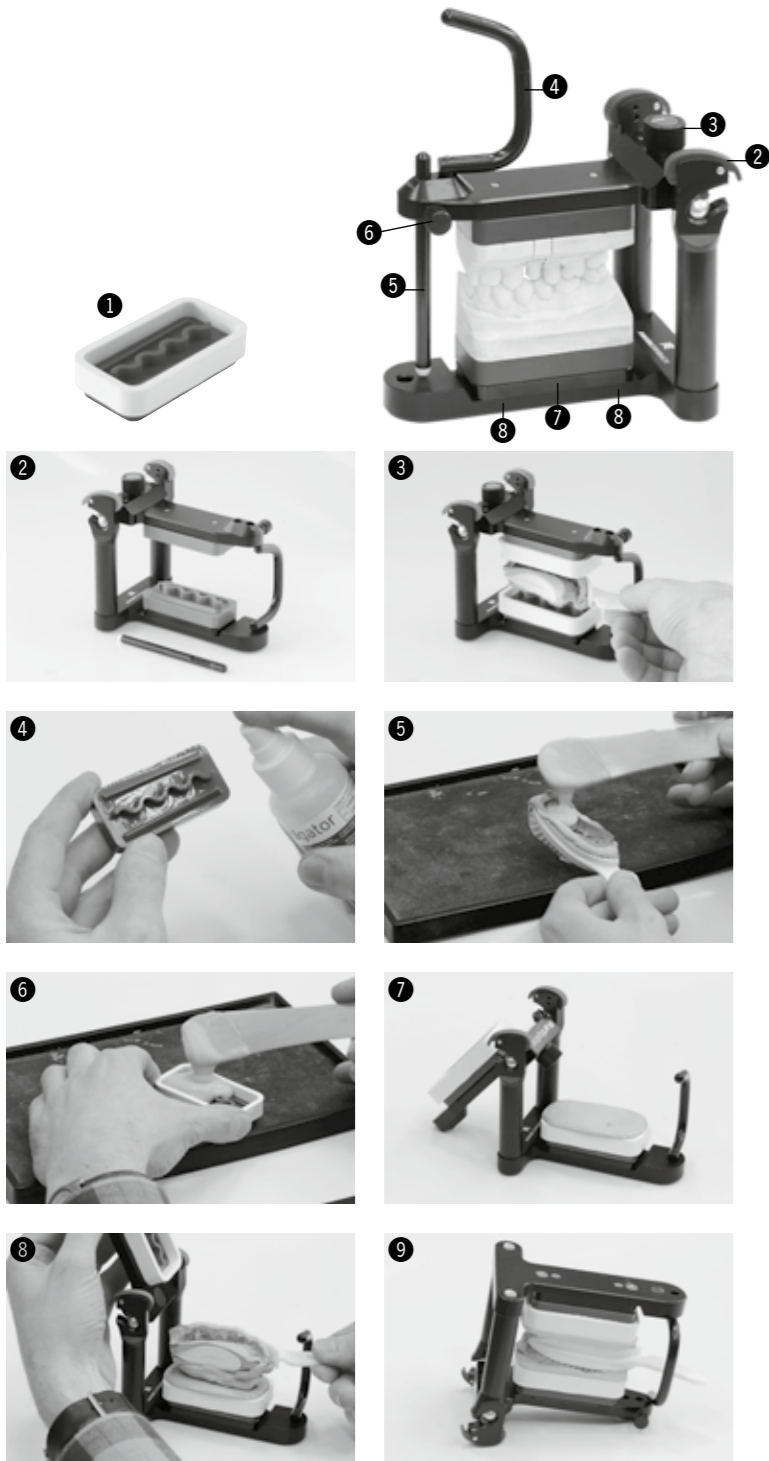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DESCRIPTION OF THE UNIT

- 1 Model plate with sleeves
- 2 Centric quick lock
- 3 Sideshift adjusting screw
- 4 Incisal pin
- 5 Incisal guidance pin
- 6 Incisal guidance pin locking screw
- 7 Magnet
- 8 Lower adjusting plate locking screws

DESCRIPTION OF THE PRODUCT

The Artigator comprises 3 parts for articulating dual arch impressions and fabricating a precise sectioned working model in one single process. The Artigator concentrates on functioning efficiently and straight-forward, timesaving handling. The centric quick lock and sideshift function enable the centre position to be reproduced exactly. Extra powerful magnets in the upper and lower members retain the pin plates. All components are manufactured and assembled highly precisely. The Artigator is robust, will not distort and is guaranteed to be long lasting.

**HANDLING
CENTRIC QUICK LOCK**

- a) Opening the centric quick lock
Place your thumb on the grooved area and press the lock upwards. The head of the condyle is then loose in the condylar path and the upper member of the Artigator can be moved backwards. The lock prevents the upper member from falling out.
- b) Removing the upper member of the Artigator
Open the centric quick lock, push the upper member back fully and hold it in this position. Use your other hand to press the lock down (close the centric). The upper member of the Artigator can then be removed backwards.

PROCEDURE

Replace the incisal guidance pin with the incisal pin (2). Lock the sideshift adjuster and position the impression correctly on the Artigator where possible, the space between the tray and Artigator should be the same all round. The pin plate sleeves are easily positioned properly (3). If the space between the pin plate and Artigator is too narrow, carefully trim any excess material off the impression with a scalpel. Remove the pin plates from the Artigator and spray them and the sleeves with the separating agent supplied. (4) Coat them thoroughly with separating agent and allow it to take effect. Then blow the surface dry. Cast the impression as usual with type 4 high strength dental stone (5). Fill the first pin plate with a sufficient amount of stone vibrating it gently (6). Replace the pin plate on the Artigator and wait until the stone is slightly viscous (7). Locate the dual arch impression tray correctly on the pin plate of the Artigator (8). Ensure that a small amount of stone remains on the edge of the pin plate sleeve (app. 8 mm height) so that later on the dental arch releases easily from the pin plate. Allow the stone to set and cast the opposite side using the same procedure. Wait until this stone is also viscous and close the Artigator. It is advisable to turn the Artigator upside down on the bench top for this procedure (9). Once the stone has set fully, remove the impression from the Artigator and release it carefully (10). To facilitate releasing the model from the model pin, place it (pin plate upwards) in the sleeve and tap it gently on the bench top (11 - 14). Trim the dental arch carefully (except the bottom) and section as required with a saw (15 - 16). Complete casting the model as shown in pictures (17 - 19).

Important:
Adhere strictly to the stone manufacturer's mixing ratios to ensure that the optimum expansion values are attained (to achieve optimum friction between the stone and pin plate). Epoxy resins must not be used.

MAINTENANCE AND CLEANING

The Artigator is maintenance-free. As the Artigator is a high precision unit, it must only be steam-cleaned (e.g. with an AmannGirrbach-Steamer). It should be cleaned as gently as possible and without using solvents. The Artigator will function long term without the use of lubricant. Do not allow it to contact powerful acids or clean it in an ultrasonic unit.

GUARANTEE

Thank you for purchasing this high quality Artigator. It was developed and manufactured in accordance with the highest quality standards.

We therefore guarantee the workmanship and perfect condition of the Artigator for 12 months. This guarantee does not cover expendable components.

If an inspection of the Artigator indicates that it has been tampered with, operated incorrectly or the instructions not adhered to, the guarantee is void. Damages are limited strictly to free repair or replacement of the unit. We expressly disclaim liability for incidental and consequential damages.

! WARNINGS AND SAFETY INFORMATION:

The Artigator was developed for use in dental practices and laboratories. It must be installed and operated as described in the instructions. The manufacturer accepts no responsibility for damage which occurs if used for other purposes or if not used according to the instructions.

In addition, if the intended application is not listed in the user's manual, the user is obliged to check personally in advance that the unit is suitable for this purpose.

Warning:
For use in dental practices by qualified, trained and skilled persons only.

Technical Data:

Weight:	0,5 kg
Condyle radius:	17,5 mm
Inclination of condylar guidance:	30°
Interior height:	66 mm
Bennett angle:	12°

Accessories / Components:

Order no.:	
218941	Base plate blue, 50 pcs.
218933	Sleeves
218948	Separator spray, 100ml
218949	Separator spray, 1l (for refill of spray bottle)

Package contents:

Order no.:	
218950	Artigator incl.
218941	Base plate blue, 2 pcs.
218933	Sleeves, 3 pcs.
218936	1 incisal guidance pin
218935	1 incisal pin
	User Manual

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Amann Girrbach AG