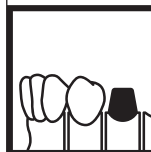
 **electronicsteamer II**

Modellherstellung
Model fabrication
Fabrication du modèle
Sviluppo dei modelli
Elaboración de modelos

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User manual

Mode d'emploi

Istruzioni d'uso

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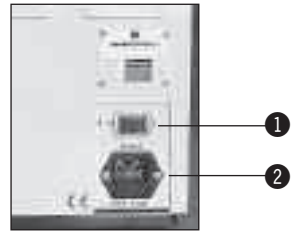
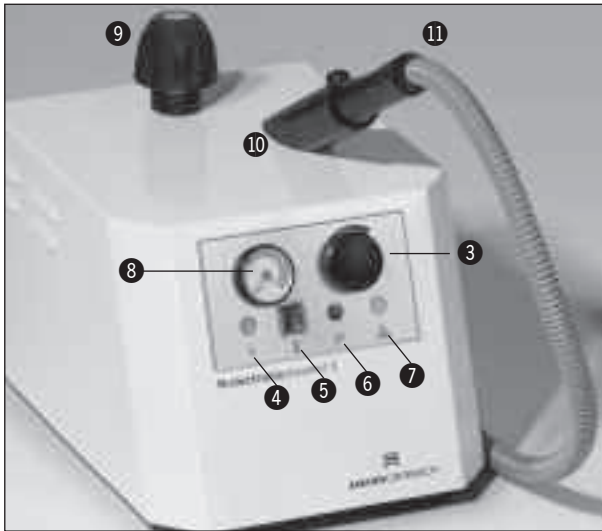
AMANNGIRRBACH

Deutsch	04 - 05
English	06 - 07
Française	08 - 09
Italiano	10 - 11
Español	12 - 13
EU-Declaration of conformity	14 - 15

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USER MANUAL

- 1 ON/OFF switch
- 2 Socket for power supply lead
- 3 Steam pressure regulator
- 4 GREEN lamp: "Power ON"
- 5 WHITE lamp: "READY"
- 6 RED lamp: "Water tank EMPTY"
- 7 ORANGE lamp: "DESCALE"
- 8 Pressure gauge
- 9 Safety cap and filler neck
- 10 Holder for steam head
- 11 Steam head
- 12 Drainage screw
- 13 Descale reset switch

PRODUCT DESCRIPTION

The Electronic Steamer II is a high-quality, universal steam cleaner for the following applications:

- _Cleaning all models, plaster and epoxy dies, crown and bridge frames, a wide range of small objects, ceramics or prosthetic plastics
- _Removing aluminium oxide residue
- _Removing wax residue
- _Cleaning articulators
- _Cleaning dental tools and instruments
- _Cleaning jewellery

The Electronic Steamer II has an integrated stainless steel water tank with a capacity of approx. 4.2 litres. A warning lamp plus an acoustic signal are activated when the minimum water level is reached. If these warning signals nevertheless go unheeded then the heater will switch off automatically. A filler neck and relief valve ensure maximum safety when water is refilled.

The automatic scale monitor tells the user in good time that the unit needs descaling, thus avoiding the need for expensive repairs. Stepless steam pressure regulation and extremely fine steam jet adjustment make the Electronic Steamer II exceptionally convenient to use. The end of the ergonomically shaped handpiece is fitted with a curved nozzle head.

USING THE STEAMER FOR THE FIRST TIME

Unpack the steamer and check to ensure that it has not been damaged in transit. Report any damage to the supplier immediately.

All our equipment leaves the factory ready to use. Before the steamer can be switched on, the tank must be filled with water and power supply lead (2) connected as instructed below.

Do not fill the tank with chemicals or other additives.

Check the specified voltage. Please ensure that your mains voltage supply matches the voltage given on the nameplate at the back of the device. Switch the power switch (1) to the zero (i.e. "OFF") position, then plug in the power supply lead. Unscrew the safety cap (9) on the filler neck. The steam boiler can now be filled using the funnel supplied. Ensure that the water does not overflow.

IMPORTANT: Screw the safety cap back onto the filler neck, making sure the cap is tightened up to its end stop. Also check that the drainage screw (12) is properly tightened. Switch on the device by activating the power switch (1). The steamer will need approximately 12 minutes before it is ready for operation. This will be indicated by the green control lamp (4) and the pressure gauge (8). The pressure gauge needle must be in the green area.

HANDLING

As soon as the pressure gauge needle is pointing to green, the device is ready for use. Take hold of the handpiece to remove the steam head (11) from its holder (10). Hold the steam head over a wash basin. Activate the steam switch to clean therequired objects. In the case of small objects, we recommend the use of a sieve (available as accessory). When you have completed the steaming operation, replace the steam head into its holder. To increase or decrease steam pressure, turn the rotary switch (3) as required.

WARNING RISK OF INJURY!

The steamer is under high pressure when switched on. The filler neck (9) or drainage screw (12) should never be opened under these circumstances. Never direct the steam jet at people or animals - risk of scalding.

Always ensure that the power is switched off (1) and there is no pressure left in the device before opening the safety cap on the filler neck or the drainage screw.

IMPORTANT

The safety cap (9) is fitted with a pressure relief valve which opens automatically if a pressure of 6 bar is reached. Normally, however, the electrically controlled and monitored heater will cut out before the pressure gauge drops into the red area. If the pressure gauge needle nevertheless shows red, switch off the steamer and disconnect it from the mains supply. If this happens, the device should be returned to the manufacturer for inspection as a safety precaution.

MAINTENANCE

The Electronic Steamer II is basically maintenance-free. However, it must be descaled at regular intervals. A control lamp (7) and acoustic signal will tell you that the device needs descaling after about 50 hours of operation.

ATTENTION

The user manual for decalcifying is added to the delimer. Do not switch on the steamer or use the steam head during the descaling process. Once the descaling and flushing operations have been completed, ensure that the drainage screw (12) on the underside of the device is screwed tightly shut. Now press the reset button (13) on the underside of the device. This will reset the scale monitor.

Do not use cleaning products containing solvents to clean the outer surfaces of the steamer.

Technical Data:

L/B/H:	360/270/290 mm
Weight:	9 kg
Mains supply:	230V 50Hz
E-fuse:	10 A inert
Power:	1430 W
Steam pressure:	4 bar
Boiler capacity:	4.2 l
Heating up time:	approx. 20 min

Accessories/single components:

Order No.	
116814	Sieve, small, dia. 50 mm
116815	Sieve, large, dia. 75 mm
116813	Funnel
516330	Clean Steamer
516340	Stymo Net
116811	Kalk-X Liquid
116835	Lid
116340	Sealing kit for the lid

1168432	O-ring set for drainage screw
1168391	O-ring set for nozzle (HS)

Delivery scope:

Order No.	
116810	Electronic Steamer, Funnel

! WARNINGS AND SAFETY ASPECTS:

The Electronic Steamer II has been developed for use in dental applications. The device must be operated and used in accordance with the manufacturer's instructions.

The manufacturer shall accept no liability whatsoever for damage resulting from inappropriate use or improper handling.

In the event that the intended applications are not mentioned in the manufacturer's instructions, the user shall be responsible for establishing the suitability of the device prior to its use.

- **Do not steam clean any live parts or equipment.**
- **Use tap water or distilled water only.**
- **When the power is switched on, the device is under pressure. Never open the filler neck or drainage screw under these circumstances.**
- **Never direct the steam jet at people or animals.**
- **Always ensure that the power switch is OFF and there is no pressure left in the steamer before opening either the safety cap on the filler neck or the drainage screw.**
- **Never grasp the nozzle head at the end of the handpiece when the steamer is ready for operation - risk of burning.**

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hereby declare that the product described below, both, in its basic design and construction and in the version marketed by us is in conformity with the relevant safety and health related requirements of the appropriate EU Directive. This declaration shall cease to be valid if modifications are made to the product without our approval.

Product:

Electronic Steamer II
Steam cleaner

Number:

116810

Applicable EU Directives:

98/37/EU - Machinery guidelines
EU Low-Voltage Equipment Directive
(73/23/EU)
EU Directive on Electromagnetic
Compatibility (89/336/EU)

Applicable

harmonized standards:

DIN EN ISO 12100-1
DIN EN ISO 12100-2
DIN EN 55014-1/2
DIN EN 61010-1

Applicable national standards:

-

It is ensured through internal measures that products will always meet the requirements of current EU Directives and relevant standards.

Pforzheim, 20.01.2005

by order Dr.-Ing. S. Cramer von Clausbruch

The signatory acts on behalf of company management and with full power of attorney.

Manufacturer | Hersteller
Distribution | Vertrieb

Amann Girrbach AG
Herrschaftswiesen 1
6842 Koblach | Austria
Fon +43 5523 62333-0
Fax +43 5523 55990

Distribution | Vertrieb D/A

Amann Girrbach GmbH
Dürrenweg 40
75177 Pforzheim | Germany
Fon +49 7231 957-100
Fax +49 7231 957-159

austria@amanngirrbach.com
germany@amanngirrbach.com

www.amanngirrbach.com

Made in the European Union

AMANNGIRRBACH