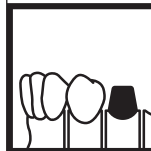
 **masterbox**

Modellherstellung
Model fabrication
Fabrication du modèle
Sviluppo dei modelli
Elaboración de modelos

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User manual

Mode d'emploi

Istruzioni d'uso

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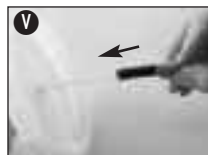
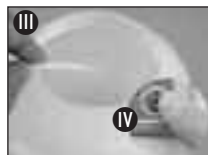
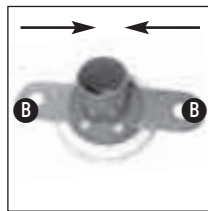
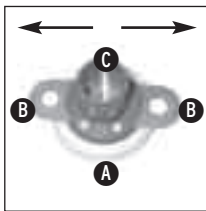
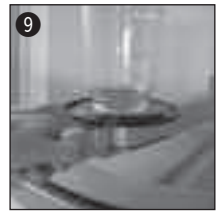
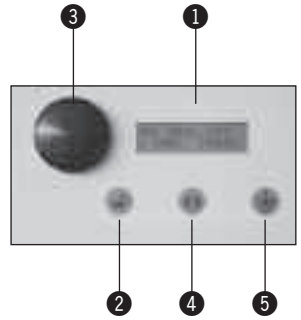
AMANNGIRSBACH

Deutsch	04 - 09
English	10 - 15
Française	16 - 21
Italiano	22 - 27
Español	28 - 33
EU-Declaration of conformity	34 - 35

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USER MANUAL

- ① LC display
- ② Enter key
- ③ Multifunctional rotary knob
- ④ Weighing machine key
- ⑤ Start/Stop" key
- ⑥ Plaster Container
- ⑦ Plaster Container lid
- ⑧ Water tank
- ⑨ Swivel arm
- ⑩ Protective cover for scales
- ⑪ Removable plaster bin

DESCRIPTION OF THE PRODUCT

The Masterbox is the first unit to use programmable software for dispensing water and plaster. All types of dental plaster can be used. This unit is constructed robustly and includes a conveyor worm on a special vibrator for dispensing the selected/saved amount of plaster accurately to within a gramme. According to experts, the water should be dispensed from the tank in the stand before dispensing the plaster. As 20 programmes can be saved, almost any desired comcontaineration of plaster/water can be selected/dispensed in the laboratory by simply pressing a key. The Masterbox is equipped with a high grade electronic control system which can save 20 dispensing programmes. Each dispensing programme includes the following settings:

1) Amount of plaster

The amount of plaster can be adjusted progressively in 1 gramme steps.

2) Plaster/water ratio

The plaster/water ratio is defined in 1 ml steps per 100 g of plaster and automatically converted to the required amount of plaster.

3) Bowl check on/off

While the bowl check is "on", nothing can be dispensed until a bowl is placed on the weighing platform.

This function also checks that the bowl is large enough for the amount of plaster selected. When the bowl check is "off", dispensing can be commenced right away. The standard settings include all the amounts required to fill AmannGirrbach mixing bowls (Smartmix vacuum mixer). To programme other permissible amounts for other bowls, refer to "Language and bowl check".

4) Programme identification

This is where the programme name is entered. The name may consist of up to 12 letters.

All settings are entered with the multifunctional rotary knob. The Masterbox was designed to be as ergonomic and straightforward as possible. This can be seen in the Masterbox controls which only consist of the multifunctional knob and 3 keys.

INSTALLATION

After unpacking the unit, please check it immediately for possible damage incurred during transport. Should the unit have been damaged during transport, please complain to your supplier immediately. Although this unit is supplied ready for use, always check the voltage setting (115/230V) immediately above the power socket. The unit can be used with a mains voltage of 100, 115 or 230 V. Check the factory voltage setting of your unit at the window of the mains socket before operating the unit. If the factory setting is not the same as the voltage in your area, remove the fuse holder from the mains socket and place the fuse in the holder so that the voltage you require is visible in the window. Then reinsert the fuse holder properly into the mains socket and connect the unit to the mains by means of the mains cable. If you are not certain which voltage is used in your country, please ask your local supplier of electrical appliances.

Caution!

Do not lift the unit by the swivel arm or scales. Risk of damage!

Ensure the unit is installed where there is adequate load-bearing capacity! (When using 10 kg containers filled to maximum, the total weight of the Masterbox is approx. 95 kg!)

FILLING THE WATER TANK (III - VI)

To fill the water tank, unscrew the plastic lid. After filling, screw the lid on again and place the water tank on top of the unit at the suction hose and insert the hose through the opening in the lid. Filling the plaster containers.

Remove the white lid from the plaster container. The opening in the container is relatively large to enable it to be filled easily. Once the plaster container is full, screw on the lid supplied with it **(A)** clockwise and tighten it. Switch off the equipment to remove the plaster container.

Caution!

Ensure that the two sliding covers **(B)** have been closed completely! The closed plaster container can then be inverted and fitted to the Masterbox. Ensure that the plaster dispensing nozzle **(C)** slips into the socket smoothly **(D)**.

Important

Then pull the sliding covers apart **(B)**. The sliding covers must be parallel to the top of the unit. The unit is then ready for use.

CHANGING THE PLASTER CONTAINER

If the plaster container is changed for dispensing a different type of plaster, two points must be taken into account. Ensure that the two sliding covers **(B)** have been closed completely. This prevents the plaster spilling out when the container is removed. Hold a bowl beneath the dispenser and activate "Dispensing plaster" (refer to the following section) to empty the remaining plaster out of the worm conveyor. The plaster container can then be removed from the unit.

SWIVEL ARM (9)

The swivel arm is used to select the plaster silo required (silo 1= left; silo 2= middle, silo 3 = right). When the arm reaches the correct position under the silo, a green LED lights up and the setting automatically changes to the dispensing programme for the silo selected. Up to 20 dispensing programmes can be preset for each silo. We recommend using the protective cover to keep the scales clean.

HANDLING

The two line LCD **(1)**, "Enter" key **(2)**, multifunctional rotary knob **(3)**, "Weighing" key **(4)** and "Start/Stop" key **(5)** on the front of the Masterbox control all its functions. This unit can be used immediately after switching it on. The display indicates the programme used last. Example: (Schematic representation of

the display) an arrow points to the programme number, programme name, amount of plaster and ml/100g plaster).

OPERATING THE WEIGHING MACHINE

The unit must be in standard mode to operate the weighing machine (LCD 1) which is activated by pressing the "Weight" key **(4)**. Any item of up to 2.5 kg can be weighed. If an item is already on the weighing platform, the "Weight" key **(4)** can be pressed again to reset the tare to zero.

DISPENSING WATER AND PLASTER MANUALLY - REMOVING REMAINING PLASTER

To dispense water and plaster manually, use the multifunctional rotary knob to skip programmes 1 - 20 in standard mode. After programme 20, the unit automatically moves to "Dispensing plaster" in the menu. When the "Start/Stop" key **(5)** is pressed, the unit begins dispensing plaster manually. This procedure is also used for emptying out the plaster when changing the plaster container (see above). Use the same procedure for dispensing water manually. Programme 22 describes "Dispensing water" in the menu.

OPTION

Dispensing water or plaster manually - including weighing:

Pressing the weighing machine key **(4)** during programmes 21 or 22 activates the weighing machine - the Start/Stop key **(5)** retains its manual plaster/water dispensing function. This allows the weighing machine to be used manually.

LANGUAGE CHECK CONFIGURATION, DEFINING THE BOWL WEIGHT AND STANDARDIZING PROGRAMMES

The Masterbox menu lists five languages (D, GB, I, F, E). If the unit is not set to your preferred language, switch it off again. Press the "Enter" key **(2)** and "Start/Stop" key **(5)** simultaneously and switch the unit on again. The unit is then in default mode. Turn the "Multifunctional" knob **(3)** to select the required language and press the "Enter" key. Once the language has been set, you will be in menu 1 where the following can be set up (select with the multifunctional knob):

CONTAINER CONFIGURATION

Presets all as yet unchanged settings for the entire system (water amount, bowl check, programme name). If „Save Yes/No“ is answered with „Yes“, all settings which have remained unchanged so far will be stored in programmes 1-20 with the values defined at this stage.

BOWL WEIGHTS

These values define the permissible weights. The weights of the AmannGirrbach mixing bowls are saved in advance. They can be overwritten with 4 custom bowl weights.

SETTING CUSTOM BOWL WEIGHTS

_Press the "Weight" key **(4)** in standard mode and weigh your mixing bowl separately. Note the weight of your mixing bowl.

_Activate the default mode. (Press the "Start/Stop" and "Enter" keys simultaneously, hold them down and switch the unit on). Press the "Enter" key **(2)** to save the desired language. Then use the multifunctional knob **(3)** to select „Bowl weights“ in the menu.

_The LCD then indicates the bowl name "250 ml" on the left side and the preprogrammed bowl weight (AmannGirrbach Smartmix bowl) in grammes on the right.

_The multifunctional knob can then be turned to adjust the preprogrammed bowl weight to the weight of your own bowl in 5 gramme stages.

_Press the "Enter" key **(2)** to proceed automatically to the second of the 4 programmable bowl weights. (250 ml, 500 ml, 750 ml and 1,000 ml corresponding to small, medium, large and extra large bowl sizes). Turn the multifunctional knob to adjust the unit to the weight of your second bowl and press the "Enter" key again **(2)**.

_The weights of the two remaining bowls can be saved using the same procedure. If no further bowl weights need to be saved, simply skip them by pressing the "Enter" key **(2)**.

_Once the last bowl weight has been saved („1000 ml“) you will be asked „Save Yes/No“. If "Yes" is selected the bowl check will only respond to the bowl weights saved by you.

_Standardizing programmes. The plaster amount is set to 500 g for all programmes (1-20) as are the water amount, bowl check and programme name settings entered in the menu under "Container configuration".

Caution!

Prior to answering the confirmation enquiry (save Yes/No), make certain that you really want to standardize all programmes! Any custom settings saved previously will be overwritten!

_Quitting configuration

Confirm this menu position to revert to standard mode.

SAVING DISPENSING PROGRAMMES

The Masterbox is programmed at three levels. The "Enter" key **(2)** switches it between the different levels. Each setting is adjusted with the multifunctional rotary knob **(3)**.

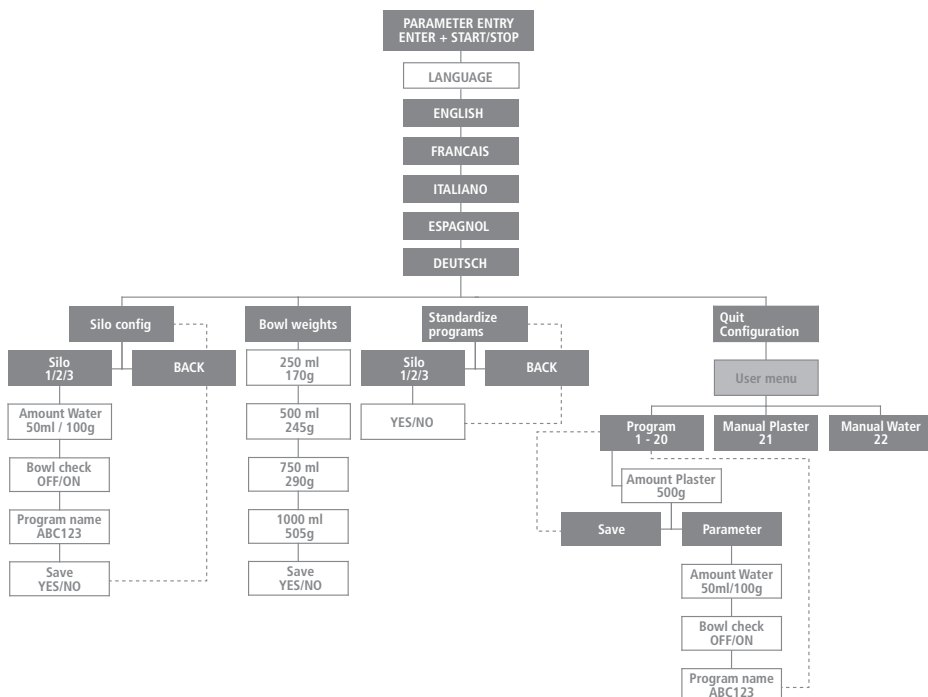
Menu

- 1 Turn the multifunctional rotary knob **(3)**, referred to hereafter as the MRK **(3)** to the next free storage location or that setting to be adjusted.
- 2 Press the "Enter" key **(2)** to reach the second level. Turn the MRK **(3)** to adjust the amount of plaster in grammes.
- 3 Press the "Enter" key **(2)** to save the settings.
- 4 If you would like to use the default settings for your dispensing programmes (settings in container configuration menu position), press the Enter key **(2)** again. If you would like to change the settings in your dispensing programme, move the option arrow by turning the MRK **(3)** to „Settings“ and press the Enter key **(2)**.

5 You are now on the third level. Turn the MRK **(3)** until the desired amount of water per 100g of plaster has been set and press the "Enter" key **(2)** to save the setting.

6 The bowl check can then be switched on or off. Press the Enter key **(2)** to confirm.

7 The programme name can be saved in the last field. To do so, turn the MRK **(3)** until the correct letter appears. Save the letter by pressing the "Enter" key **(2)** and the cursor automatically moves to the next position. Names with up to 12 letters can be saved. If you do not require all 12 symbols, fill the remaining spaces with blanks. Once the last symbol has been entered, the programme is saved and you can begin dispensing.

PLEASE NOTE

The settings always refer to 1 programme, i.e. you can save up to 20 custom programmes.

DISPENSING

Select the correct size of mixing bowl for the required amount of water and plaster. If the "bowl check" is activated, the correct bowl must be used for dispensing (refer to the "bowl check" setting). The MRK **(3)** is turned to select programmes 1 (P01) to 20 (P20).

PLEASE NOTE

Once a dispensing programme has been saved, the amounts are easily changed. Press the "Enter" key **(2)** and turn the MRK **(3)** to set the desired amount of plaster. Press the "Enter" key **(2)** twice to save the new amount to be dispensed. Press the "Start/Stop" key **(5)** to commence dispensing and ensure that the mixing bowl is in the centre of the weighing platform. The water is dispensed before the plaster (as recommended by experts). The amounts of water (ml) and plaster (g) are counted on the LCD **(1)** during dispensing. Once they have been dispensed completely, the total amounts (dispensing programme) are shown on the LCD **(1)** and a beeper can be heard.

SERVICING

Like most AmannGirrbach equipment, the Masterbox is designed to be maintenance-free and includes a maintenance-free vibrator and dispenser. Simply keep the unit clean. Liquids must also be kept away from the conveyor worm.

Never clean the Masterbox with compressed air or a steam cleaner! If the Masterbox has to be transported, the feed screw must be cleaned completely with a vacuum cleaner. The Masterbox must be transported in its original packaging and with its plaster inlet and outlet nozzles sealed. If the original packaging is in very poor condition or missing, please contact your dental dealer.

POSSIBLE FAILURE REPORTS

Container fault:

The container has not been fully inserted into the plaster inlet! The sliding covers must be parallel to the operating panel!

Defective reed contact! -> Please contact an authorized repair centre!

Plaster worm conveyor overloaded:

If the worm conveyor jams permanently, after a few seconds „worm conveyor overloaded“ will be displayed and the unit will automatically cease conveying the plaster!

Press any key to transfer to programme 21, for dispensing plaster manually, where the Enter key can be pressed briefly together with the Start/Stop key to reverse the worm conveyor.

CAUTION!

The worm conveyor must only be reversed briefly and the cause of the fault eliminated!

! WARNINGS AND SAFETY ASPECTS

The Masterbox was developed for use in dental laboratories. It must be installed and operated as described in the user's manual.

The manufacturer is not liable for damage caused by using the unit for other purposes or handling it incorrectly.

In addition, if the intended application is not listed in the user's manual, the user is obliged to check personally in advance that the unit is suitable for this purpose.

- **Do not clean the Masterbox with liquids containing solvent!**
- **Keep the container lid and plaster dispensing nozzle clean and dry!**
- **Keep the weighing machine clean and dry!**
- **Only fill the container with dental plaster and never with investment or other materials!**
- **For safety reasons, this unit can only be operated with the plaster container positioned correctly. (Sliding covers positioned parallel to the top of the unit - horizontally)**
- **Never grasp into the plaster dispensing nozzle while the unit is switched on.**
- **Always unplug the unit when changing the plaster container!**
- **Never clean the Masterbox with compressed air!**
- **Always clean the feed screw fully before transporting the Masterbox!**
- **Never expose the Masterbox to increased atmospheric humidity (e.g. steam cleaner, ultrasonic cleaner or wash basin)!**
- **Never run the Masterbox water pump without water.**
- **Only fill the water tank with pure water, with no added chemicals!**

Technical data:

D/W/H:	700 x 460 x 800 mm
Weight:	ca. 56 kg
Power supply:	110/115/230V 50/60Hz
Fuse:	T2A
Power uptake:	80 W
Max. amount dispensed:	1,000 g
Min. amount dispensed:	50 g
Water tank:	Approx. 5 l

Accessories and single components:

Art.Nr.:	
116160	Masterbox, complete
116125	Plaster container with lid, 5 kg
116120	Plaster container with lid, 10 kg
116130	Plaster container lid

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Amann Girrbach AG
Amann Girrbach GmbH