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**User Manual**

**Mode d'emploi**

**Istruzioni d'uso**

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Deutsch	04 - 06
English	07 - 09
Française	10 - 12
Italiano	13 - 15
Español	16 - 18
EG-Declaration of conformity	20 - 21

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## USER MANUAL

- ① ON/OFF switch vacuum
- ② Vacuum gauge
- ③ Timer knob
- ④ Paddles Start key

## PRODUCT DESCRIPTION

The Minimix vacuum mixer mixes all types of plaster, investment material and silicones extremely well. This unit is built robustly. The powerful vacuum pump builds up a high level of vacuum very rapidly, which holds the mixing vessel in place automatically so that no extra catch is needed.

Although this unit is very easily operated, it still meets all the basic requirements demanded of a modern vacuum mixer.

The integral electronic timer is used for setting the mixing time and starting mixing - a beeper indicates that mixing has been completed. The vacuum holds the mixing vessel in place and is maintained until the mixer is switched off - this prevents air bubbles forming in the material after mixing and produces a homogeneous mix for perfect processing.

5 different sizes of mixing vessel are available, which helps economize on the amount of material mixed. The standard version of this unit is wall-mounted. A firm stand is available as an optional extra.

### ■ INSTALLATION

After unpacking the unit, please check it immediately for possible damage incurred during transit. Should the unit have been damaged during transit, please complain to your supplier immediately.

Although this unit is supplied ready for use, always check the voltage setting (100/115/230V) immediately above the power socket. If you are not certain which voltage is used in your country, please ask your local supplier of electrical appliances.

### ■ FITTING TO THE STAND

The optional stand consists of a firm, heavy plate with back plate. The stand is screwed to the bottom of the Minimix using the screws supplied with it. The mixer is then ready for use.

### ■ FITTING TO A WALL

A template is supplied with the unit for drilling the holes in the wall quickly and exactly. Please ensure that the holes are aligned exactly horizontally (check with a spirit level if necessary). Press the rawplugs into the holes. Screw in the screws to within 5-6 mm before hanging up the unit.

**Caution: When fixing the mixer to plasterboard walls, special rawplugs must be used to support its weight!**

### ■ HANDLING

The Minimix is switched on with the On/off switch on the bottom of the unit. This mixer has an On/Off switch for vacuum (1), vacuum gauge (2) and timer (3) on its front panel. The mixing paddles are operated with the Start button (4) on the timer.

### ■ SETTING THE MIXING TIME

The desired mixing time is set with the mixing time button. It remains stored until a new mixing time is programmed. **(A)**

### ■ MIXING

Select the correct size of mixing vessel and prepare the required amount of plaster or investment material. Mix gently by hand, wipe the top edge of the mixing vessel clean and put the lid on.

**Please note: Never start mixing without having a mixing bowl in position!**

Press the button (1) to switch the vacuum pump on and place the mixing vessel on the suction nozzle from beneath. **(B)**

After only a few seconds, the vacuum holds the mixing vessel on the suction nozzle. Wait until the vacuum gauge is in the green zone and press the Start button (4). A beeper sounds once mixing has been completed. The vacuum cannot be switched

off until the mixing time has elapsed!

During mixing, the vacuum pump sucks the mixing vessel onto the mixer. Press the ON/OFF button with one hand to deactivate the vacuum and hold the mixing vessel with the other hand as it will be released from the suction nozzle when the vacuum drops. Once mixing has been completed, the timer indicates the previous mixing time. This time can either be left for mixing new material or reset.

## ■ MAINTENANCE

Like most AmannGirrbach equipment, the Minimix is maintenance-free, including the vacuum pump. It simply has to be kept clean. The sealing surface of the suction nozzle and the lid must be kept especially clean to ensure that optimum vacuum is achieved. The seal can be kept supple by coating it with Vaseline or similar grease from time to time. An easily accessible preliminary filter and the filter inside the mixer guarantee that it can be operated maintenance-free.

## ■ REPLACING THE (PRELIMINARY) FILTER

The (preliminary) filter only has to be replaced if it becomes blocked or contaminated due to being used incorrectly.

Carefully pull the filter holder out of the suction nozzle. **(C)** Remove the contaminated filter from the filter holder and replace it with the spare filter enclosed. Press the filter holder back into its original position. **(D)**

**Technical data:**

L/B/H:	200 x 180 x 260 mm
Weight:	10,5 kg
Power supply:	115/230 V, 50/60 Hz
Motor speed agitator:	350 min <sup>-1</sup>
Delivery rate:	15 l/min
Vacuum power:	200 mbar

**Accessories/Components:**

Order No.:	
115680	Minimix stand
115620	Beaker with mixing blade 100 ml
115630	Beaker with mixing blade 250 ml
115640	Beaker with mixing blade 500 ml
115650	Beaker with mixing blade 750 ml
115660	Beaker with mixing blade 1000 ml
115631	Beaker 250 ml
115641	Beaker 500 ml
115651	Beaker 750 ml
115661	Beaker 1000 ml
115606	Spare filter, 5 pcs.

**Packing list:**

Order No.:	
115690	wall mounting 230 V (110/115V = 115670V115) Minimix standard unit, excl. mixing vessel, wall-mounted
	1 Spare filter
	1 Users' manual
	1 Mains cable
	1 Drilling template for wallmounting
	4 Screws and rawp lugs for wallmounting

**WARNINGS AND SAFETY ASPECTS:**

The Minimix was developed for use in dental laboratories. It must be installed and operated as described in the user's manual. The manufacturer is not liable for damage caused by using the unit for other purposes or handling it incorrectly.

In addition, if the intended application is not listed in the user's manual, the user is obliged to check personally in advance that the unit is suitable for this purpose.

- \_ **Do not clean the Minimix with liquids containing solvent!**
- \_ **Keep the lid and mixing vessel clean - especially those areas which form a seal!**
- \_ **Do not allow the vacuum pump to idle for too long (without a mixing vessel)!**
- \_ **The vacuum cannot be switched off until the mixing time has elapsed!**
- \_ **Always hold the mixing vessel in your hand when deactivating the vacuum!**
- \_ **Check if the (preliminary) filter is blocked. If so, replace it to ensure that the vacuum is attained continuously.**
- \_ **Never start mixing without having a mixing bowl in position!**

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hereby declare that the product described below, both, in its basic design and construction and in the version marketed by us is in conformity with the relevant safety and health related requirements of the appropriate EU Directive. This declaration shall cease to be valid if modifications are made to the product without our approval.

Product:

**Minimix  
Vacuum mixer**

Number:

115690

Applicable EU Directives:

98/37/EU - Machinery guidelines  
EU Low-Voltage Equipment Directive  
(73/23/EU)  
EU Directive on Electromagnetic  
Compatibility (89/336/EU)

Applicable

harmonized standards:

**DIN EN ISO 12100-1  
DIN EN ISO 12100-2  
DIN EN 55014-1/2  
DIN EN 61010-1**

Applicable national standards:

-

It is ensured through internal measures that products will always meet the requirements of current EU Directives and relevant standards.

Koblach, 20.01.2005

by order Dr.-Ing. S. Cramer von Clausbruch

The signatory acts on behalf of company management and with full power of attorney.

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**Amann Girrbach AG**
**Amann Girrbach GmbH**