Deutsch

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English

Description of the equipment

- 1 On/Off switch
- 2 Belt/Cutter function switch
- 3 Cutter housing
- 4 Belt changing screw
- 5 Belt centring screw
- 6 Angle adjuster
- 7 Cutter unit with cover
- 8 Spacer rings for cutter unit
- 9 SW 15 & SW 17 spanners
- 10 Mandrel with grinding heads
- 11 Cylindrical tungsten carbide cutter
- 12 Suction hose
- 13 Mains cable
- 14 Connecting cable
- 15 Grinding belts

a) Accessories/single components:

- 315202 Conical tungsten carbide cutter
- 315203 Cylindrical tungsten carbide cutter
- 315204 Grinding belt, 100 grit (pack of 10)
- 315205 Grinding belt, 60 grit (pack of 10)
- 315206 Grinding heads, fine (pack of 25)
- 315207 Grinding heads, coarse (pack of 25)
- 315208 Mandrel
- 345105 Suction hose 1.5 m / 38 mm ø
- 345106 Connecting cable
- 315209 Spacer rings for dental arch cutter

b) Package contents:

- Order No. 315041 Multitrim complete incl.
- 1 Conical tungsten carbide cutter
 - 2 Grinding belts, 60 grit
 - 2 Grinding belts, 100 grit
 - 1 Suction hose, 1.5 m / 38 mm ø
 - 1 Cutter cover
 - 1 Mandrel
 - 1 Grinding head, fine
 - 1 Grinding head, coarse
 - 1 SW 17 spanner
 - 1 SW 15 spanner
 - 1 Connecting cable
 - 1 Operating instructions

c) Technical Data:

LWH: 390/320/340 mm
 Weight: 29 kg
 Power supply: 230 V/50 Hz
 Belt motor fuse: T10A
 Cutter motor fuse: T1A
 Belt motor: 1,100 Watt
 135 Watt
 Belt motor speed: 3,000 r.p.m.
 Cutter motor speed: 2,800 r.p.m.
 Chuck diameter: ø 5.5 - 6 mm
 Concentricity: <0.03 mm

Description of the product

The Multitrim is an ideal multi-purpose unit for fabricating models. It was specially developed as a combining grinding and trimming unit for dry preparation of stone and investment models and also dental arches.

Dry preparation prevents additional water contamination, thus avoiding any further unintentional expansion of the stone.

The adjustable model platform on the belt trimmer section can be angled between 85° - 100°. Belt guidance is extremely precise and can be adjusted laterally. A built-in spring enables the belt to be changed very easily. The choice of different grits of belt makes the Multitrim ideal for use with the most varied types of stone and investment.

The cutter for preparing stone dental arches is driven by a separate motor and is also very quickly and easily changed. The equipment includes conical and cylindrical cutters together with grinding heads in two different grit sizes. The various heights of the three spacer rings can be used to raise the platform in stages of 0.5 cm, thus reducing the amount of material removed.

The highly precise mechanical parts guarantee vibration-free rotation. The stone dust is extracted to the side of the trimmer unit. Both units can be connected to a powerful extractor unit (AS400/AS400micro).

The Multitrim is dimensionally very compact and space-saving (its base is equivalent to DIN A3). It is therefore especially suitable for smaller and private practice laboratories. Its user-friendly design is complemented by its maintenance-free construction.

Installation

Please check the unit immediately after unpacking for any possible damage incurred during transit. In case of damage, please complain to your supplier immediately. After checking, remove the transit bolt on the base of the unit. The unit is supplied with a cutter fitted. The other components are enclosed and must be assembled according to these instructions.

This unit is supplied ready for use. However, please check the mains voltage setting (230 V) directly above the mains plug (E). If you are not sure of the voltage used in your country, please ask your electrical dealer. The unit can now be connected to the mains and the extractor unit fitted (the cable for connecting the extractor to the unit is supplied). The suction hose is then connected to the back of the unit.

Fitting a grinding belt

Please ensure that the unit is switched off (1), loosen the left screw (4) and remove the cover. Turn the same screw into the thread on the top (A). The belt is now loose and can be removed sideways (B). Now centre the new belt. Note the grinding direction when fitting it (an arrow on the belt indicates the clockwise direction of rotation). The tension of the belt is optimized by unscrewing the screw further. Replace the safety cover from the side and secure it using the same screw.

Trim run

Turn the "On/Off" switch (1) "On": to ascertain whether the grinding belt is correctly centred, turn the function switch (2) to the left for a second and then centre, according to the marking, using the belt centring screw (3). A red arrow indicates displacement to the right, a green arrow indicates displacement to the left (C). Only slight adjustments should be made since the drive belt

needs a little time to become centred. Check whether the extractor unit has come on. The belt trimmer can be switched off by returning the function switch to the middle position.

The model platform can now be used to full advantage by varying the angles from 85° - 100° as required. Switch on the belt trimmer by turning the "On/Off" switch to the left and place the model on the platform, holding it firmly with both hands and pressing it lightly against the belt.

Installation of the cutter

Remove the cutter cover and switch on the cutter by turning the function switch to the right. Check whether the cutter is centred in the chuck and running vibration-free. If necessary, trim a model to test the cutter. The stone dental arch is now placed on the base plate and the inner and outer sections trimmed (D).

IMPORTANT: The dental arch should be pressed firmly onto the base plate using both hands, otherwise the model might accidentally slip.

To change the cutter, switch the unit off and remove the cutter from the chuck using the two spanners provided and change the cutter/mandrel (E). Insert the cutter as far as it will go and then close both chuck jaws firmly. Switch on the unit and check whether the cutter is centred and running true. Belt and cutter should be changed when worn.

Optional special accessories: "3 spacer rings for the dental arch cutter"

Indication: The spacer rings are used in conjunction with the AMANN dental arch trimmer for trimming stone dental arches or casts. The various heights of the three spacer rings can be used to raise the platform in stages of 0.5 cm, thus reducing the amount of material removed.

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IMPORTANT: Always disconnect the unit from the mains

Remove the old fuse downwards and replace it with a new fuse. Replace the fuse housing and reconnect the unit.

Warnings and safety aspects:

The Multitrim was developed for use in dentistry. Installation and operation must be carried out according to the operating instructions. The manufacturer accepts no responsibility for damage which occurs if used for other purposes or if not used according to instructions. In addition, if the intended application is not listed in the user's manual, the user is obliged to check personally in advance that the unit is suitable for this purpose.

- Always switch off the unit before changing the belt or cutter!
- Never use the belt grinder without the side cover of the cutter
- Always secure the model when working on it (hold firmly with both hands) Never use the Multitrim without the extractor, otherwise plaster dust will collect on the mechanical components inside the unit
- Always disconnect the unit before changing the fuse
- Wear safety glasses
- Anyone with long hair must wear a hairnet

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EU Declaration of Conformity	
We	Fa. D. Amann GmbH Herrschaftswiesen 1 A-6842 Koblach
Hereby declare that the design, construction and commercial execution of the machine designated below is in conformity with the relevant safety and health requirements of the appropriate EU directive.	
Any modifications performed on the machine without our approval shall invalidate this declaration.	
Machine designation:	Dry milling and grinding device
Machine Type:	Multitrim
Serial No.:	315041
Relevant EU directives:	89/392/EGW Machine-directives 73/23 EWG Low voltage directive 89/336/EGW EMV directive
Applicable harmonized standards:	EN 292-1 / EN292-2 EN 61010-1 / EN 55014
Koblach, 11.3.1999	Ing. Thomas König

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