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EN User manual

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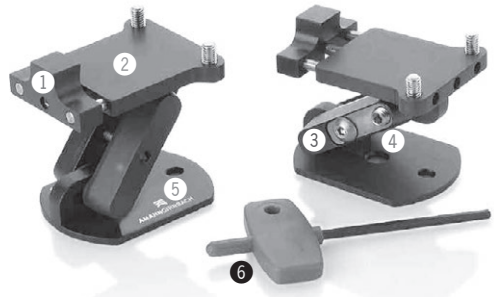
Art.-Nr.: 217480C Noplast für Artex C

DESCRIPTION OF THE UNIT

- ① Allen screw for gripping model
- ② Model platform
- ③ Adjusting screw
- ④ Positioning screw
- ⑤ Base plate
- ⑥ Allen Key

DESCRIPTION OF THE PRODUCT

The Noplast comprises 2 identical parts for mounting upper and lower plaster models in an articulator. When using a Noplast, in many cases articulating plaster can be dispensed with completely and the models can be articulated quickly, neatly and precisely. The technician can also adjust the bite quickly and easily at a later stage.

HANDLING

The upper and lower models are attached to the identical parts. Before doing so, the models must be trimmed to the required size.

Before being articulated, the models must be gripped on the Noplast by tightening the Allen screw ① on the back of the model platform ②.

The Noplast platform with the lower model is usually fitted to the articulator first.

The lower model is gripped by using the Allen key to tighten the central adjusting screw ③, followed by the front positioning screw ④. When articulating the models according to average values, they can be aligned with the Bonwill triangle marker on the articulator.

The upper model is then positioned correctly on the lower model (possibly using the bite registration). The upper model is fixed in position in the same manner as the lower model: First tighten the central adjusting screw ③, and then the front positioning screw ④. It is advisable to hold both models with your hand during this procedure.

To loosen or change the model position you proceed in the reversed manner, this means you first loosen the front positioning screw ④ and then the central adjusting screw ③.

CLEANING AND MAINTENANCE

The Noplast is maintenance-free. It should be cleaned as carefully as possible and without using solvents. The Noplast will continue to function long-term without being lubricated.

Technical data (per part)

D/H/W:	67mm / 35mm / 46mm
Weight:	137g

Accessories and single components

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Amann Girrbach AG