Was-tun-wenn

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What to do if...

Problem	Cause	Solution
Device does not acknowledge the lid is closed		Adjust lid and check microswitch, call service department*
Liquid escapes	Liquid outlet clogged	Clean clip, refer to cleaning instructions of liquid outlet
Liquid squirts out, even after cleaning		Check pump settings, call service department*
Improper amount of dose used	Dirt underneath the scale	Clean all spaces, refer to instructions on how to clean the scale
Required amount of powder cannot be achieved	Hopper empty	Press "Start" button to abort dosing process, refill hopper
	Powder is not dispensed	Press "Start" button to abort dosing process, tap on hopper wall to cause powder to drop down, clean hopper walls, refer to cleaning instructions of powder hopper
	Worm gear blocked	Check and clean hopper outlet, refer to cleaning instructions of powder hopper; if you still require assistance, call the service department
Required amount of liquid not reached	Container empty	Press "Start" button to abort dosing process and check the respective container
	Liquid outlet and hose clogged	Clean clip and hose, refer to cleaning instructions for liquid outlet, if hose still clogged, call service department*
* If the device must be returned for service, please ensure all containers are completely empty		

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