Bedienungsanleitung
User manual

DIN EN ISO 13485

AMANNGIRRBACH

www.heilblau.com 911608IN 2008-12-19

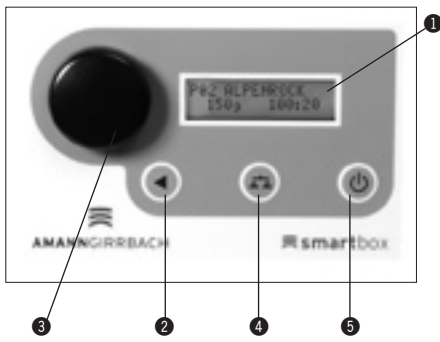
Manufacturer | Hersteller
Distribution | Vertrieb

Amann Girrbach AG
Herrschaftswiesen 1
6842 Koblach | Austria
Fon +43 5523 62333-0
Fax +43 5523 55990
austria@amanngirrbach.com
www.amanngirrbach.com

D

KALIBRIERANLEITUNG SMARTBOX

Smartbox Kalibriergewicht, 2500 g

- 1 Display
- 2 Enter Taste
- 3 Multifunktionaler Drehknopf
- 4 Waage-Taste
- 5 Start/Stop-Taste

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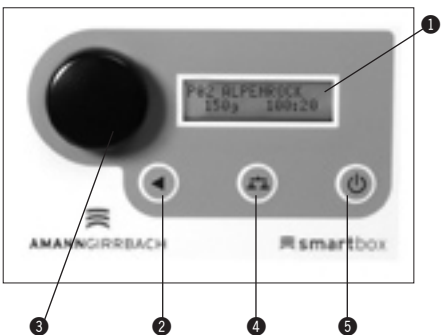
Calibration

To do the calibration you need a calibration weight of exactly 2.5 kg (5.51lbs)!

1. Switch OFF the unit at the main switch.
2. Push Enter- ② and Start/Stop button ⑤ at same time and switch the unit ON at the main switch.
3. If necessary change the language with the rotary button ③. Then push the Start/Stop button for approx. 3 seconds.
4. You have entered the service parameters now. Turn the rotary button ③ until you see „calibration“ on the display (see picture).

5. Confirm with the Enter button ②.

6. Put the black plastic protection onto the scale.

Left side on display ① shows "0g" and on the right hand you see a floating number. Wait a few seconds until this number is stabilized. Confirm by pushing the enter button ② for approx. 2 seconds

7. Now the display shows 2500 g on the left side. Put the calibration weight (2.5 kg) onto the scale and wait until value on the right side of display is stabilized. Confirm by pushing enter button for approx. 2 seconds again.

8. Unit asks, if you want to save this calibration (yes/ no). Turn rotary button ③ to "yes" and confirm with the enter button. ②

9. Take calibration weight from scale and go "back" with the enter button. Then turn the rotary button until you see "quit configuration" and confirm with enter again.

10. You are back to the standard user menu now. Push the scale button ④ and see if "tare" works - scale should show 0g. Put a weight or mixing bowl onto the scale and see if the scale works correct!

11. If the calibration was successfully you can start dispensing with your programm.

ATTENTION: Avoid to get into the "clear memory" menu. Confirming causes an overall clearance of the calibration and all your stored programs! Do not change anything in the menu "parameter" without getting in contact with **AMANN GIRRBACH** first!

Packing list:

Order No.:

116135 Smartbox calibration weight 2500 g