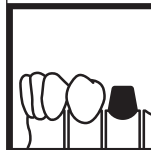
 **smartmix**

**Modellherstellung**  
*Model fabrication*  
*Fabrication du modèle*  
*Sviluppo dei modelli*  
*Elaboración de modelos*

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**User manual**

**Mode d'emploi**

**Istruzioni d'uso**

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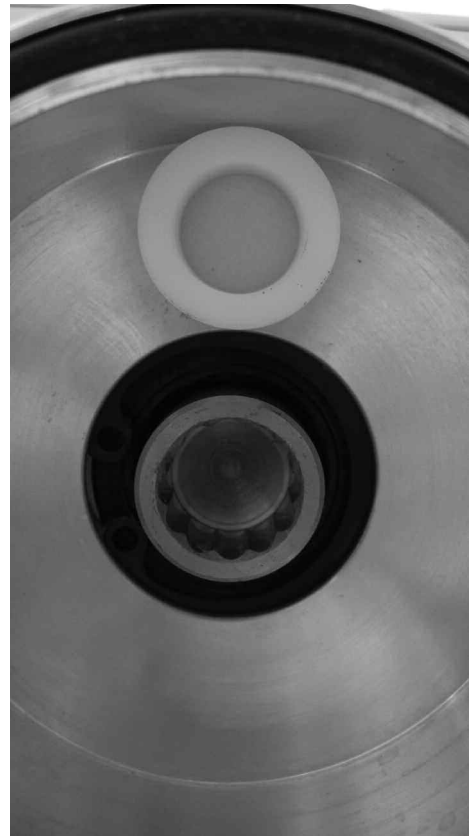
**AMANN GIRRBACH**

Deutsch	04 - 08
English	09 - 12
Française	13 - 16
Italiano	17 - 20
Espagnole	21 - 24
EG-Declaration of conformity	25 - 26

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## DESCRIPTION OF THE UNIT

The Smartmix vacuum mixer is for mixing all types of plaster, investment material and silicone paste. This unit is constructed robustly. A twin-chamber vacuum pump creates a high vacuum in a very short time. As the vacuum sucks the mixing beaker onto the unit, no additional catches are required.

The Smartmix includes a high quality electronic control system which can be used to store up to 20 mixing programmes. Each mixing programme includes the following settings:

### 1. Mixing time

The mixing time can be set progressively from 0 seconds to 5 minutes.

### 2. Speed (100 - 450 r.p.m.)

The speed can be set in stages of 10 from 100 - 450 r.p.m.

### 3. Premixing

The premixing stage can be switched on or off. When switched on, an interval of 5 - 20 seconds (in stages of 5) can be programmed. To prevent the powder being whirled up, premixing commences at a speed of 100 r.p.m. and increases linear to the speed set in section 2). If the premixing stage is activated, no hand mixing is required.

### 4. Prevacuum

This stage can also be switched on or off. When it is switched on, a constant vacuum is created in the mixing vessel without actually mixing. The intervals can be set between 2 and 60 seconds (progressively).

### 5. Interval time

The interval time denotes after how many seconds the direction of rotation of the mixing blade should be reversed. The shortest interval time which can be set is two seconds. The direction cannot be reversed if the interval is set at less than two seconds.

### 6. Programme description

The name of a programme can be entered here. The name can have up to 12 characters. All settings are entered with the multifunctional rotary knob. The Smartmix was designed to be as ergonomic and straightforward as possible. The controls of the Smartmix show how simple it is - they consist of the multifunctional rotary knob and 2 buttons only.

## INSTALLATION

After unpacking the unit, please check it immediately for possible damage incurred during transport. Should the unit have been damaged during transport, please complain to your supplier immediately. Although this unit is supplied ready for use, always check the voltage setting (115/230 V) immediately above the power socket. If you are not certain which voltage is used in your country, please ask your local supplier of electrical appliances.

## INSTALLING ON A BENCH

The stand is available as an accessory and consists of a firm, heavy base with a back. The stand is screwed to the bottom of the Smartmix with the screws supplied. The unit is then ready for use.

## FITTING TO A WALL

A wall drilling template is supplied with the unit for drilling holes in the wall quickly and exactly. Ensure that the holes are aligned exactly and horizontally (check with a spirit level if necessary). Once the holes have been drilled, press the rawplugs into them. Screw in the screws until approximately 5 - 6 mm protrude and hang the unit on them.

**Caution: Special rawplugs must be used for sandwich-type plaster board walls!**

1. Stick template onto required wall mounting position.
2. Mark the center of the 4 crossed lines using a tip end instrument.
3. Remove template and drill the holes. Now you can press the enclosed dowels into the holes. Screw the enclosed bolts into the dowels and leave a 0.15 inch space before they align with the dowels.
4. The bigger holes on the back (marked black on this template) will help you finding the screw-positions when hanging Smartmix onto the wall. Check carefully if the device has engaged correctly on your wall before removing the weight from your arms when holding the device.

## OPERATING THE UNIT

The Smartmix is switched on with the On/Off-switch on the bottom of the unit. The front panel of this unit includes a two-line LC display **(A)**, the "Enter" button **(B)**, the multifunctional rotary knob **(C)** and the Start/Stop button **(D)**. All the Smartmix programmes can be operated with these controls. This unit is ready for use immediately after switching on. The mixing programme used last is displayed.

### Example:

Programme number	Programm description
P01	MODELPLASTER
MixingTime	0:46

Mixing time

## LANGUAGE

The Smartmix includes a multilingual menu (German, English, French, Italian and Spanish). Should you prefer a different language to that which is set, switch the unit off, press the "Enter" button **(B)** and "Start/Stop" button **(D)** simultaneously and switch the unit on again. The unit is then in its programming mode. Rotate the multi-functional knob **(C)** to set your preferred language and confirm your selection with the "Enter" button. The unit is then ready for use.

## MIXING

Select the correct size of mixing beaker and prepare the material for mixing. If the "Premixing" stage is switched on (refer to the section on "Storing a mixing programme"), the material does not have to be premixed in advance. Wipe the top edge of the mixing vessel clean and fit the lid. The multifunctional knob **(C)** can be turned to select the programme - from programme 1 (P01) to programme 20 (P20).

**Please note:** You can also use the Smartmix without storing a special mixing programme in advance. Simply press the "Enter" button **(B)**, set the desired mixing time with the multifunctional knob **(C)** and press "Enter" again to store it. Press the "Enter" button again to start the unit. Press the Start/Stop button **(D)** to start mixing and raise the mixing beaker onto the suction nozzle. After only a few seconds, the suction holds the vessel in place. Once full vacuum has been attained, mixing commences automatically. The mixing time is counted backwards

on the LC display. An alarm can be heard once mixing has been completed.

During mixing, the vacuum pump sucks the beaker onto the unit firmly. Once mixing has been completed, press the Start/Stop button again to switch off the vacuum pump. Please hold the mixing beaker firmly at this stage to prevent it falling off the unit once the vacuum is released.

## STORING MIXING PROGRAMMES

The Smartmix is operated at three levels. The "Enter" button **(B)** should be pressed to switch from level to level. The settings are adjusted with the multifunctional knob **(C)**.

## MENU

	LEVEL 1
	P01 MODELPLASTER MixingTime 0:46 Programme 1-20c
	MixingTime 0:46 <small>0 sec. to 5 min.</small>
	Store >Mixing Settings
	LEVEL 2
	LEVEL 3
	Speed 350 rpm <small>100 - 450 rpm</small>
	Premixing 15 sec <small>off on 1 - 20 sec</small>
	Prevacuum off <small>off on 2 - 60 sec</small>
	Intervall time 15 sec <small>off on 2 - 60 sec</small>
	Programme name MODELPLASTER <small>ALPHABET...</small>

1. Turn the multifunctional rotary knob **(C)**, referred to as the "MRK" hereafter) to the next vacant storage location or to where an adjustment is required.
2. Press the "Enter" button **(B)**. You are now at level 2 and can turn the MRK **(C)** to adjust the mixing time.
3. Press the "Enter" button **(B)**.
4. Should you wish to use the standard settings for your mixing programme, press the "Enter" button **(B)** again. The unit is then ready for use (refer to the "Mixing" section for further instructions).
5. Should you wish to adjust your mixing programme exactly, move the selection arrow by turning the MRK **(C)** to "Mixing settings" and press the "Enter" button **(B)**.
6. You are now at level 3. Turn the MRK **(C)** until you have reached the desired speed and press the "Enter" button **(B)**.

7. The premixing stage can be switched off or on in the next field. Again, turn the MRK to the desired setting and press the "Enter" button **(B)**.
8. This setting can be used to place the material under vacuum without mixing it, before the actual mixing process starts. The prevacuum stage can be switched off or on (with time setting). Turn the MRK to set it and press the "Enter" button **(B)**.
9. The interval time denotes after how many seconds the mixing blade should change direction.
10. A programme description can be entered in the last field. To do so, continue turning the MRK **(C)** until the correct letter is reached. Press the "Enter" button **(B)** to confirm the letter and the cursor will jump to the next position automatically. A name of up to 12 letters can be stored. If not all 12 letters are required, fill the vacant spaces with blank characters. Once the last character has been entered, the programme will have been stored and mixing can begin. (Refer to the "Mixing" section for further instructions).

### **VACUUM DISPLAY**

While the mixer is in operation, the current vacuum level is displayed instead of the programme name. The vacuum level is displayed.

### **REPLACING THE PRELIMINARY FILTER**

The preliminary filter is inside the suction nozzle. The main filter inside the unit and the preliminary filter protect the vacuum pump against possible contamination. To replace the preliminary filter, pull the filter holder downwards and fit a new filter. The filter should fit the filter holder flat and without leaving a gap. Then press the filter holder as far as possible into the opening.

Never operate the Smartmix without a preliminary filter!

### **SERVICING**

Like most AmannGirrbach equipment, the Smartmix is designed to be maintenance-free and includes a maintenance-free vacuum pump. Simply keep the unit clean. The sealing surfaces of the suction nozzle and lid must be kept particularly clean to ensure that the vacuum is optimum. The seal can be kept supple by coating it periodically with a thin coat of Vaseline or similar grease.

**Technical data:**

Depth/Width/Height:	210 x 148 x 270 mm
Weight:	10 kg
Power supply:	115/230 V - 50/60 Hz
Fuse:	T2A
Power uptake:	160 W
Mixing speed:	Up to 450 r.p.m.
Vacuum:	100 mbar
Pump capacity:	11 l/min.

**Accessories and single components:**

Art.Nr.	
115610	Smartmix stand
115620	Beaker with lid and blade S, 100 ccm
115630	Beaker with lid and blade S, 250 ccm
115640	Beaker with lid and blade S, 500 ccm
115650	Beaker with lid and blade S, 750 ccm
115660	Beaker with lid and blade S, 1,000 ccm
115621	Replacement breaker, 100 ccm
115631	Replacement breaker, 250 ccm
115641	Replacement breaker, 500 ccm
115651	Replacement breaker, 750 ccm
115661	Replacement breaker, 1,000 ccm
115605	Filter holder
115606	Spare filters (5 pcs.)

**Packing list:**

Art.-Nr.	
115600	Smartmix unit excluding the mixing beaker, for wall fitting

**WARNINGS AND SAFETY ASPECTS:**

The Smartmix was developed for use in dental laboratories. It must be installed and operated as described in the user's manual. The manufacturer is not liable for damage caused by using the unit for other purposes or handling it incorrectly.

In addition, if the intended application is not listed in the user's manual, the user is obliged to check personally in advance that the unit is suitable for this purpose.

- **Do not clean the Smartmix with liquids containing solvent**
- **Keep the lid and mixing beaker clean - especially those parts which create the seal**
- **Do not run the vacuum pump too long without connecting a mixing beaker**
- **When releasing the vacuum, always hold the mixing beaker with your hand**

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Fa. Amann GIRRbach GmbH
**Amann Girrbach AG**
**Amann Girrbach GmbH**