
 **steamerplus**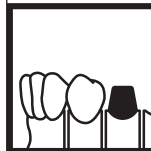

Modellherstellung
Model fabrication
Fabrication du modèle
Sviluppo dei modelli
Elaboración de modelos

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
User Manual

Mode d'emploi

Istruzioni d'uso

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




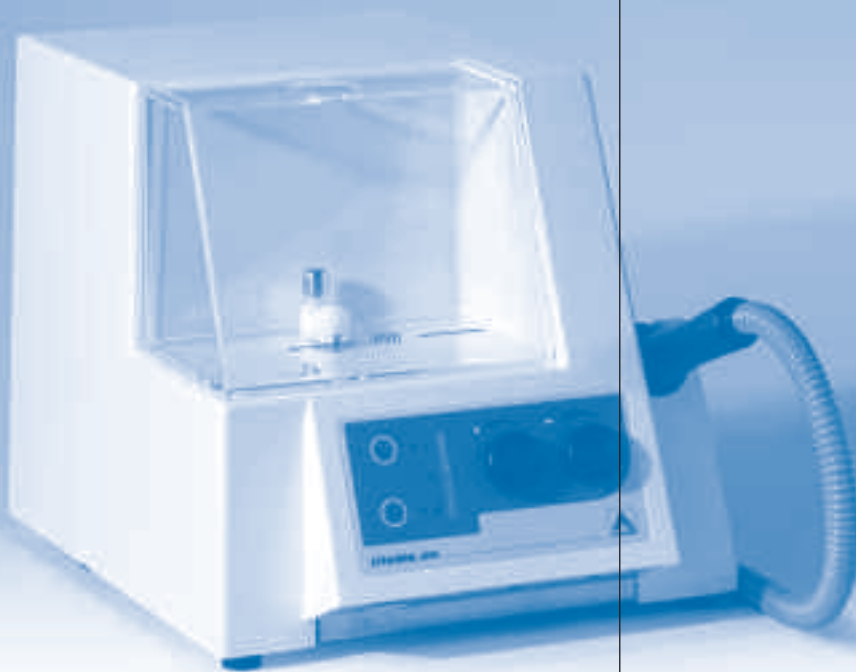


AMANNGIRSBACH

Deutsch	04 - 07
English	08 - 11
Française	12 - 15
Italiano	16 - 19
Español	20 - 23

EG-Declaration of conformity	24 - 25
---------------------------------	---------

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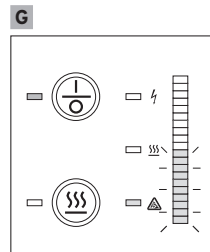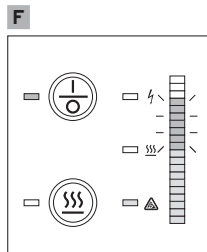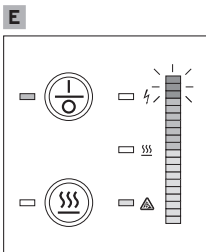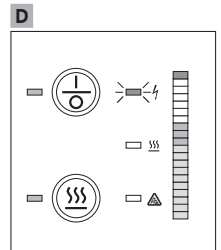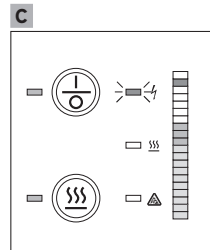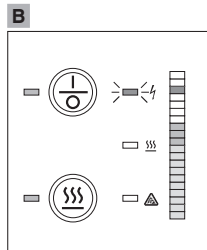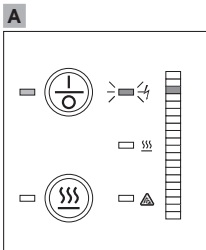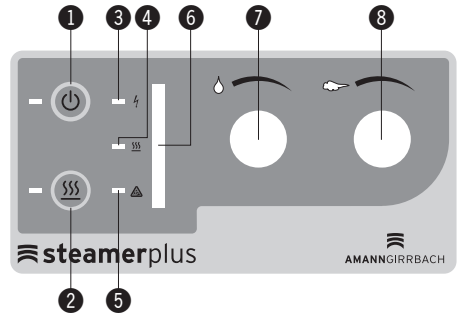USER MANUAL

- ① ON/OFF touch key and indicator light
- ② Heating touch key and indicator light
- ③ Red LED display: Malfunction
- ④ Green LED display: Heating on/off
- ⑤ Orange LED display: Descale unit
- ⑥ Pressure indicator
- ⑦ Saturated steam control
- ⑧ Steam pressure control
- ⑨ Water tank (removable)
- ⑩ Steam gun holder
- ⑪ Steam gun
- ⑫ Steam button
- ⑬ Drain valve
- ⑭ Socket for mains plug

PRODUCT DESCRIPTION

The Steamer Plus is a high-quality, universal steam cleaner that can be used for:

- _ cleaning all types of models, dies, metal frameworks, a large variety of small items, porcelain or prosthetic acrylics
- _ removing aluminium oxide residue
- _ removing wax residue
- _ cleaning articulators
- _ cleaning dental tools/instruments
- _ cleaning jewellery.

The unit has a total water capacity of 4.4 l. The 2-tank system allows the unit to be filled during operation and provides a constant check of the water level.

As the steam pressure of the Steamer Plus is infinitely variable, it can produce an extremely fine steam jet. Hot water can also be added using the saturated steam control to ensure optimum cleaning power even with ingrained dirt. The automatic descaler display indicates when the unit requires descaling.

INSTALLATION

After removing the unit from the packaging, check it immediately for any damage caused during transit. If any damage has occurred, inform the

supplier immediately. The unit is always supplied ready for operation. Check the voltage. Check whether the voltage on the rating plate on the back of the housing is the same as the mains voltage. Then simply connect the mains cable (14) and, before operating the unit, check that the drain valve (13) on the back of the unit is closed. Then fill the tank (9) with water. The tank can be removed by simply lifting it upwards. This allows it to be filled easily and quickly.

CAUTION! It is essential to adhere to the prescribed descaling periods if tap water is used! Any damage due to lime scale, obviously caused by disregarding the descaling intervals, renders the guarantee invalid! Never add chemical products or other additives!

The unit can now be switched on using the ON/OFF key (1) (the green indicator to the left of the key lights up). Water from the tank is now pumped automatically into the hot water boiler until the boiler is filled to maximum. The heating is also immediately activated so that the water in the boiler is heated as quickly as possible. This is indicated by the green light to the left of the heating key (2). The unit now takes approx. 15 min to heat up. The green LED (4) indicates that the unit is heating up and the pressure indicator (6) displays the increase in pressure in the tank. The pressure indicator is an LED bar and is graduated upwards into 10 orange, 7 green and 3 red bars.

OPERATION

The Steamer Plus is ready to operate when the pressure indicator (6) reaches approximately the middle of the green area and the heating indicator light (4) goes out. Remove the handpiece of the steam gun (11) from its holder (10). Press the steam button (12) to steam clean the items. Hold the steam gun over a washbasin or protective tray when steam cleaning. When cleaning small items, we recommend using the sieve, which is available as an accessory. The steam pressure can be increased or decreased as required using the steam pressure control (8). Hot water can also be added using the saturated steam control (7). After steam cleaning replace the steam gun in the holder.

IMPORTANT!

The saturated steam control **(7)** and the steam pressure control **(8)** should be opened fully at least once a week and operated to prevent lime scale deposits in the connecting pipes of the unit.

WATER LEVEL CHECK

The heating is automatically reactivated when the pressure decreases. The water level in the boiler is also monitored and more water is pumped into the boiler from the transparent tank when the level decreases. When the water level in the plastic tank is below minimum, the bottom red LED of the pressure indicator **(6)** lights up and the red malfunction indicator **(3)** simultaneously starts flashing. The water tank **(9)** should then be refilled. Ensure that the tank is properly placed on the unit again after refilling. The malfunction indicator **(3)** switches off when the tank is properly in position and shortly afterwards the red LED of the pressure indicator also switches off. The Steamer Plus is then ready for operation again.

CAUTION - RISK OF INJURY!

The unit may be highly pressurised when switched on. Never open the drain valve (13). Never point the steam jet at people or animals as this could cause burns.

MALFUNCTION INDICATOR

The unit has a malfunction indicator and an acoustic warning signal to indicate a problem.

**A PYROMETER ERROR
(ALSO ACOUSTIC SIGNAL)**

The pyrometer is defective and the unit should be checked. Contact an authorised servicing agent.

B WATER LEVEL WARNING

- _ The water level in the tank **(9)** is below minimum and water should be added.
- _ The water tank **(9)** is not on the unit or has not been placed properly on the unit.

**C WATER LEVEL ERROR
(ALSO ACOUSTIC SIGNAL):**

The water in the pressure tank does not reach the correct level, even though the pump is operating. The unit should be checked. Contact an authorised servicing agent.

**D STEAM PRESSURE ERROR
(ALSO ACOUSTIC SIGNAL):**

The required pressure is not reached in the pressure tank, even though the temperature is correct. Contact an authorised servicing agent.

NOTE

If the pump has drawn in air, take the following steps:

- _ Switch off the heating **(2)** and remove the water tank **(9)**.
- _ Allow steam to escape by pressing the steam button **(12)** until the pressure in the steamboiler is reduced.
- _ Clean the water tank, fill it and replace it on the unit. The pump should now suck up water properly and pump it into the steamboiler.
- _ Switch on the heating **(2)** to prepare the unit for operation.

IMPORTANT! The steam boiler has a safety valve that automatically opens if the pressure reaches 6 bar. If the safety valve opens, switch off and unplug the Steamer Plus. The unit should be checked by the manufacturer as a safety precaution.

SERVICE/MAINTENANCE

The Steamer Plus is virtually maintenance-free. It must however be descaled regularly. A special LED **(5)** indicates when the unit requires descaling. The electronic descaling programme (refer to section Electronic Descaling Programme) ensures that descaling is carried out easily and correctly.

ELECTRONIC DESCALING PROGRAMME

Always ensure the unit is cold and non-pressurised before starting the descaling programme!

1. The unit should be connected to the mains

during the entire descaling procedure.

2. Fully open the saturated steam control **(7)** and steam control **(8)**.
3. Switch on the unit **(1)** Abb. von der Folie platzieren and deactivate the heating **(2)** Abb. von der Folie platzieren
4. Hold the unit over a washbasin.

Ensure that the drain valve **(13)** is over the washbasin. Leave the steam handpiece on the edge of the washbasin, as water or steam escapes from the nozzle during descaling.

5. **Start the automatic descaling programme E** :
press the ON/OFF key **(1)** for 5 sec with the unit switched on. All the LEDs on the pressure indicator **(6)** light up and the red bars start to flash.
6. **Empty the unit:**
Remove the water tank **(9)**. Hold the steam handpiece in the washbasin and press the steam button **(12)** until no more steam or water comes out. Open the drain valve **(13)**. Activate the pump to empty the unit by pressing the steam key **(12)** until no more water comes out of the boiler. Then close the drain valve **(13)** again and press the heating key **(2)** Abb. von der Folie platzieren 1x briefly to ensure the tank is empty. The flashing LEDs go out when the tank is empty.
7. Empty any residual water from the water tank **(9)** and add AmannGirrbach descaler concentrate in a ratio of 1:5 (1 bottle of descaler: 5 bottles of lukewarm water) to the tank. Replace the water tank. The unit automatically pumps the descaling solution into the boiler. Water may come out of the steam handpiece during this procedure.
8. Descaling requires approx. 1 h and the process **F** can be followed on the green LEDs of the pressure indicator **(6)**. Descaling is complete as soon as all the green bars go out and the orange bars flash **G**.
9. After descaling is complete, empty the unit -> see section 6.

10. Rinsing:

empty any residual descaling solution from the water tank, rinse the water tank and fill with clean water. Replace the tank on the unit. The unit now pumps the clean water into the boiler.

11. After filling it with clean water, empty the unit again to rinse it -> see section 6.
12. The descaling programme is complete when all the LEDs on the pressure indicator **(6)** go out. To end the descaling programme press the ON/OFF key **(1)** for 5 sec. Three acoustic signals confirm that the programme has ended. Switch on the Steamer Plus and fill the water tank in the usual way. As soon as the unit has filled the boiler to maximum, press the steam button **(12)** briefly to reduce the excess pressure built up during filling. After the unit has been heated up, it is ready again for operation.

CAUTION! After the descaling programme has started, all other functions of the unit are shut off. Do not disconnect the unit from the mains supply or the descaling procedure will not be properly completed or stored.

! WARNINGS AND SAFETY ASPECTS:

The Steamer Plus was developed for use in the dental sector. The instructions for use should be adhered to during installation and operation. The manufacturer does not accept liability for damages arising from any other use or improper handling. In addition, if the intended application is not listed in the user's manual, the user is obliged to check personally in advance that the unit is suitable for this purpose.

- **Live components and units must not be steam cleaned!**
- **Never point the steam jet at people or animals!**
- **Adhere to the prescribed descaling periods and descaling instructions!**
- **The unit may be pressurised when switched on! Only open the drain valve when cold!**
- **Never touch the jet nozzle on the hand piece when the unit is ready for operation as this could cause burns!**
- **Always have repairs carried out by authorised trained personnel!**
- **Always use original AmannGirrbach accessories and replacement components!**

Technical data:

L/B/H:	400 x 330 x 316 mm
Weight:	12 kg
Fuse:	T10A
Output:	1700 W
Power supply:	230 V / 50 Hz
Steam pressure:	4.2 bar
Boiler capacity:	1.8 l
Water tank capacity:	2.6 l
Heating-up time: approx.	15 min

Accessories/Components:

Order No.:	
116814	Sieve, small, dia. 50 mm
116815	Sieve, large, dia. 75 mm
516330	Clean Steamer
516340	Stymo Net
116811	Kalk-X Liquid
1168391	O-ring set for nozzle (HS)

Packing list:

Order No.:	
116850	Steamer Plus 230V incl. mains cable and instructions for use

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hereby declare that the product described below, both, in its basic design and construction and in the version marketed by us is in conformity with the relevant safety and health related requirements of the appropriate EU Directive. This declaration shall cease to be valid if modifications are made to the product without our approval.

Product:

Steamer Plus
Steam cleaner

Number:

116850

Applicable EU Directives:

98/37/EU - Machinery guidelines
EU Low-Voltage Equipment Directive
(73/23/EU)
EU Directive on Electromagnetic
Compatibility (89/336/EU)

Applicable

harmonized standards:

DIN EN ISO 12100-1
DIN EN ISO 12100-2
DIN EN 61000-3-2/3
DIN EN 61010-1
DIN EN 61326

Applicable national standards:

-

It is ensured through internal measures that products will always meet the requirements of current EU Directives and relevant standards.

Pforzheim, 20.01.2005

by order Dr.-Ing. S. Cramer von Clausbruch

The signatory acts on behalf of company management and with full power of attorney.

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Amann Girrbach AG
Amann Girrbach GmbH