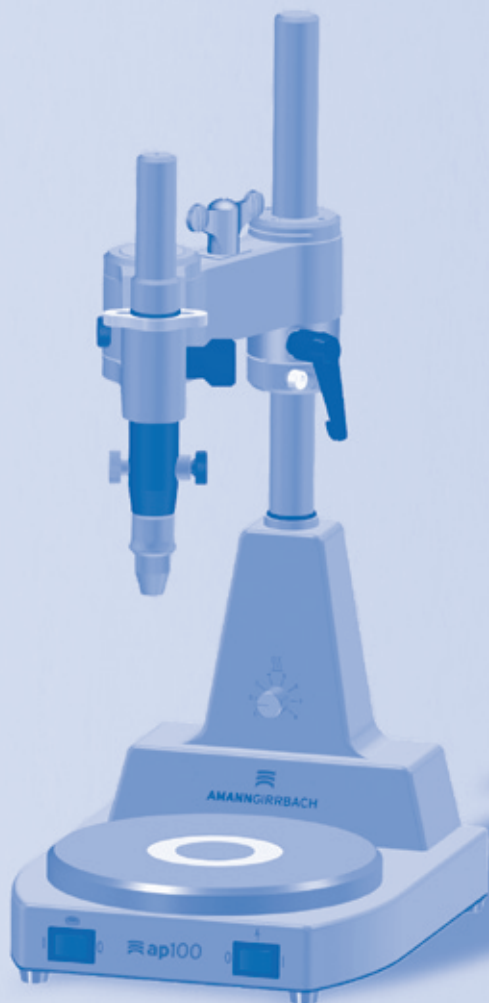
 ap100

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OPERATING INSTRUCTIONS

AP100 SURVEYOR

Translation of the original operating instructions

- 1 Setting ring
- 2 Hinged arm
- 3 Pillar
- 4 Surveying head
- 5 Milling head
- 6 Surveying head/milling head setting and height adjustment screw
- 7 Chuck knurled nut
- 8 Opening for adapter cable bush
- 9 Adapter cable bush set screw
- 10 On/Off switch
- 11 Blockout heating controller
- 12 Model platform magnet on/off switch
- 13 Surveying head setting and height
- 14 Handpiece adapter sleeve set screw
- 15 Spring bolt

Accessories:

- 5 Milling head
- 14 Handpiece adapter sleeve set screw
- 15 Spring bolt

of the hinged arm locks the two hinges exactly and firmly with a screw. The centering wedge on the setting ring can be used to swivel the rigid arm outwards when, for example, changing instruments. When swivelled back, the hinged arm resumes its previous position exactly. Special bearings enable the hinged arm and surveying head to move very smoothly. The level of the surveying head can be adjusted. An MT2 model platform is supplied with the unit. The MT2 can be tilted by up to 90° in all directions and fixed in any position. The "zero degrees" position can be reproduced with a locking mechanism. The AP100 block-out set can be used with the AP100 for blocking out. This set comprises a B2 Blockout holder, 5 gold-plated tips and an adapter cable. The AP100 can also be equipped with a milling head and an adjustable Halogenlamp.

UNPACK AND CHECK

After unpacking the unit, please check it immediately for possible damage incurred during transport. Should the unit have been damaged during transport, please complain to your supplier immediately. This unit is supplied ready for use.

PRODUCT DESCRIPTION

The AP100 is a universal surveyor for the following applications:

- _ Surveying models
- _ Marking undercuts
- _ Carving wax components
- _ Positioning precision attachments
- _ Transferring milled and attachment components
- _ Blocking out - When used with the AP100 Block-out set

The magnetizable base plate in the base enables the model platform to be fixed in place by simply pressing a button. The AP100 also includes an electric controller for heating the blocking out components. The patented locking mechanism

MONTAGE AP100

Place the setting ring **1** and centering wedge (pointing upwards) over the pillar and tighten them. Place the hinged arm **2** on the pillar **3** of the base, making certain to position it parallel and without wedging it. Loosen the spring bolt in the milling head, rotate it through 180° and re-attach it. Insert the surveying head **4** or milling head **5** into the hinged arm and tighten it with the screw **6**. This screw is also used for setting the basic level of the surveying head.

The surveying head is tightened with the knurled nut **13**, for example, for placing an attachment. Rotary instruments (1 - 3 mm shanks) are tightened with the chuck knurled nut **7**.

■ FITTING THE BLOCKOUT SET

(AP100 BLOCKOUT SET)

Plug the adapter cable plug into the back of the base. Then insert the cylindrical section (bush) of the adapter cable into the opening ⑧ in the hinged arm and tighten it with the screw ⑨. Fix the adapter cable to the pillar with the cable holder. Insert the Blockout holder into the surveying head and tighten it with the knurled nut ⑦. Finally, insert the Blockout holder plug into the adapter cable socket from beneath. The Blockout set can be controlled progressively with the knob ⑩ on the base.

■ AP100 (FITTING THE MILLING HEAD)

Grip the surveying head and loosen the screw ⑥ to remove it. Insert the milling head into the opening in the hinged arm from beneath and tighten it with the screw ⑥.

■ INSERTING A HANDPIECE INTO THE MILLING HEAD

Press the correct handpiece adapter sleeve onto the handpiece from the back. Then insert both into the opening in the milling head and tighten them with the screw ⑬.

■ SERVICING

These units do not require servicing. Should they become dirty, keep them dry when cleaning.

CAUTION! Do not steam-clean or use solvents for cleaning!

Technical Data:

D/B/H:	210 x 180 x 400 mm
Weight:	6,2 kg
Chuck range:	1 - 3 mm
Power supply:	230 V - 50 Hz
Power uptake:	22 W
Fuse:	1A time-lag

Packing list:

Order No.:	
177700	AP100 unit including
	P Hinged arm
	Surveying head
	User's manual
	Guarantee

Accessories/ Individual parts:

Order No.:	
177960	AP100 Block-out set
177450	Halogen lamp (for the AP100)
177071	Milling head (for the AP100)
873001	Starter kit - Milling

WARNING AND SAFETY INSTRUCTIONS:

The unit described in this manual was developed for dental applications. It must be installed and operated as described in the user's manual. The manufacturer is not liable for damage caused by handling the equipment incorrectly. In addition, if the intended application is not listed in the user's manual, the user is obliged to check personally in advance that the unit is suitable for this purpose.

CAUTION - RISK OF INJURY:

_Do not steam-clean or clean with solvents.
_If the AP100 Blockout set is fitted, there is a risk of burning if the Blockout set is used at full power.

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Amann Girrbach AG
Amann Girrbach GmbH