ceramill® motion

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User Manual

Mode d'emploi

Instruzioni d'uso

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Deutsch	3 - 5
English	6 - 8
Français	9 - 11
Italiano	12 - 14
Español	15 - 17

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Table of Contents

11	Explanation of Symbols	6
2	Suitable Personnel	7
3	Product Description	7
4	Application	7
4.1	Loading the adapter	7
4.2	Inserting the adapter	8
4.3	Starting the milling program	8
5	Warranty/Exclusion of Liability	8
6	Download Infos	8

1 Explanation of Symbols

Warning indications

Warning indications in the text are marked with a colour-backed triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- _ **NOTE** means that property damage can occur.
- _ **CAUTION** means that light to fairly serious personal injury can occur.
- _ **WARNING** means that serious personal injury can occur.
- _ **DANGER** means that serious personal injury can occur.

Important information

Important information that do not lead to hazards for humans or property damage are marked with the icon aside and are also boxed.

Other symbols in the Manual

Symbol	Meaning
▷	Item of an operation description
_	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

Tab. 1

2 Suitable Personnel

NOTE:

The product may only be used by trained specialised personnel.

3 Product Description

The 3-fold adapter (179245) is an accessory for dry milling of composite blanks in the Ceramill Motion, Ceramill Motion 2 4X and Ceramill Motion 2 5X. It is used to hold up to 3 blanks.

Delivery Scope

_ 3-fold adapter for composite blanks

4 Application

4.1 Loading the adapter

The adapter can also be inserted into the Ceramill Motion before loading it (see figure 2).

- ▷ Depending on your selection in the Ceramill Match 2, insert up to three blanks into the adapter with the groove facing frontward, and secure them with the Allen key.

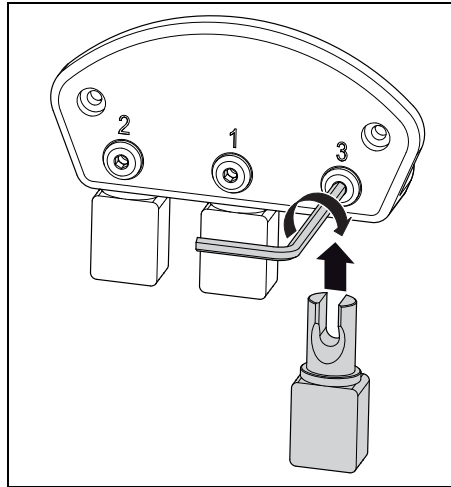
Fig. 1

4.2 Inserting the adapter

- ▷ Insert the adapter into the blank holder of the milling machine and secure with the Allen screws.

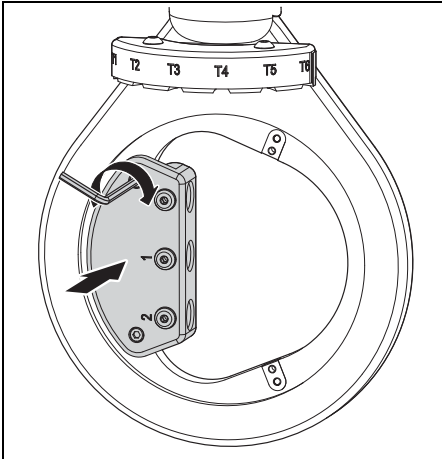
Fig. 2 Ceramill Motion / Ceramill Motion 2 4X

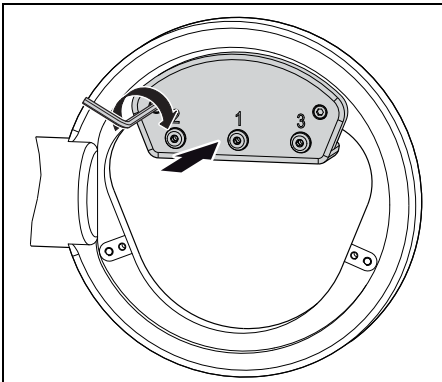
Fig. 3 Ceramill Motion 2 5X

- ▷ Load the adapter, if not already done (see figure 1).

4.3 Starting the milling program

- ▷ Shut the milling machine door.
- ▷ Transfer the milling file.
- ▷ Start the milling program

5 Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

6 Download Infos

Further instructions/manuals and video tutorials can be downloaded under www.ceramill-m-center.com and/or www.amangirrbach.com.

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Amann Gırrbach AG
Amann Gırrbach GmbH