Bedienungsanleitung  
 User manual  
 Mode d'emploi  
 Istruzioni d'uso  
 Modo de empleo

Manufacturer | Hersteller  
 Distribution | Vertrieb

**Amann Girrbach AG**  
 Herrschaftswiesen 1  
 6842 Koblach | Austria  
 Fon +43 5523 62333-0  
 Fax +43 5523 55990  
 austria@amanngirrbach.com  
 www.amanngirrbach.com

Made in the European Union

**AMANNGIRRBACH**

www.heilblau.com 31157-FB 2008-11-20

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### Teleskopkit Starterset

#### Anwendung:

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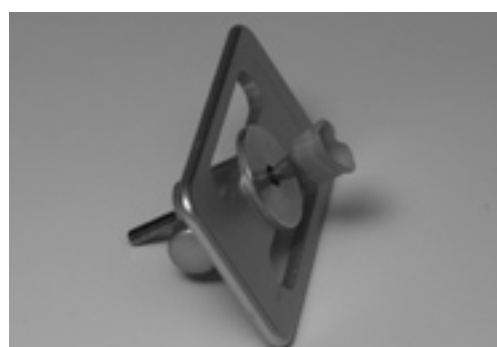
### Telescope Starter Kit


#### Application:

The model fixation pin will be fixed in the parallelometer to the milled Telescope using Ceramill Gel.

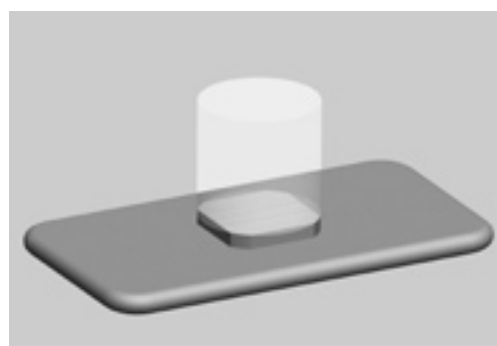The model fixation pin is connected to the model fixation plate and fixed using a knurled-head screw.

The blank is fixed to the blank holding plate with superglue so that the rounded edges of the base which have been glued and the outer surface of the blank conform with one another. Degrease the adhesive base prior to first use.

Both holding plates are attached to the milling device. If the retaining plates in the model or framework blanks have some play, fix them with a bit of Ceramill gel: at the front (right and left next to the clamp lever at the junction of holding plate to support plate) and on both sides of the plate (junction of the retaining plate to the support plate).

The vertical positioning and examination of the dimensions of the Telescope are performed in the device through the use of a sensor and the milling unit. It is important that the Telescope be positioned vertically in such a fashion that 0.5mm of material can be milled off from the side of the blank. Initially, the superfluous material is milled off using a standard pair of rough machining tools.

The Telescope Kit tools are employed in order for the surface to be milled with the proper angle.

Ball tools (KT/KF2.5) or conical tools (UF/UT3) are used for machining out the occlusion area of the Telescope. Hereby, the milling table is tilted in the appropriate direction. The Telescope is removed using a diamond cutting wheel or a staggered-tooth cutter.

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