MILLING UNIT 5X

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2-4

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
INSTRUCTIONS FOR AUTOCALIBRATION

1. AUTOCALIBRATION AIDS

EN

Calibration tool with round end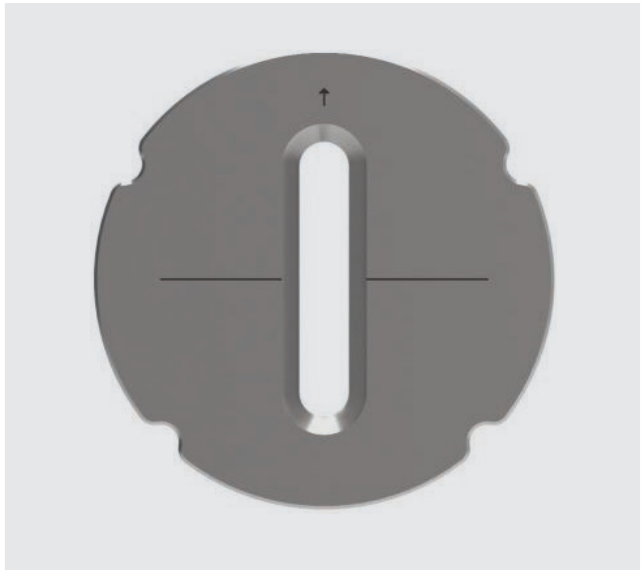


Calibration body

2. GENERAL INFORMATION REGARDING AUTOCALIBRATION

	<p>INFORMATION</p> <p>Observe the following instructions to guarantee correct autocalibration</p> <ol style="list-style-type: none">1. The interior of the milling machine should be thoroughly cleaned before performing calibration. Dust or contamination could be whirled up during the calibration process and critically influence the result2. Clean the chuck before calibration. Please note Section 7.1.1 of the instructions for use when cleaning the chuck3. The calibration body and calibration instrument must be free of dust and grease. <p>Failure to follow instructions can result in incorrect calibration or, in the worst-case scenario, damage to the machine</p>
---	---

3. PERFORMING AUTO-CALIBRATION

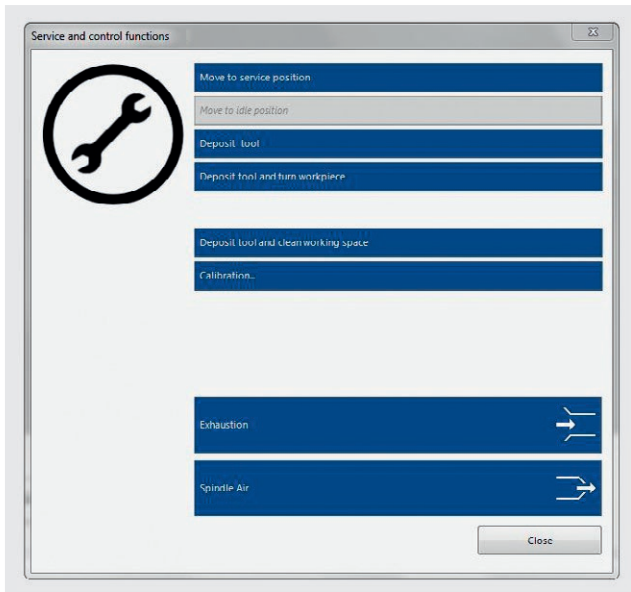STEP 1

Autocalibration can be started in the menu "Service and control functions" at "Calibration ..."

STEP 2

Instructions are given on making the necessary aids available.

Confirm with "OK, start calibration"


STEP 3

It is essential to thoroughly clean the machine and chuck before fitting the calibration body.

The calibration body can then be fitted.

During fitting, ensure that the arrow on the calibration body and fixation screw are pointing in the same direction.

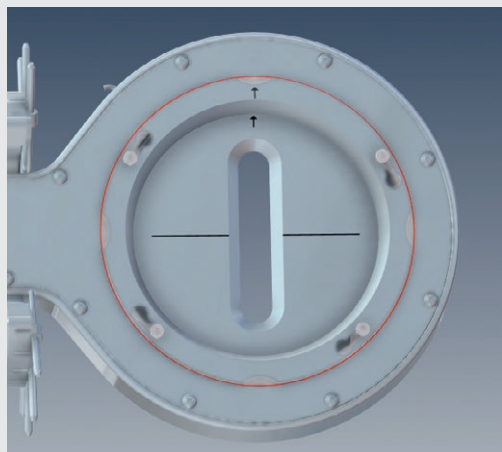INFORMATION

It is essential that any adapters (71 mm/ 98 mm) have been removed from the blank holder before fitting the calibration body.

The calibration body is a highly precise measuring instrument. When fitting it is essential that the calibration body rests fully on the base of the holder.


We recommended fitting the calibration body by hand to avoid tilting it.

STEP 4


Remove tool T1 and insert calibration pin.

Ensure that the round end faces towards the blank holder.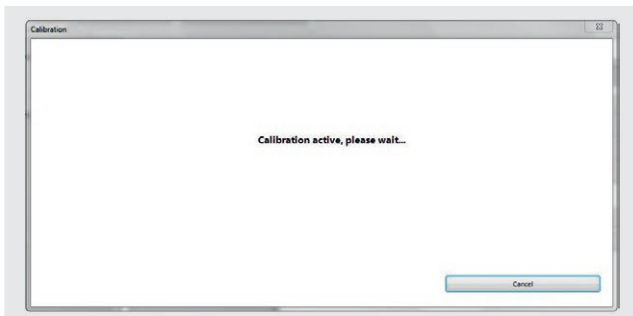

STEP 5

Calibration is now performed.

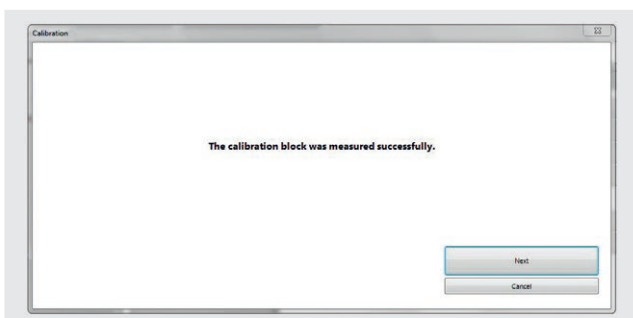STEP 6

Following successful calibration, the machine can be returned to the initial state in the following steps.

Manufacturer | Hersteller
Distribution | Vertrieb

Amann Girrbach AG
Herrschaftswiesen 1
6842 Koblach | Austria
Fon +43 5523 62333-105
Fax +43 5523 62333-5119
austria@amangirrbach.com
www.amangirrbach.com

Made in the European Union