
# **ceramill**<sup>®</sup> mindserve

<b>DE</b> Handbuch	2-11
<b>EN</b> Manual	12-21
<b>FR</b> Mode d'emploi	22-31
<b>IT</b> Manuale	32-41
<b>ES</b> Manual de instrucciones	42-51

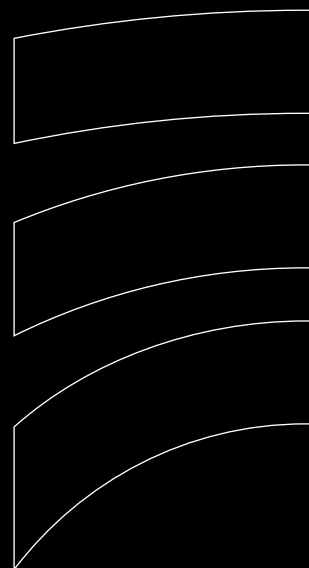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**CONTENTS**

1. Safety guidelines .....	13
2. Product description .....	13
3. Correct use.....	13
4. System requirements.....	14
5. Package contents .....	14
6. Ceramill Mindserve installation.....	14
7. Configuration of the Ceramill Mind computer .....	16
7.1 Version 1: You currently use one or more workstations with a shared Ceramill Mind database and want to transfer these to the Ceramill Mindserve .....	16
7.2 Version 2: You already have a Ceramill Mindserve in use and want to configure an additional workstation for use of the server.....	18
7.3 Version 3: You currently use several workstations with different local Ceramill Mind databases and want to work in future on the Ceramill Mindserve with a common database.....	19
7.4 Version 4: You currently use several workstations with different Ceramill Mind databases and want to continue using different databases on the Ceramill Mindserve.....	19
8. External hard drive.....	20
9. Warranty .....	20
10. Technical data .....	21
11. Declaration of conformity.....	21

## 1. SAFETY GUIDELINES

- \_Protect the Ceramill Mindserve against direct sunlight and chemicals. Ensure that there are no extreme fluctuations of temperature or humidity in the immediate vicinity.
- \_Place the Ceramill Mindserve in a vertical position. Do not place it horizontally.
- \_Do not place the Ceramill Mindserve near any type of liquids.
- \_First remove the power cable before cleaning. Wipe the Ceramill Mindserve using moist cleansing tissues. Do not use any chemical or aerosol cleaners.
- \_Do not place the Ceramill Mindserve on a trolley, table, desk or any other unstable surface where it could fall off.
- \_The power cable must be connected to the correct supply voltage. Ensure that the connected supply voltage is correct and stable.
- \_Ensure that all electrical cables have been disconnected from the electricity supply to cut off all the electrical current from the unit.

### PLEASE NOTE

**The Ceramill Mindserve requires an uninterrupted electricity supply to ensure interference-free functioning. The installation site must be selected, so that a stable supply of electricity is guaranteed. If you want to disconnect the Ceramill Mindserve from the mains, press the ON/OFF key for 5 seconds, so that the Ceramill Mindserve shuts down.**

## 2. PRODUCT DESCRIPTION

The new Ceramill Mindserve was designed for networking several Ceramill CAD/CAM computers. The compact unit is connected with all Ceramill CAD/CAM computers in the laboratory via a standard network connection. Outgoing data from several workstations can be managed and secured with the aid of the Ceramill Mindserve.

## 3. CORRECT USE

The pre-configured Ceramill Mindserve was specially designed for networking several Ceramill CAD/CAM computers.

#### 4. SYSTEM REQUIREMENTS

These system requirements must be guaranteed by an IT Administrator from the customer:


- \_ A gigabit network with automatic IP address allocation must be available.
- \_ The existing computer must also be connected to the network.
- \_ Check whether the latest version of the Ceramill software is installed.
- \_ Installation can only be completed as Administrator.
- \_ The network and file sharing must be activated on the workstations. If in doubt, contact your computer administrator.

EN

#### 5. PACKAGE CONTENTS

- \_ Pre-configured Ceramill Mindserve (2 TB or 4 TB)
- \_ Two RJ45 network cables
- \_ Adapter for Ceramill Mindserve
- \_ USB 3.0 back-up hard drive
- \_ Connection diagram

#### 6. CERAMILL MINDSERVE INSTALLATION

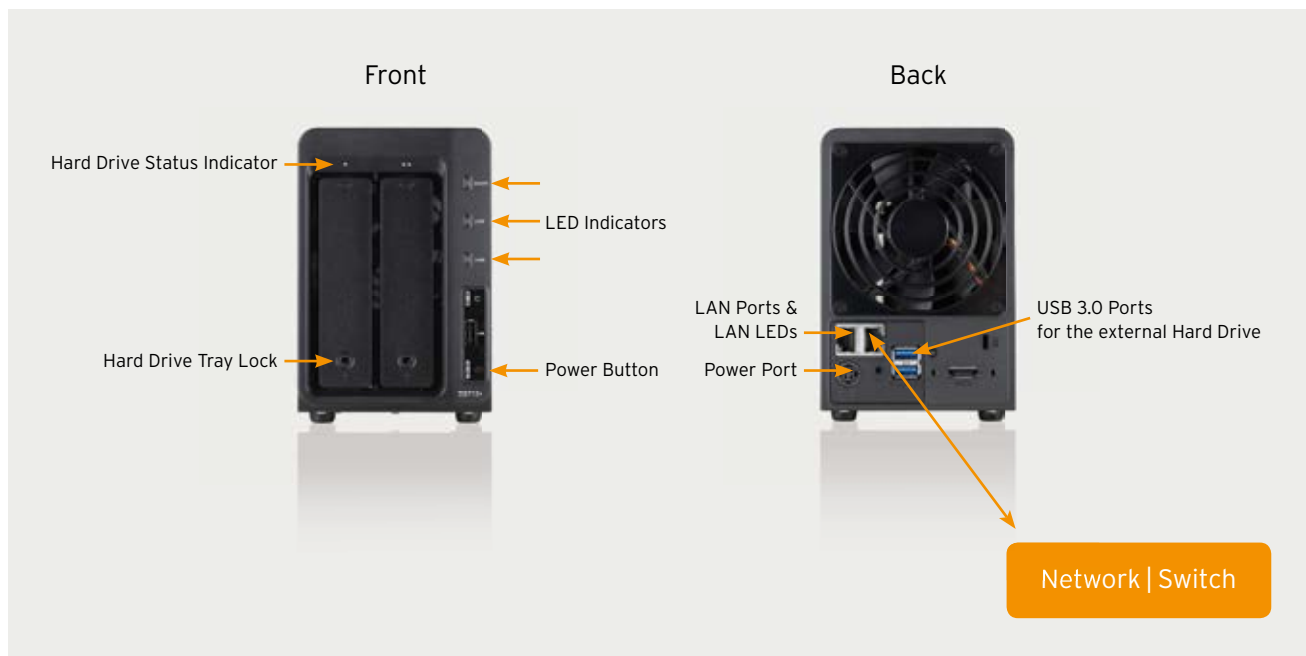Select the installation site, so that accidental manipulation of the connection cables can be excluded.

\_The Ceramill Mindserve should be connected to the mains using the adapter supplied.

\_Connect the network connections on the back of the Ceramill Mindserve with the existing switch and network distributor using the black network cables included (or at least with one Class CAT 5e UTP cable).

\_The external back-up hard drive should be connected to one of the USB connections on the back of the Ceramill Server. The external 2TB hard drive is supplied with power via the Ceramill Mindserve. Connect the external 4TB hard drive to the mains using the adapter supplied.

\_Press the red power button on the front.

\_The blue LED flashes and the server starts. Wait approx. one minute until the blue LED lights up continuously, the Status LED flashes and there is a single beep signal. The Ceramill Server is now started and ready to use.

\_Register as Administrator on your computer and use the following data:  
User: Administrator / Password: ceramill mall

\_The installer and manual are on the USB stick supplied.

\_Connect the network connections on the back of the Ceramill Mindserve with the existing switch and network distributor using the black network cables included (or at least with one Class CAT 5e UTP cable).

\_Update the Ceramill software to the latest version, if necessary.

\_Start via the Start menu / All programmes / Ceramill software the Ceramill Mindserve programme and follow the instructions on the screen.

#### PLEASE NOTE

**First read the manual to avoid loss of data. It is not possible to merge several databases.**

**If you use several databases, you must decide for one or several databases on the Ceramill Mindserve.**

## 7. CONFIGURATION OF THE CERAMILL MIND COMPUTER

The corresponding procedure must be followed regardless of the current configuration.

### 7.1 Version 1: You currently use one or more workstations with a shared Ceramill Mind database and want to transfer these to the Ceramill Mindserve

\_ Register as Administrator on your Ceramill CAD/CAM computer at the workstation. If you already work with several computers on a common Ceramill Mind database, first carry out the installation on a computer that shares the database.

\_ All Ceramill applications must be closed.

\_ Start the Ceramill Mindserve programme using Start - All programmes - Ceramill software


\_ Also click Search server.

\_ When prompted to restart, carry this out and restart the programme.

\_ Select the first option.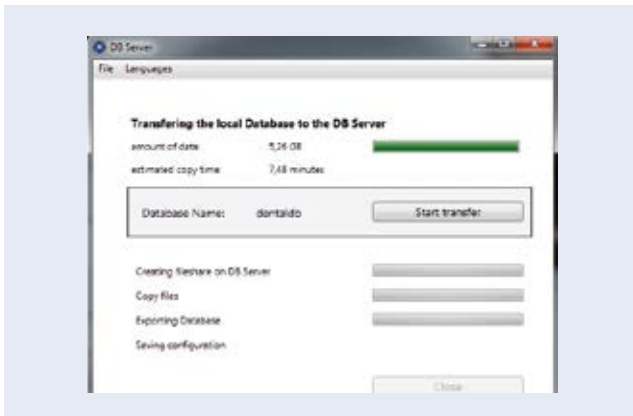


\_ The estimated transfer time is calculated and displayed.

\_ Please note the transfer time. Only start the transfer if sufficient time is available.

\_ No other computer in the network may access the data during transfer.

EN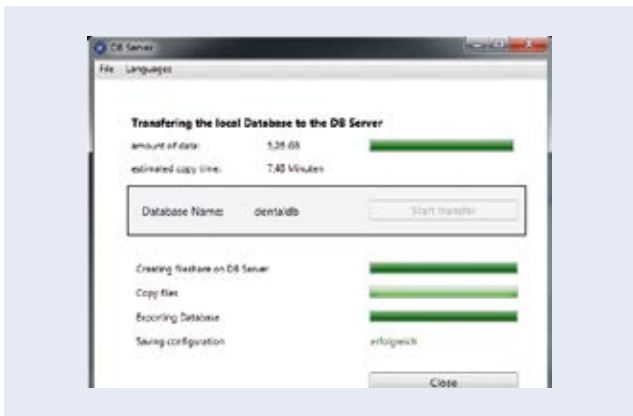

\_ Close the window with "Close" once the transfer is complete.


\_ Restart the computer once the installation is complete. You can then continue to work as usual with the Ceramill Mind. The data are now on the Ceramill Server.

\_ Carry out the steps at Point 7.2 on the computer to configure other computers for use with this database.

## 7.2 Version 2: You already have a Ceramill Mindserve in use and want to configure an additional workstation for use of the server

- \_ Register as Administrator on your Ceramill CAD/CAM computer that you want to configure for the use of an existing Ceramill Mindserve database.
- \_ Close all Ceramill applications.
- \_ Start the Ceramill Mindserve programme using Start menu / All programmes / Ceramill software the Ceramill Mindserve programme and follow the instructions on the screen.

EN

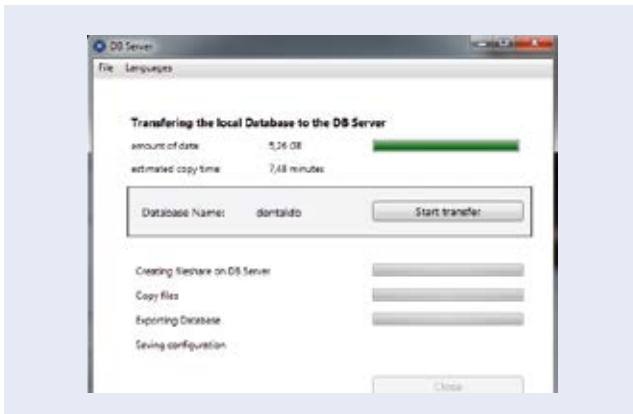
\_ The estimated transfer time is calculated and displayed.

\_ Please note the transfer time. Only start the transfer if sufficient time is available.

\_ No other computer in the network may access the data during transfer.

\_ Select the second option.

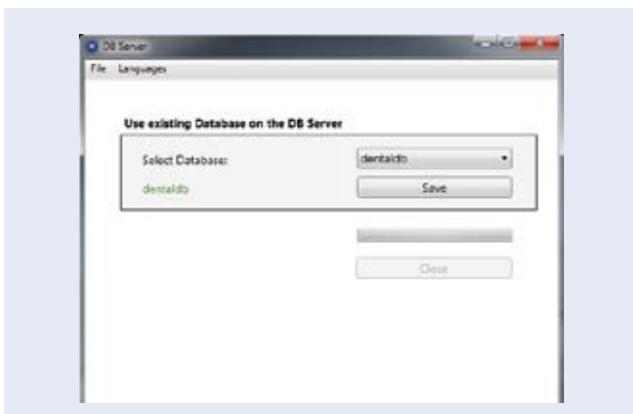\_ Select the required data in the dropdown menu and click on save.

\_ Then close the window with "Close".

- \_ Restart the computer once the installation is complete. You can then continue to work as usual with the Ceramill Mind. The data and database on the Ceramill Mindserve are now used.

### **7.3 Version 3: You currently use several workstations with different local Ceramill Mind databases and want to work in future on the Ceramill Mindserve with a common database**

#### **PLEASE NOTE**

It is not possible to merge several databases.

If you use several databases, you must decide for one or several databases on the Ceramill Mindserve.

Perform step 7.1 on the computer, which contains the database required as your central database on the Mindserve.


Then perform step 7.2 on all other computers with which the central database is to be used on the Mindserve.

### **7.4 Version 4: You currently use several workstations with different Ceramill Mind databases and want to continue using different databases on the Ceramill Mindserve**

\_It is possible to transfer several databases to the Ceramill Mindserve. Perform step 7.1 on each computer whose database should be transferred and use the first option "Transfer local database" for transferring.

\_Perform step 7.2 to connect any other computer with a database.

## 8. EXTERNAL HARD DRIVE

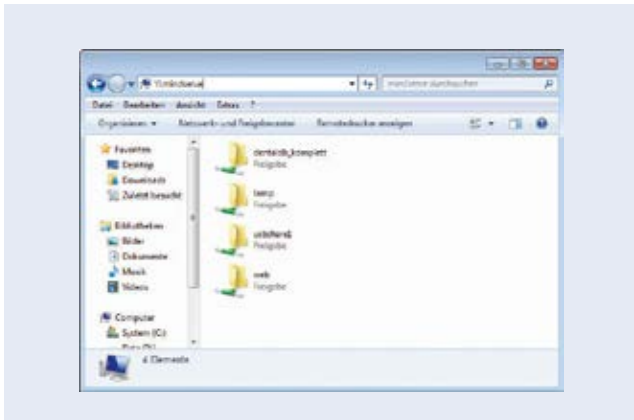\_ If you want to check the back-up of your data on the external hard drive, you can view the data on the hard drive under the path shown.

\_ Path: \\mindserve

\_ Directory: usbshare1

EN

## 9. WARRANTY

The warranty complies with the legal requirements. Further information is contained our General Terms and Conditions (GTC). Users are solely responsible for backing up their data.

Furthermore, warranty and liability claims with damages to persons and property are excluded, if they are due to one or more of the following reasons.

\_ Any person who is involved in the operation, maintenance or repair of the system should have read and understood the manual. We do not accept any liability for damages, malfunctions or data loss caused by failure to follow the operating instructions.

\_ Improper mounting, installation, operation or maintenance of the system.

\_ Operation of the system with defective safety devices or incorrectly fitted or non-functioning safety or protective devices.

\_ Failure to follow commissioning, installation, maintenance, operating instructions in the manual.

\_ Unauthorised design changes to the system.

\_ Unauthorised changes to the operating parameters.

\_ Unauthorised changes of settings as well as programme changes.

\_ Parts and accessories are specially coordinated with the Ceramill Mindserve. Only these Ceramill Mindserve original parts should be used when exchanging parts. Otherwise the warranty is voided. Amann Girrbach does not accept any liability for damages, which are caused by the use of non-original parts.

\_ Catastrophes caused by foreign bodies and force majeure.

## 10. TECHNICAL DATA

<b>Dimensions (H x W x D)</b>	157 x 103.5 x 232 mm
<b>AC Input voltage</b>	100 V to 240 V
<b>Power frequency</b>	50 Hz to 60 Hz, single-phase

EN

## 11. DECLARATION OF CONFORMITY

The design and operational performance of this product complies with the European Directives and additional national requirements. Conformity has been confirmed with the CE Mark.

The Declaration of Conformity of the product can be called up at [www.amanngirrbach.com](http://www.amanngirrbach.com).

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**Amann Girrbach AG**
**Amann Girrbach GmbH**