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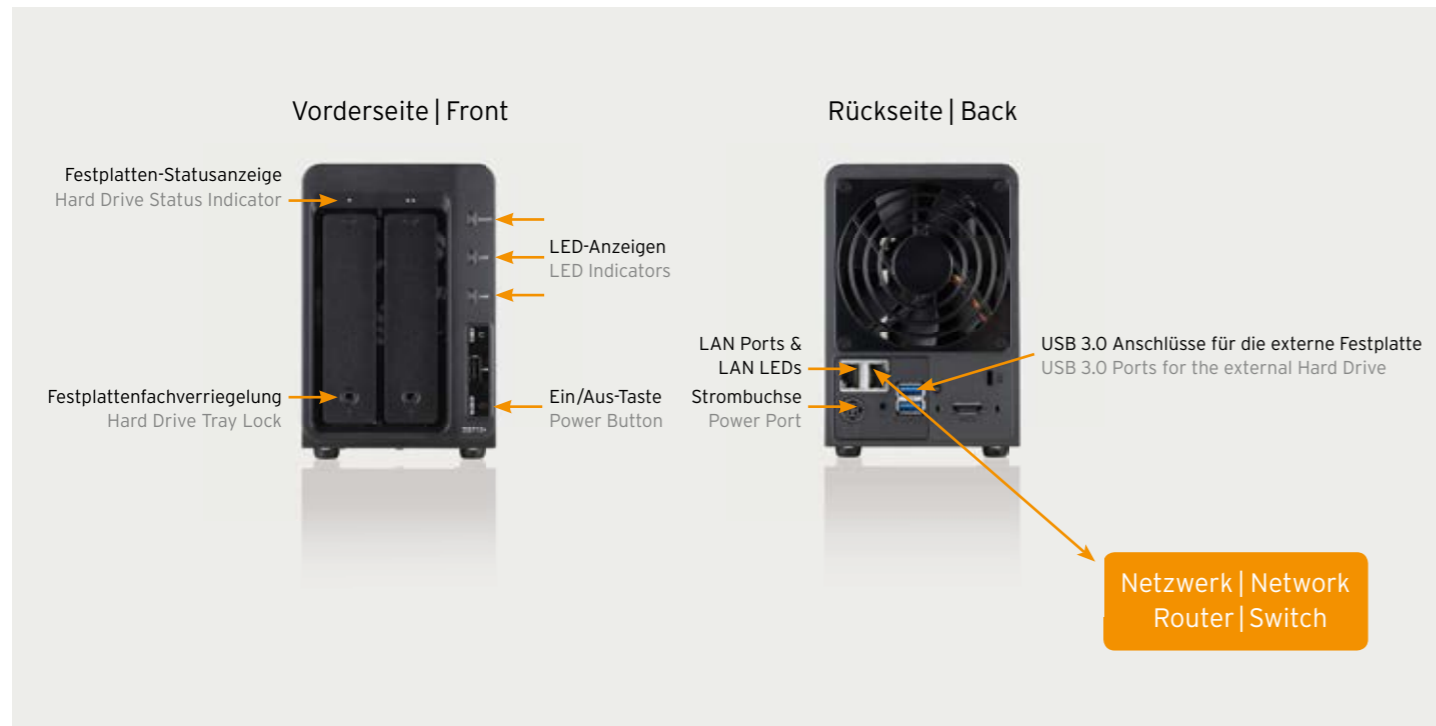
Connection diagram | Diagramme de raccordement | Schema di collegamento | Diagrama de conexiones

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These system requirements must be guaranteed by an IT Administrator from the customer:

- _ A gigabit network with automatic IP address allocation must be available.
- _ The existing computer must also be connected to the network.
- _ Check whether the latest version of Ceramill Mind is installed.
- _ Installation can only be completed as Administrator.
- _ The network and file sharing must be activated on the workstations. If in doubt, contact your computer administrator.

Ceramill Mindserve installation

Select the installation site, so that accidental manipulation of the connection cables can be excluded.

- _ The Ceramill Mindserve should be connected to the mains using the adapter supplied.
- _ Connect the network connections on the back of the Ceramill Mindserve with the existing switch and network distributor using the black network cables included (or at least with a Class CAT 5e UTP cable).
- _ The external back-up hard drive should be connected to one of the USB connections on the back of the Ceramill Server. The external 2TB hard drive is supplied with power via the Ceramill Mindserve. Connect the external 4TB hard drive to the mains using the adapter supplied.
- _ Press the red power button on the front.
- _ The blue LED flashes and the server starts. Wait approx. one minute until the blue LED lights up continuously, the Status LED flashes and there is a single beep signal. The Ceramill Server is now started and ready to use.
- _ Register as Administrator on your computer and use the following data:
User: Administrator / Password: ceramill mall
- _ The installer and manual are on the USB stick supplied.
- _ You can now insert the installation data carrier (USB stick) in a free USB port of your Ceramill PC and start the Ceramill mindserve.exe file.
- _ Update the Ceramill software to the latest version, if necessary.
- _ Start via the Start menu / All programmes / Ceramill software the Ceramill Mindserve programme and follow the instructions on the screen.

Please note: First read the manual to avoid loss of data. It is not possible to merge several databases. If you use several databases, you must decide for one or several databases on the Ceramill Mindserve.

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