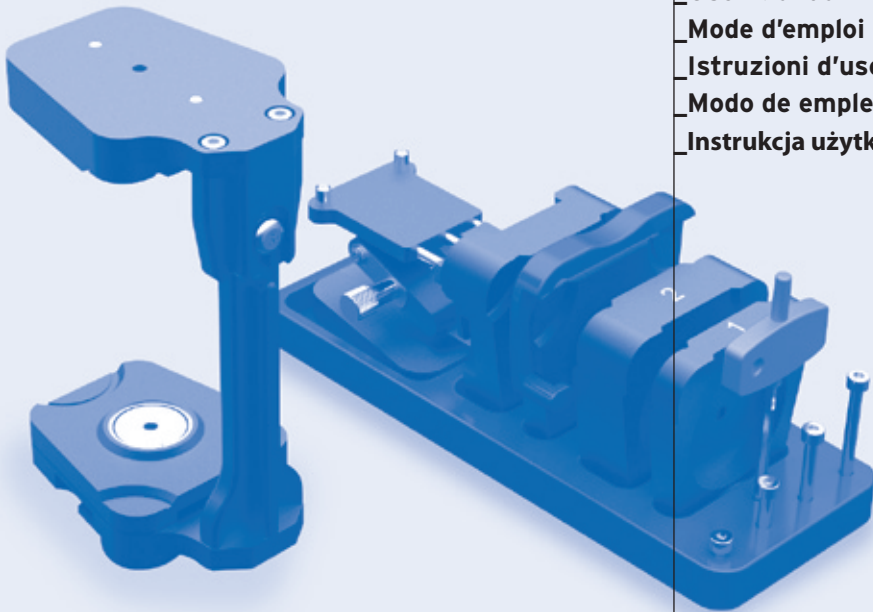
ceramill transferkit

Ceramill Map300

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Modo de empleo
Instrukcja użytkownika

AMANN GIRRBACH

Deutsch	04 - 05
English	06 - 07
Française	08 - 09
Italiano	10 - 11
Español	12 - 13
Polski	14 - 15

CERAMILL TRANSFERKIT

ANWENDUNG

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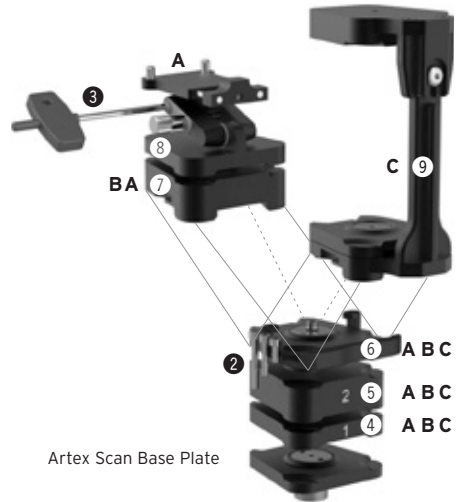
APPLICATION

CONTENTS AND OVERVIEW

- 1 Plate holder
- 2 Screws, various lengths according to distance plates
- 3 Allen Key
- 4 Distance plate 1 (10 mm) optional, according to model height
- 5 Distance plate 2 (20 mm) optional, according to model height
- 6 Fixation plate with clasp
- 7 Model plate
- 8 No Plast
- 9 Artex Fixator


- A Scan apparatus for models without articulator base
- B Scan apparatus for models with an articulator base
- C Scan apparatus for 2 models in relation to the articulator

Artex Scan Base Plate

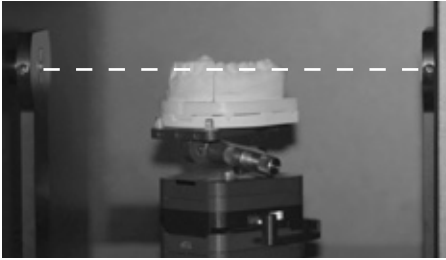The Artex Scan Base Plate is firmly attached to the Scanner Ceramill Map300. The distance plates 1 4 and 2 5 and the fixation plate with clasp 6 are screwed onto this as required. The type of distance plate and screw 2 required depend on the height of the model to be scanned. The model must be positioned within the scanner so that it lies directly in line with the scanning focus, at the level of the rotational axis.

All other plates must be clipped onto the fixation plate with the clasp.

Using the No Plast: Fix the model in the No Plast Loosen the screws at the middle of the No Plast and position the model in the scanning focus. Fix the position with the knurled head screw and then tighten with the Allen screw using the Allen Key ③.

COORDINATING THE ARTEX FIXATOR

Before the Artex Fixator can be used in the scanner, it must first be encoded using the Splitex Key.

Directions:

- _ Open fixator
- _ Place Splitex key onto upper section
- _ Place Splitex plate onto key
- _ Apply glue (Loctite) to the underside of the Splitex plate
- _ Spray activator onto the lower section of the fixator
- _ Close fixator down to the end position and fix it with the lateral screw
- _ Cure according to glueing instructions
- _ Open fixator
- _ Remove Splitex key

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Amann Girrbach AG
Amann Girrbach GmbH