SCANAccessories

DE Betriebsanleitung	3 - 11
EN User Manual	12 - 20
FR Mode d'emploi	21 - 29
IT Istruzioni d'uso	30 - 38
ES Modo de empleo	39 - 47
PT Instruções de operação	48 - 56

M-Die

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DE

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DE

Table of Contents

Explanation of Symbols	12
Suitable Personnel	13
Product Description	14
Usage	15
Accessories	20

Explanation of Symbols

Warning indications

Warning indications in the text are marked with a triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- _ **NOTE** means that property damage can occur.
- _ **CAUTION** means that light to fairly serious personal injury can occur.
- _ **WARNING** means that serious personal injury can occur.
- _ **DANGER** means that life-threatening personal injury can occur.

Important information

Important information that do not lead to hazards for humans or property damage are marked with the icon aside and are also boxed.

Other symbols in the Manual

Symbol	Meaning
▷	Item of an operation description
–	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

Suitable Personnel

NOTE:

Starting-up and operation of the machine may only be carried out by trained specialised personnel.

Product Description

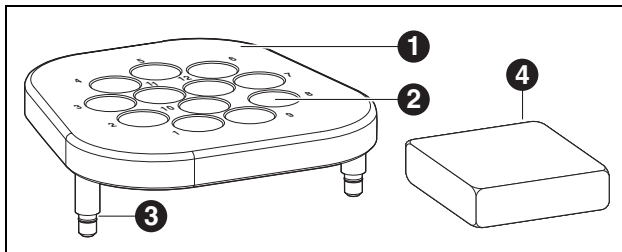
Fig. 1 Delivery scope

- | | |
|-------------------------------------|--------------------------|
| 1 M-Die | 3 Holding pin |
| 2 Compartments for abutments | 4 Fixing compound |

M-Die is an additional component for the “Intelligent Multi-Die” and “Multi-Cap” scanning processes:

- _ With “Intelligent Multi Die”, up to twelve abutments of a patient can be captured in a single scan.
- _ With “Multi-Cap”, up to twelve jaw and patient-independent single abutments can be captured.

These indications are limited to off-set copings.

M-Die can only be used in conjunction with the universal holder.

Usage

The usage is described in the “Intelligent Multi Die” mode.

A gap is provided for between the M-Die and the universal holder. This gap allows for placement of the M-Die onto the universal holder without having to firstly remove the modeling wax from the holder.

Global Scan

The placement of the model on the universal holder should be performed outside of the scanner.

▷ Fasten the model to be scanned (upper or lower jaw) centrally on the universal holder.

Always position models in such a manner that they are within the scan focus.

▷ Position the universal holder into the scanner.

▷ Scan the models.

Positioning abutments

- ▷ Mount the M-Die [3] onto the universal holder [4].
- ▷ Fill fixing compound [1] into the M-Die compartments.
- ▷ At the position indicated in the software [3], work the abutments to be scanned [2] centrally into the M-Die (here compartment 7).

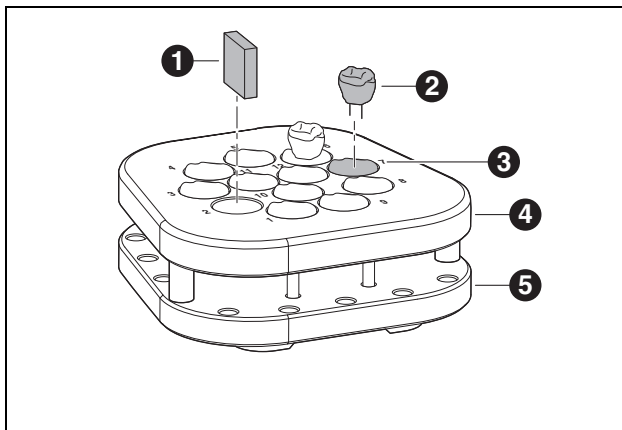
Fig. 2 Positioning abutments

- 1** Fixing compound
- 2** Abutment
- 3** Compartment 7
- 4** M-Die
- 5** Universal holder

- ▷ Press the abutments into the fixing compound in such a manner that they face flat and level against the surface of the M-Die ([A]).
- Ensure that both fixing compound and abutment do not project out above the surface of the M-Die/compartments [B].
 - Make sure that there is no gap between abutment and compound [C].
 - Do not press abutment too deep into compound [D].

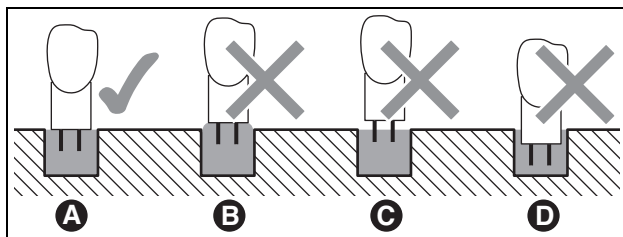
Fig. 3 Press-in-depth of the abutments

Scanning

- ▷ Depending on scanner, place the universal holder with M-Die directly onto the scan plate or model plate.

The integrated magnet provides for sufficient fixing of the universal holder.

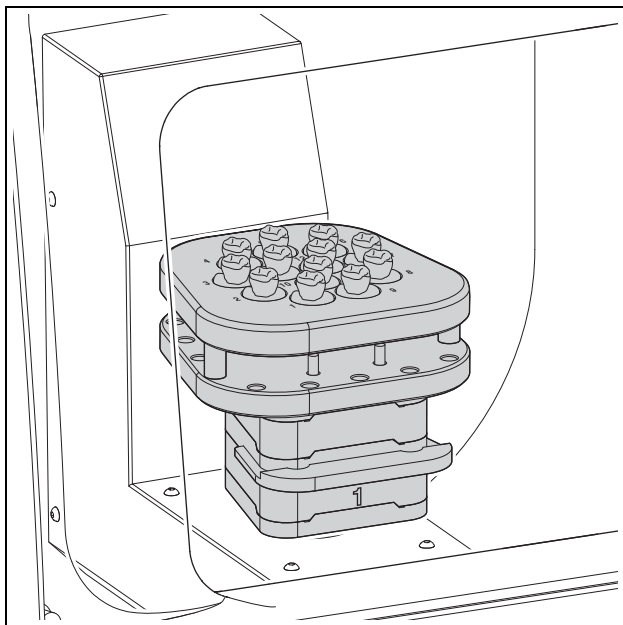
Fig. 4 Universal holder with M-Die on model plate

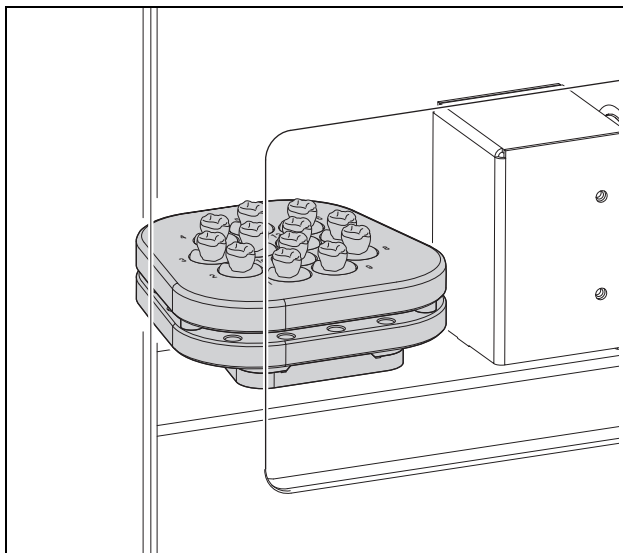
Fig. 5 Universal holder with M-Die on scan plate

▷ Scan the abutments in the “Intelligent Multi Die” mode.

The scanned abutments are automatically assigned to the correct position in the dental arch when scanning in “Intelligent Multi Die” mode.

▷ To remove the universal holder from the scanner: Grasp the holder at the rear or by the side and lift it off.

Accessories

Art. No.	Designation
179118	Universal holder
179123	Modelling wax
179116	Fixing compound

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Amann Girrbach AG