ceramill[®] a-splint

DE Gebrauchsanweisung	3 - 11
EN Instruction Manual	12 - 20
FR Instructions d'utilisation	21 - 29
IT Istruzioni per l'uso	30 - 38
ES Instrucciones de uso	39 - 47

- Original Gebrauchsanweisung -

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- Translation of the original Operating Instructions -

Table of Contents

Explanation of Symbols	13
General Safety Instructions	15
Suitable Personnel	15
Properties	16
Warranty/Exclusion of Liability	18
Testing the Blanks	18
Mounting to the Worktable	18
Application	18
Environmental Protection	20
Accessories	20
Download information	20

Explanation of Symbols

Warning indications

Warning indications in the text are marked with a triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- _ **NOTE** means that property damage can occur.
- _ **CAUTION** means that light to fairly serious personal injury can occur.
- _ **WARNING** means that serious personal injury can occur.
- _ **DANGER** means that life-threatening personal injury can occur.

Important information

Important information that do not lead to hazards for humans or to property damage are marked with the icon aside and are boxed.

Other symbols in the Manual

Symbol	Meaning
▷	Item of an operation description
—	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

Other symbols on the product

Symbol	Meaning
	Catalog number
	Batch code
	Manufacturer
	Consult instructions for use
	Use by YYYY-MM-DD or YYYY-MM
Rx only	Symbol that may be used in place of statement “CAUTION”. US Federal law restricts this device to sale by or on the order of a dentist.
	Serial number

General Safety Instructions

CAUTION:

Health impairment due to dust from PMMA!

When processing constructions made of Ceramill A-Splint, dusts can develop that may lead to mechanical irritation of the eyes and respiratory tract.

- ▷ Always make sure that the extraction system for the milling machine and the workplace for individual reworking operates properly.
- ▷ When processing, wear personal protective equipment (dust protection mask, safety glasses/goggles, ...).
- ▷ Further safety-relevant information can be found in the safety data sheet.

Suitable Personnel

NOTE:

The product may only be processed by trained dental technicians/dentists.

Properties

Intended Use

Ceramill A-Splint are ready to use, polymethyl methacrylate based CAD-CAM blanks, for milling splints and for verifying the fit on the plaster model/mouth before the final restoration is manufactured.

Product Description

Ceramill A-Splint are milling blanks for the production of splints (occlusion splints). These splints eliminate overloading and/or incorrect loading of the tooth substance, the periodontium and the mandibular joints. The maximum wearing period is three years.

Safety Data Sheet/Declaration of Conformity

The safety data sheet is available for download on the Amann Girrbach website under *Services > Downloads > Ergänzende Unterlagen*. The product's declaration of conformity is available upon request from the manufacturer.

Technical Data

	Unit	Ceramill A-Splint (DIN EN ISO 20795-2)
3-point bending strength	MPa	> 100
Density	g/cm ³	1.19
Vickers hardness	HV0,2	24
Water absorption	μg/mm ³	< 25
Solubility	μg/mm ³	< 0.7
Residual monomeric content	%	< 1
Fracture toughness	MPa√m	> 3.5

Chemical composition

_ Polymethylmethacrylate (PMMA)

Shelf life of the blanks

When appropriately stored, the blanks can be used 4 years after the manufacturing date.

Storage

- _ Store only in original packaging.
- _ Store separated from foodstuffs.
- _ Keep container tightly sealed.

Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

Testing the Blanks

Upon receipt, it is imperative to visually check the proper condition of the blanks. After using a (transport-) damaged blank, complaint claims are no longer possible.

Mounting to the Worktable

The mounting of the blanks is described in the Operating Instructions of the respective milling machine.

Application

Indications

- _ Splints for therapeutic care for the correction of temporomandibular joint problems and the chewing level with a maximum wearing period of three years.
- _ Verifying the fit on the plaster model/mouth before the final restoration.

Contraindications

- _ Use for definitive restorations
- _ Known incompatibilities with the components
- _ All indications not listed under "Indications"

Cutting Out and Processing

- ▷ Use cross-toothed tungsten carbide cutters or cutting discs suitable for plastic, for cutting out and processing constructions from the blank.
- ▷ Carefully polish the surface to a high gloss.
- ▷ To avoid fitting inaccuracies after the construction has been manufactured: Prevent intense heat generation during processing and polishing.

Expanding

Constructions made of Ceramill A-Splint can be reconstructed and supplemented at any time with all commercially available cold-curing polymers based on MMA.

- ▷ Condition the surface beforehand, preferably with a comparable bonding agent on the basis of MMA.
- ▷ Follow the manufacturer's instructions of the expansion-material supplier.

Environmental Protection

Packaging

In terms of packaging, the manufacturer participates in country-specific recycling systems, which ensure optimal recycling.

All packaging materials used are environmentally-friendly and recyclable.

Disposal of the blanks

Dispose of the contents/container in accordance with local/regional/international regulations.

Always dispose of according to official regulations.

Accessories

For more information on product-specific accessories, please refer to www.amanngirrbach.com.

Download information

Further instructions/manuals can be downloaded under www.amanngirrbach.com/instruction-manuals.

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Amann Girrbach AG
Amann Girrbach GmbH