 **ceramill® a-temp**  
 **ceramill® a-temp multilayer**

<b>DE</b> Gebrauchsanweisung	3 - 13
<b>EN</b> Instruction Manual	14 - 24
<b>FR</b> Instructions d'utilisation	25 - 35
<b>IT</b> Istruzioni per l'uso	36 - 46
<b>ES</b> Instrucciones de uso	47 - 57

- Original Gebrauchsanweisung -

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- Translation of the original Operating Instructions -

## **Table of Contents**

<b>Explanation of Symbols</b> .....	<b>15</b>
<b>General Safety Instructions</b> .....	<b>17</b>
<b>Suitable Personnel</b> .....	<b>17</b>
<b>Properties</b> .....	<b>18</b>
<b>Warranty/Exclusion of Liability</b> .....	<b>20</b>
<b>Testing the Blanks</b> .....	<b>20</b>
<b>Mounting to the Worktable</b> .....	<b>20</b>
<b>Application</b> .....	<b>21</b>
<b>Fixation</b> .....	<b>24</b>
<b>Environmental Protection</b> .....	<b>24</b>
<b>Accessories</b> .....	<b>24</b>
<b>Download information</b> .....	<b>24</b>

## Explanation of Symbols

### Warning indications

Warning indications in the text are marked with a triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- \_ **NOTE** means that property damage can occur.
- \_ **CAUTION** means that light to fairly serious personal injury can occur.
- \_ **WARNING** means that serious personal injury can occur.
- \_ **DANGER** means that life-threatening personal injury can occur.

### Important information

Important information that do not lead to hazards for humans or to property damage are marked with the icon aside and are boxed.

## Other symbols in the Manual

Symbol	Meaning
▷	Item of an operation description
—	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

## Other symbols on the product

Symbol	Meaning
<b>REF</b>	Catalog number
<b>LOT</b>	Batch code
	Manufacturer
	Consult instructions for use
	Use by YYYY-MM-DD or YYYY-MM
<b>Rx only</b>	Symbol that may be used in place of statement “CAUTION”. US Federal law restricts this device to sale by or on the order of a dentist.
<b>SN</b>	Serial number

## General Safety Instructions

### CAUTION:

Health impairment due to dust from PMMA!

When processing constructions made of Ceramill A-Temp/A-Temp ML, dusts can develop that may lead to mechanical irritation of the eyes and respiratory tract.

- ▷ Always make sure that the extraction system for the milling machine and the workplace for individual reworking operates properly.
- ▷ When processing, wear personal protective equipment (dust protection mask, safety glasses/goggles, ...).
- ▷ Further safety-relevant information can be found in the safety data sheet.

### NOTE:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## Suitable Personnel

### NOTE:

The product may only be processed by trained dental technicians/dentists.

## Properties

### Intended use

Ceramill A-Temp/A-Temp ML are ready to use, polymethyl methacrylate based CAD-CAM blanks, for milling long-term temporary crowns and bridges and for verifying the fit on the plaster model/mouth before the final restoration is manufactured.

### Product Description

Ceramill A-Temp/A-Temp ML are tooth-coloured milling blanks. These replace lost tooth substance in permanent dentures in the form of temporary crowns and bridges in the anterior and posterior region. The maximum wearing period is three years.

### Safety Data Sheet/Declaration of Conformity

The safety data sheet is available for download on the Amann Girrbach website under *Services > Downloads > Additional documents*. The product's declaration of conformity is available upon request from the manufacturer.

**Technical Data**

**Ceramill A-Temp/A-Temp ML**

	<b>Unit</b>	<b>(DIN EN ISO 10477)</b>
3-point bending strength	MPa	> 135
Density	g/cm <sup>3</sup>	1.19
Vickers hardness	HV0.2	24
Water absorption	µg/mm <sup>3</sup>	< 25
Solubility	µg/mm <sup>3</sup>	< 0.6
Residual monomeric content	%	< 1

**Chemical composition**

- \_ Polymethylmethacrylate (PMMA)
- \_ Colour pigments

**Shelf life of the blanks**

When appropriately stored, the blanks can be used 4 years after the manufacturing date.

**Storage**

- \_ Store only in original packaging.
- \_ Store separated from foodstuffs.
- \_ Keep container tightly sealed.

## **Warranty/Exclusion of Liability**

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

## **Testing the Blanks**

Upon receipt, it is imperative to visually check the proper condition of the blanks. After using a (transport-) damaged blank, complaint claims are no longer possible.

## **Mounting to the Worktable**

The mounting of the blanks is described in the Operating Instructions of the respective milling machine.

We recommend the following CAD/CAM systems to be used for fabricating our device: Ceramill Mikro 4X, Ceramill Mikro 5X, Ceramill Motion 2, Ceramill Matik.

## Application

### Indications

- \_ Temporary anterior and posterior crowns with a wearing time of maximum three years
- \_ Temporary anterior and posterior bridges with a maximum of two connected pontics and a maximum wearing time of three years
- \_ Verifying the fit on the plaster model/mouth before the final restoration

### Contraindications

- \_ Bridge constructions with more than two connected pontics
- \_ Use for definitive restorations
- \_ Known incompatibilities with the components
- \_ All indications not listed under "Indications"

### Material-specific frame parameters


The following material-specific frame parameters must be complied with when fabricating restorations with Ceramill A-Temp/A-Temp ML:

Minimum wall thickness in mm		Connector cross-section in mm <sup>2</sup>			
		Front-tooth bridge		Posterior bridge	
Occlusal	Circular	with one intermediate unit	with two intermediate units	with one intermediate unit	with two intermediate units
1.5	0.8	12	12	12	16

### Shade selection and placement of the restoration in the blank Ceramill A-Temp ML

The tooth shade of the blank covers off the imprinted area (e. g. O/A1). When a higher blank is selected, the respective lighter or darker shade can be achieved more significantly by vertically moving in the nesting. The closer the work is positioned to the imprinted side, the larger the lighter shade portion will be.

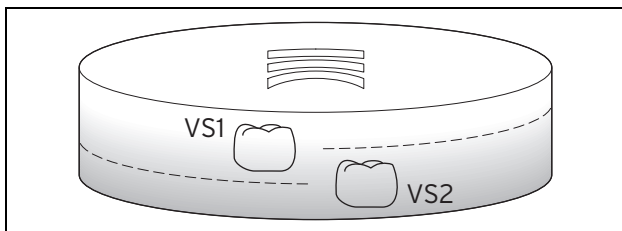*Fig. 1 Placement of the restoration in the blank*

**VS1** Tooth shade 1 (lighter)

**VS2** Tooth shade 2 (darker)

### **Cutting Out and Processing**

- ▷ Use cross-toothed tungsten carbide cutters or cutting discs suitable for plastic, for cutting out and processing constructions from the blank.
- ▷ Carefully polish crown margins, occlusal surfaces and basal contact surfaces to a high gloss.
- ▷ To avoid fitting inaccuracies after the construction has been manufactured: Prevent intense heat generation during processing and polishing.

### **Veneering**

Milled constructions made of Ceramill A-Temp/A-Temp ML can be veneered with conventional veneering materials.

- ▷ Prefer materials based on MMA.
- ▷ Follow the manufacturer's instructions of the veneering-material supplier.

When veneering with composite, the connection to Ceramill A-Temp/A-Temp ML must always be made according to the instructions of the respective manufacturer.

### **Relining**

Constructions made of Ceramill A-Temp/A-Temp ML can be relined at any time with all commercially available cold-curing polymers based on MMA.

- ▷ Condition the surface beforehand, preferably with a comparable bonding agent on the basis of MMA.
- ▷ Follow the manufacturer's instructions of the relining-material supplier.

## Fixation

\_ Provisional, eugenol-free cements/fixing materials

## Environmental Protection

### Packaging

In terms of packaging, the manufacturer participates in country-specific recycling systems, which ensure optimal recycling.

All packaging materials used are environmentally-friendly and recyclable.

### Disposal of the blanks

Dispose of the contents/container in accordance with local/regional/international regulations.

Always dispose of according to official regulations.

## Accessories

For more information on product-specific accessories, please refer to [www.amanngirrbach.com](http://www.amanngirrbach.com).

## Download information

Further instructions/manuals can be downloaded under [www.amanngirrbach.com/instruction-manuals](http://www.amanngirrbach.com/instruction-manuals).

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**Amann Girrbach AG**
**Amann Girrbach GmbH**