# ceramill® liquid

<b>DE</b> Gebrauchsanweisung	3 - 18
<b>EN</b> Instruction Manual	19 - 33
<b>FR</b> Instructions d'utilisation	34 - 50
<b>IT</b> Istruzioni per l'uso	51 - 69
<b>ES</b> Instrucciones de uso	70 - 86

- Original Gebrauchsanweisung -

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- Translation of the original Operating Instructions -

## Table of Contents

<b>Explanation of Symbols</b> .....	<b>20</b>
<b>General Safety Instructions</b> .....	<b>22</b>
<b>Suitable Personnel</b> .....	<b>23</b>
<b>Warranty/Exclusion of Liability</b> .....	<b>23</b>
<b>Properties</b> .....	<b>24</b>
<b>Application</b> .....	<b>26</b>
<b>Pre-drying and Sintering</b> .....	<b>29</b>
<b>Environmental Protection</b> .....	<b>32</b>
<b>Accessories</b> .....	<b>33</b>
<b>Download Infos</b> .....	<b>33</b>

EN

## Explanation of Symbols

### Warning indications

Warning indications in the text are marked with a colour-backed triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- \_ **NOTE** means that property damage can occur.
- \_ **CAUTION** means that light to fairly serious personal injury can occur.
- \_ **WARNING** means that serious personal injury can occur.
- \_ **DANGER** means that life-threatening personal injury can occur.

### Important information

Important information that do not lead to hazards for humans or property damage are marked with the icon aside and are also boxed.

**Other symbols in the Manual**

Symbol	Meaning
▷	Item of an operation description
—	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

**Other symbols on the product**

Symbol	Meaning
	Catalog number
	Batch code
	Manufacturer
	Consult instructions for use
	Date of manufacture
	Health hazard/Hazardous to the ozone layer
<b>Rx only</b>	Symbol that may be used in place of statement “CAUTION”. US Federal law restricts this device to sale by or on the order of a dentist.

## General Safety Instructions

EN

**CAUTION:**

Health hazard!

Harmful if swallowed.

Causes severe skin burns and eye damage.

May cause respiratory irritation.

▷ Avoid breathing dust/fume/gas/mist/vapours/spray.

▷ Wear protective gloves/protective clothing/eye protection/face protection.

▷ IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing.

**NOTE:**

Damage to the environment!

Toxic to aquatic life with long lasting effects.

May be corrosive to metals.

**NOTE:**

As of 2014, Ceramill Liquids new formula with modified basic formula in 16 A-D shades will specifically be available for Ceramill zirconium oxide. The Ceramill zirconium oxide Preshades are not released for application with Liquids.

- ▷ Use Ceramill Liquids new formula only with white Ceramill zirconium oxide.
- ▷ Do not mix Ceramill Liquids new formula A1 - D4 with the old Ceramill Liquids CL1 - CL4.
- ▷ Observe the respective Ceramill Liquids Instruction Manual.

## Suitable Personnel

**NOTE:**

The product may only be used by trained dental technicians.

## Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

## Properties

### Intended use

Dye solutions for staining zirconium oxide blanks.

### Product description

Ceramill Liquid new formula are water-based dye solutions for Ceramill zirconium oxide, for complete or partial staining of milled and pre-sintered frames of white Ceramill zirconium oxide.

Ceramill Liquids new formula are available in 16 base shades and 5 effect shades:

- \_ 16 Vita classical shades (A-D)
- \_ Effect shade OR (orange) for the fissure and cervical area
- \_ Effect shades GR (grey) and VIO (violet) for the cusp area
- \_ Effect shades RO (rose) and GIN (gingiva) for the gum area

### Safety data sheet/Declaration of conformity

The data sheet on safety and the declaration of conformity are available on request under [www.amangirrbach.com](http://www.amangirrbach.com).

### Shelf life

Ceramill Liquids new formula have a shelf life of 2 years.

The expiry date and batch number are indicated on the original packaging.

### Storage

Store in the original packaging in a dry location.

Within their shelf life, the dye solutions can change their natural colour during storage or usage. The colouration of the sintered restoration is not affected through this.

**Packaging sizes**

<b>Art. No.</b>	<b>Designation</b>	<b>Description</b>
760469	Ceramill Liquid Set update	(A2, A3, A3,5, B3, OR, GR, RO, VIO, Ceramill Liquid Brush Size 1, Ceramill Liquid Brush Size 3, plastic tweezers, eight dyeing cans, timer)
760430	Ceramill Liquid A1	With dyeing can
760432	Ceramill Liquid A2	With dyeing can
760433	Ceramill Liquid A3	With dyeing can
760434	Ceramill Liquid A3.5	With dyeing can
760435	Ceramill Liquid A4	With dyeing can
760436	Ceramill Liquid B1	With dyeing can
760437	Ceramill Liquid B2	With dyeing can
760438	Ceramill Liquid B3	With dyeing can
760439	Ceramill Liquid B4	With dyeing can
760440	Ceramill Liquid C1	With dyeing can
760442	Ceramill Liquid C2	With dyeing can
760443	Ceramill Liquid C3	With dyeing can

Tab. 1

Art. No.	Designation	Description
760444	Ceramill Liquid C4	With dyeing can
760445	Ceramill Liquid D2	With dyeing can
760446	Ceramill Liquid D3	With dyeing can
760447	Ceramill Liquid D4	With dyeing can
760449	Ceramill Dimmer Liquid Refill	
760488	Ceramill Liquid OR	Colour orange, with dyeing can
760489	Ceramill Liquid GR	Colour grey, with dyeing can
760485	Ceramill Liquid RO	Colour pink, with dyeing can
760486	Ceramill Liquid GIN	Colour gingiva, with dyeing can
760487	Ceramill Liquid VIO	Colour gingiva, with dyeing can

Tab. 1

## Application

The shade allocations of the dye solutions were produced based on sample restorations with a standardised frame thickness **GS** of 1.2 mm. For this frame thickness, the Ceramill Liquids 5 × were applied on the inside and outside of the crowns with a size 3 brush. For restorations with a thinner or thicker GS, the liquid must be applied accordingly less or more often.

The shade allocations are to be considered a suggestion on behalf of Amann Girschbach. The shade perception is subjective and can differ considerably from user to user. Before staining the objects, it is recommended to create colour samples.

## Colouring of frames through immersion

### NOTE:

Before staining:

- ▷ Clean the frames from residual zirconia dust using a brush and compressed air.

### NOTE:

Ensure there are no metal traces on the objects.

- ▷ Use only plastic tweezers for immersing.

Process the milled frames of Ceramill Zolid as follows:

- ▷ Select the desired dye solution according to colour coding A-D.
- ▷ To achieve the desired dentine shade, thin the selected colouring liquid A-D with 25 % of Ceramill Dimmer Liquid (760449).
- ▷ Immerse the frame for 10 seconds in the dye solution.
- ▷ Dab off excess liquid with a lint-free disposable cloth.
- ▷ Recommendation: Empfehlung: Pre-dry for 1 h (see page 29)
- ▷ Sinter in the Ceramill Therm (178300/178350) in the Zolid sintering bowl (178370).

For further information on the immersion procedure, please refer to the step-by-step instructions. These can be downloaded under [www.amanngirrbach.com](http://www.amanngirrbach.com).

Further procedures on the colouring technique of Ceramill Zolid fx and important information and instructions on the material Ceramill Zolid fx can be downloaded under [www.amanngirrbach.com](http://www.amanngirrbach.com).

## Staining of monolithic crowns and bridges with brush technique

### NOTE:

Before staining:

- ▷ Clean the frames from residual zirconia dust using a brush and compressed air.

### NOTE:

Ensure there are no metal traces on the objects.

Metal ferrules on the brush can leave traces on the zirconium oxide and additionally influence the dye solution!

- ▷ Use only metal-free brushes with synthetic bristles.

Brushes with synthetic bristles are less sensitive against dye solutions and alcohol than brushes with real hair.

- ▷ As soon as a new mixing ratio or a different shade is being used, wash out reusable brushes with alcohol (e.g. ethanol) and afterwards with distilled water.

For detailed information on the brush technique and combination with the immersion technique for work with monolithic component, please refer to the illustrated step-by-step Ceramill Zolid instructions. These can be downloaded under [www.amanngirrbach.com](http://www.amanngirrbach.com).

## Pre-drying and Sintering

### Pre-drying

After staining with Ceramill Liquid, the Ceramill Zolid should be pre-dried. This is to avoid the formation of spots (homogenisation). For large jobs, the risk of cracks and fissures is reduced.

Material	Pre-drying	
	Temperature	Duration
Ceramill Zolid frames (without auxiliary sintering block)	80 °C	60 min

Tab. 2

### Suitable equipment

After staining the white Ceramill Zolid, the work can be dried using a ceramic furnace. The ceramic furnace and the further ceramic firing cycles are not influenced through the drying procedure. However, it is recommended to occasionally burn out the ceramic furnace at 1150 °C for 5 minutes. As an alternative, a drying cabinet or mini oven can also be used.

Suitable equipment for pre-drying:

- \_ Domestic ovens/mini ovens ✓✓✓
- \_ Drying cabinet ✓✓✓
- \_ Ceramic furnace ✓✓
  - In case the ceramic furnace cannot be programmed to 80 °C, enter the lowest possible temperature and leave furnace door open.
  - Use a thermometer to check, whether the 80 °C are reached on the object.
- \_ Ceramill Therm (178350 - Version since 2008) ✓
- \_ Curing lamps ✓
  - Curing lamps generally do not achieve the recommended drying temperature. Therefore, when using curing lamps, the drying duration must be accordingly prolonged.

## Sintering

### Basic rule for sintering multi-unit restorations:

All stained restorations **without** auxiliary sintering block (approx. to 9 units) can principally be sintered in standard program 1. After staining, all work **with** auxiliary sintering block require the slower sintering program 4 indicated below. In the Ceramill Therm 2, this must be programmed on program position 4. Smaller jobs can be integrated in the sintering process in this program.

### Adjustments on the sintering furnace

▷ Adjust the following parameters on the sintering furnace:

Material	Sintering furnace	Sintering program	Recommended pre-drying
Ceramill Zolid without sintering block	Therm 1	Program 1	1 h at 80 °C (external, e.g. ceramic furnace)
	Therm 2	Program 1	1 h at 80 °C (external, e.g. ceramic furnace or programmable on program position 4)
Ceramill Zolid with sintering block	Therm 1	not possible	-
	Therm 2	Program 4 <sup>1)</sup>	Pre-drying already included in the program's slow heating phase

Tab. 3 Sintering temperatures and durations

- 1) 3 °C/min heating rate, 2 h dwell time at 1450 °C, long-term cooling as in program 3

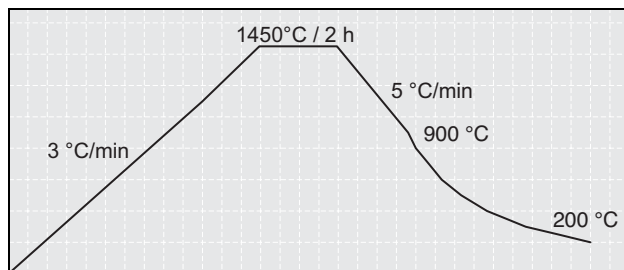
Fig. 1 Sintering program Ceramill Zolid in the Ceramill Therm 2 for large bridges with auxiliary sintering block

- ▷ Sinter in the Ceramill Therm (178300/178350) in the Zolid sintering bowl (178370).

**NOTE:**

Loss of translucence!

- ▷ The sintering bowl with the pearls are to be used only for Ceramill Zolid and **not** for Ceramill ZI applications!
- ▷ Do not sinter Ceramill Zolid and Ceramill ZI frames together in one sintering bowl!

Ceramill Zolid frames and ZI frames may be sintered together in one sintering process in separate, stacked sintering bowls. Sintering bowls can only be stacked in the Ceramill Therm 2 (178350).

Restorations with Ceramill Liquids new formula and Ceramill Liquid old formula can be sintered together, yet in separate sintering bowls. In this case, the pre-drying and the sintering program are to be observed, and if required, to be adapted.

## Environmental Protection

### Packaging

In terms of packaging, Amann Girrbach participates in country-specific recycling systems, which ensure optimal recycling.

All packaging materials used are environmentally-friendly and recyclable.

### Disposal of completely emptied containers

Non-contaminated and emptied containers can be sorted for environmental-friendly recycling.

Always dispose of according to official regulations.

### Disposal of non-cleaned, emptied containers

This product and its container are to be disposed of as hazardous waste.

Dispose of non-cleanable packaging properly.

Dispose of according to official regulations for uncleaned packaging.

## Accessories

For more information on product-specific accessories, please refer to [www.amanngirrbach.com](http://www.amanngirrbach.com).

## Download Infos

Further instructions/manuals and video tutorials can be downloaded under [www.ceramill-m-center.com](http://www.ceramill-m-center.com) and/or [www.amanngirrbach.com](http://www.amanngirrbach.com).

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**Amann Girrba**ch AG
**Amann Girrba**ch GmbH