ceramill® liquid fx

Gebrauchsanweisung
Instruction Manual
Instructions d'utili-
sation
Istruzioni per l'uso
Instrucciones de uso

Deutsch	3 - 17
English	18 - 31
Français	32 - 47
Italiano	48 - 62
Español	63 - 77

- Original Gebrauchsanweisung -

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- Translation of the original Operating Instructions -

Table of Contents

Explanation of Symbols	19
General Safety Instructions	21
Suitable Personnel	22
Warranty/Exclusion of Liability	22
Properties	22
Application	25
Pre-drying and Sintering	28
Environmental Protection	30
Accessories	31
Download Infos	31

Explanation of Symbols

Warning indications

Warning indications in the text are marked with a colour-backed triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- _ **NOTE** means that property damage can occur.
- _ **CAUTION** means that light to fairly serious personal injury can occur.
- _ **WARNING** means that serious personal injury can occur.
- _ **DANGER** means that life-threatening personal injury can occur.

Important information

Important information that do not lead to hazards for humans or property damage are marked with the icon aside and are also boxed.

Other symbols in the Manual

Symbol Meaning

▷	Item of an operation description
–	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

Other symbols on the product

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	Catalog number
	Batch code
	Manufacturer
	Consult instructions for use
	Date of manufacture
	Health hazard/Hazardous to the ozone layer
Rx only	Symbol that may be used in place of statement “CAUTION”. US Federal law restricts this device to sale by or on the order of a dentist.

General Safety Instructions

CAUTION:

Health hazard!

Harmful if swallowed.

Causes severe skin burns and eye damage.

May cause respiratory irritation.

- ▷ Avoid breathing dust/fume/gas/mist/vapours/spray.
- ▷ Wear protective gloves/protective clothing/eye protection/face protection.
- ▷ IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

NOTE:

Damage to the environment!

Toxic to aquatic life with long lasting effects.

May be corrosive to metals.

NOTE:

- ▷ Use Ceramill Liquids FX only with white Ceramill Zolid FX.
- ▷ Do not mix Ceramill Liquid FX with other Ceramill Liquids.
- ▷ Observe the respective Ceramill Liquids Instruction Manual.
- ▷ Pre-stained blanks are not released for application with Ceramill Liquid FX.

Suitable Personnel

**NOTE:**

The product may only be used by trained dental technicians.

Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

Properties

Intended use

Dye solutions for staining zirconium oxide blanks.

Product description

Ceramill Liquid FX are water-based dye solutions for complete or partial staining of milled and pre-sintered frames of white Ceramill Zolid FX.

Ceramill Liquid FX are available in 16 base shades and 6 effect shades:

- _ 16 Vita classical shades (A-D)
- _ Effect shade OR (orange) for the fissure and cervical area
- _ Effect shades GR (grey), BL (blue) and VIO (violet) for the cusp area
- _ Effect shades RO (rose) and GIN (gingiva) for the gum area

Safety data sheet/Declaration of conformity

The data sheet on safety and the declaration of conformity are available on request under *www.amanngirrbach.com*.

Shelf life

Ceramill Liquids FX have a shelf life of 2 years.

The expiry date and batch number are indicated on the original packaging.

Storage

Store in the original packaging in a dry location.

 Within their shelf life, the dye solutions can change their natural colour during storage or usage. The colouration of the sintered restoration is not affected through this.

Packaging sizes

Art. No.	Designation	Description
761400	Ceramill Liquid FX Set	(A2, A3, A3.5, B3, OR, GR, BL, VIO, Ceramill Liquid Brush Size 1, Ceramill Liquid Brush Size 3, plastic tweezers, eight dyeing cans, timer)

Tab. 1

Art. No.	Designation	Description
761399	Ceramill Liquid FX Set/USA	(A2, A3, A3.5, B3, OR, GR, BL, VIO, Ceramill Liquid Brush Size 1, Ceramill Liquid Brush Size 3, plastic tweezers, eight dyeing cans, timer)
761401	Ceramill Liquid FX A1	With dyeing can
761402	Ceramill Liquid FX A2	With dyeing can
761403	Ceramill Liquid FX A3	With dyeing can
761404	Ceramill Liquid FX A3.5	With dyeing can
761405	Ceramill Liquid FX A4	With dyeing can
761406	Ceramill Liquid FX B1	With dyeing can
761407	Ceramill Liquid FX B2	With dyeing can
761408	Ceramill Liquid FX B3	With dyeing can
761409	Ceramill Liquid FX B4	With dyeing can
761410	Ceramill Liquid FX C1	With dyeing can
761411	Ceramill Liquid FX C2	With dyeing can
761412	Ceramill Liquid FX C3	With dyeing can
761413	Ceramill Liquid FX C4	With dyeing can
761414	Ceramill Liquid FX D2	With dyeing can
761415	Ceramill Liquid FX D3	With dyeing can
761416	Ceramill Liquid FX D4	With dyeing can
761417	Ceramill Liquid OR	Colour orange, with dyeing can

Tab. 1

Art. No.	Designation	Description
761418	Ceramill Liquid GR	Colour grey, with dyeing can
761419	Ceramill Liquid RO	Colour pink, with dyeing can
761420	Ceramill Liquid GIN	Colour gingiva, with dyeing can
761421	Ceramill Liquid VIO	Colour gingiva, with dyeing can
761422	Ceramill Liquid BL	Colour blue, with dyeing can

Tab. 1

Application

The shade allocations of the dye solutions were produced based on sample restorations with a standardised frame thickness **GS** of 1.2 mm. For this frame thickness **GS**, the Ceramill Liquid FX 5 x were applied on the inside and outside of the crowns with a size 5 brush.

For restorations with a thinner or thicker frame thickness **GS**, the Ceramill Liquid FX must be applied accordingly less or more often. Due to the high translucency of Ceramill Zolid FX, it is recommended to apply the Ceramill Liquid FX effect shades less intensively.

The shade allocations are to be considered a suggestion on behalf of Amann Girrbach. The shade perception is subjective and can differ considerably from user to user.

Before staining the objects, it is recommended to create colour samples.

Staining of monolithic crowns and bridges with brush technique

NOTE:

Before staining:

- ▷ Clean the frames from residual zirconia dust using a brush and compressed air.

NOTE:

Ensure there are no metal traces on the objects.

Metal ferrules on the brush can leave traces on the zirconium oxide and additionally influence the dye solution!

- ▷ Use only metal-free brushes with synthetic bristles.

Brushes with synthetic bristles are less sensitive against dye solutions and alcohol than brushes with real hair.

- ▷ As soon as a new mixing ratio or a different shade is being used, wash out reusable brushes with alcohol (e.g. ethanol) and afterwards with distilled water.

For detailed information on the brush technique and combination with the immersion technique for work with monolithic component, please refer to the illustrated step-by-step Ceramill Zolid instructions. These can be downloaded under www.amanngirrbach.com.

Colouring of frames through immersion

NOTE:

Before staining:

- ▷ Clean the frames from residual zirconia dust using a brush and compressed air.

NOTE:

Ensure there are no metal traces on the objects.

- ▷ Use only plastic tweezers for immersing.

Process the milled frames of Ceramill Zolid FX as follows:

- ▷ Select the desired dye solution according to colour coding A-D.
- ▷ To achieve the desired dentine shade, thin the selected colouring liquid A-D with 10 % of Ceramill Dimmer Liquid (760449).
- ▷ Immerse the frame for 10 seconds in the dye solution.
- ▷ Dab off excess liquid with a lint-free disposable cloth.
- ▷ Recommendation: Pre-dry for 1 h (see page 28).

The sintering bowl used for Ceramill Zolid can also be used with Ceramill Zolid FX.

- ▷ Sinter in the Ceramill Therm (178300/178350) in the Zolid/Zolid FX sintering bowl (178370).

For further information on the immersion procedure, please refer to the step-by-step instructions. These can be downloaded under www.amangirrbach.com.

Pre-drying and Sintering

Pre-drying

After staining with Ceramill Liquid FX, the Ceramill Zolid FX should be pre-dried. This is to avoid the formation of spots (homogenisation). For large jobs, the risk of cracks and fissures is reduced.

Material	Pre-drying	
	Temperature	Duration
Ceramill Zolid FX frames	80 °C	60 min

Tab. 2

Suitable equipment

After staining the white Ceramill Zolid FX, the work can be dried using a ceramic furnace. The ceramic furnace and the further ceramic firing cycles are not influenced through the drying procedure. However, it is recommended to occasionally burn out the ceramic furnace at 1150 °C for 5 minutes. As an alternative, a drying cabinet or mini oven can also be used.

Suitable equipment for pre-drying:

- _ Domestic ovens/mini ovens ✓✓
- _ Drying cabinet ✓✓✓
- _ Ceramic furnace ✓✓
 - In case the ceramic furnace cannot be programmed to 80 °C, enter the lowest possible temperature and leave furnace door open.

- Use a thermometer to check, whether the 80 °C are reached on the object.
- _ Ceramill Therm (178350 - Version since 2008) ✓
- _ Curing lamps ✓
 - Curing lamps generally do not achieve the recommended drying temperature. Therefore, when using curing lamps, the drying duration must be accordingly prolonged.

Sintering

Basic rule for sintering multi-unit restorations

Adjustments on the sintering furnace:

▷ Adjust the following parameters on the sintering furnace:

Material	Sintering furnace	Sintering program	Recommended pre-drying
Ceramill Zolid FX	Therm 1	Program 1	1 h at 80 °C (external, e.g. ceramic furnace)
	Therm 2	Program 1	1 h at 80 °C (external, e.g. ceramic furnace or program-mable on program position 4)

Tab. 3 Sintering temperatures and durations

▷ Sinter in the Ceramill Therm (178300/178350) in the Zolid/Zolid FX sintering bowl (178370).

**NOTE:**

Loss of translucence!

- ▷ Use this sintering bowl with sintering pearls only for Ceramill Zolid/Zolid FX and **not** for Ceramill ZI applications!
- ▷ Do not sinter Ceramill Zolid/Zolid FX frames and Ceramill ZI frames together in one sintering bowl!

Ceramill Zolid/Zolid FX and ZI frames may be sintered together in one sintering process in separate, stacked sintering bowls. In this case, the pre-drying and the sintering program are to be observed, and if required, to be adapted. Sintering bowls can only be stacked in the Ceramill Therm 2 (178350).

Environmental Protection

Packaging

In terms of packaging, Amann Girrbach participates in country-specific recycling systems, which ensure optimal recycling.

All packaging materials used are environmentally-friendly and recyclable.

Disposal of completely emptied containers

Non-contaminated and emptied containers can be sorted for environmental-friendly recycling.

Always dispose of according to official regulations.

Disposal of non-cleaned, emptied containers

This product and its container are to be disposed of as hazardous waste.

Dispose of non-cleanable packaging properly.

Dispose of according to official regulations for uncleaned packaging.

Accessories

For more information on product-specific accessories, please refer to www.amangirrbach.com.

Download Infos

Further instructions/manuals and video tutorials can be downloaded under www.ceramill-m-center.com and/or www.amangirrbach.com.

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Amann Girrbach AG
Amann Girrbach GmbH