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INSTRUCTIONS FOR USE Ceramiil TEMP, TOOTH SHADES

Translation of the original instructions for use. Please read carefully!

INTENDED USE

Ceramill TEMP blanks are milling blanks from which resin crowns, bridges and frameworks can be fabricated for dental restorations in temporary, pre-prosthetic treatment in the anterior and posterior regions.

TECHNICAL DATA

PHYSICAL VALUES FOR CERAMILL TEMP

Test according to

| | | |
|-----------------------|-------------------------|---------------------|
| Density | 1,19 g/cm ³ | EN ISO 10477 |
| Solubility | 0,2 µg/mm ³ | EN ISO 10477 |
| Flexural strength | 91,5 MPa | EN ISO 10477 |
| Water uptake | 26,5 µg/mm ³ | EN ISO 10477 |
| Modulus of elasticity | 2773 MPa | EN ISO 20795-1:2009 |
| Flexural strength | 96,6 MPa | EN ISO 20795-1:2009 |

COMPOSITION

- _ Polymethyl methacrylate (PMMA) and methacrylic acid ester-based cross-linked polymers
- _ Dye
- _ Residual peroxide
- _ Methyl methacrylate (MMA) may be contained up to a maximum of 1% as residual monomer

SHADES

Available in 4 different dentine shades (bleach, light, middle, dark).

| Blank | Tooth shade |
|--------|------------------------------------|
| bleach | BL1 |
| light | A1; B1; D2; C1 |
| middle | A2; A3; B2; B3; B4; C2; C3; D3; D4 |
| dark | A3,5; A4; C4 |

PACKAGE SIZES

| | | | | |
|--------|----------------------------|-----------|------------------|---------|
| 760320 | Ceramill TEMP light 71 XS | H = 13 mm | Dental arch form | 1 Blank |
| 760321 | Ceramill TEMP middle 71 XS | H = 13 mm | Dental arch form | 1 Blank |
| 760322 | Ceramill TEMP dark 71 XS | H = 13 mm | Dental arch form | 1 Blank |
| 760323 | Ceramill TEMP light 71 L | H = 20 mm | Dental arch form | 1 Blank |
| 760324 | Ceramill TEMP middle 71 L | H = 20 mm | Dental arch form | 1 Blank |
| 760325 | Ceramill TEMP dark 71 L | H = 20 mm | Dental arch form | 1 Blank |
| 760334 | Ceramill TEMP bleach 71 XS | H = 13 mm | Dental arch form | 1 Blank |
| 760335 | Ceramill TEMP bleach 71 L | H = 20 mm | Dental arch form | 1 Blank |

STORAGE

No special measures required.

HAZARD WARNINGS

Dust is created during milling of Ceramiil TEMP blanks that can cause irritation of the eyes, skin and airways. Therefore always ensure that the extractor on your milling machine is functioning perfectly. Further safety information can be obtained from the safety data sheet.

WEARING DURATION

Up to 12 months.

USER INFORMATION

Design:

- _ For crowns, do not fall below the minimum wall thickness of 1 mm.
- _ When designing bridges with one connected bridge unit, the connection cross-section between crown and connected bridge unit may not be less than 9 mm².
- _ When designing bridges with a range of up to two connected intermediate links, the connection cross-sections between crown and connected bridge units may not be less than 12 mm².
- _ The user must decide in each case whether the PMMA - OMP-N® material guarantees adequate masticatory stability for the restoration according to the indication by selection of the appropriate luting system.

Veneering Ceramiil TEMP frameworks:

- _ A milled Ceramiil TEMP framework can be faced using conventional veneering materials. MMA-based materials are the preferred choice for veneering.
- _ When veneering with composite, Ceramiil TEMP must always be bonded as described in the instructions of the respective manufacturer.

FINISHING

- _ Cross-cut tungsten carbide cutters or separating discs, which are suitable for acrylic, should be used for trimming and separating frameworks from the blank.
- _ Excessive heat build-up should be avoided during finishing and polishing to prevent fitting inaccuracies after fabrication of the framework.

RESTORING

Frameworks, full crowns and bridges, which have been fabricated using Ceramiil TEMP blanks, can be restored and adjusted at any time using any conventional MMA-based cold-curing resin such as artConcept®.

OPTIONS FOR LUTING INTRAORALLY

- _ for temporary restorations: temporary cements
- _ for long-term temporary restorations: glass ionomer cements without bonder

SPECIAL INFORMATION

Enter the LOT No. with each procedure that requires identification of the material.

DISPOSAL

Ceramill TEMP is insoluble in water, inactive, does not pose a risk for the ground water and can therefore be disposed of with normal household waste.

SAFETY DATA SHEETS / DECLARATION OF CONFORMITY

Safety data sheets and declaration of conformity are available on request and under www.amanggirrbach.com

Information as of 2014-01

The product properties are based on complying with and observing these instructions for use.

Manufacturer | Hersteller
Distribution | Vertrieb

Amann Girrbach AG
Herrschaftswiesen 1
6842 Koblach | Austria
Fon +43 5523 62333-105
Fax +43 5523 62333-519
austria@amanggirrbach.com
www.amanggirrbach.com

CE 0123

ISO 13485
ISO 9001

AMANNGIRRBACH