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## INSTRUCTIONS FOR USE Ceramill TEMP Multilayer TOOTH SHADES WITH COLOR GRADIENT

Translation of the original instructions for use. Please read carefully!

### INTENDED USE

Ceramill TEMP Multilayer blanks are milling blanks in tooth shades with color gradient from which resin crowns, bridges and frameworks can be fabricated for dental restorations in temporary, pre-prosthetic treatment in the anterior and posterior regions.

### TECHNICAL DATA

PHYSICAL VALUES FOR CERAMILL TEMP MULTILAYER

#### Test according to


Solubility	<0.8 µg/mm <sup>3</sup>
Bending strength	>100 MPa
Water uptake	<20 µg/mm <sup>3</sup>
Modulus of elasticity	2200 MPa
Tensile strength	>75 MPa
Residual monomer content	<0.7 %

### COMPOSITION

\_ Polymethyl methacrylate (PMMA)  
 \_ Dye

### SHADES

Available in 4 different dentine shades: A1/A2, A3, B1/B2 and C2/C3. Three of four blanks are available in double-colors. While nesting in the upper part of the blank the brighter color is achieved, in the lower part of the blank the darker color is achieved.

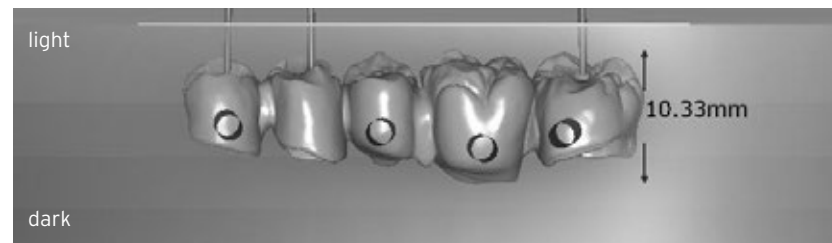Note: The software shows 4 color gradients for better illustration, although the material has several color gradients.

### PACKAGE SIZES

760380	Ceramill TEMP Multilayer A1/A2 71 XS	H = 13 mm	Dental arch form	1 Blank
760381	Ceramill TEMP Multilayer A3 71 XS	H = 13 mm	Dental arch form	1 Blank
760382	Ceramill TEMP Multilayer B1/B2 71 XS	H = 13 mm	Dental arch form	1 Blank
760383	Ceramill TEMP Multilayer C2/C3 71 XS	H = 13 mm	Dental arch form	1 Blank
760384	Ceramill TEMP Multilayer A1/A2 71 L	H = 20 mm	Dental arch form	1 Blank
760385	Ceramill TEMP Multilayer A3 71 L	H = 20 mm	Dental arch form	1 Blank
760386	Ceramill TEMP multilayer B1/B2 71 L	H = 20 mm	Dental arch form	1 Blank
760387	Ceramill TEMP multilayer C2/C3 71 L	H = 20 mm	Dental arch form	1 Blank

### STORAGE

Keep away from direct sun light and store at room temperature.

### HAZARD WARNINGS

Dust is created during milling of Ceramill TEMP Multilayer blanks that can cause irritation of the eyes, skin and airways. Therefore always ensure that the extractor on your milling machine is functioning perfectly. Further safety information can be obtained from the safety data sheet.

### WEARING DURATION

Up to 12 months.

### USER INFORMATION

#### Design:

\_ For crowns, do not fall below the minimum wall thickness of 1mm in occlusal and 0.6mm in cervical area.  
 \_ When designing bridges the maximum number of intermediate links is two.  
 \_ When designing bridges the connection cross-sections between crown and connected bridge units may not be less than 3.6mm anterior and 4.4mm posterior.

### FINISHING

\_ Cross-cut tungsten carbide cutters or separating discs, which are suitable for acrylic, should be used for trimming and separating frameworks from the blank.  
 \_ Excessive heat build-up should be avoided during finishing and polishing to prevent fitting inaccuracies after fabrication of the framework.

**Important information:** Avoid overheating during finishing and polishing!

### RESTORING

Frameworks, full crowns and bridges, which have been fabricated using Ceramill TEMP Multilayer blanks, can be restored and adjusted at any time using any conventional MMA-based cold-curing resin.

### OPTIONS FOR LUTING INTRAORALLY

\_ for temporary restorations: eugenol-free temporary cements

### SPECIAL INFORMATION

Enter the LOT No. with each procedure that requires identification of the material.

### SHELF-LIFE

10 years from the date of manufacture.

### DISPOSAL

Ceramill TEMP Multilayer is insoluble in water, inactive, does not pose a risk for the ground water and can therefore be disposed of with normal household waste.

### SAFETY DATA SHEETS / DECLARATION OF CONFORMITY

Safety data sheets and declaration of conformity are available on request and under [www.amanngirschbach.com](http://www.amanngirschbach.com)

Information as of 2015-01

**The product properties are based on complying with and observing these instructions for use.**