 **ceramill® zolid**  
preshades

**Gebrauchsanweisung**  
**Instruction Manual**  
**Instructions d'utili-**  
**sation**  
**Istruzioni per l'uso**  
**Instrucciones de uso**

Deutsch	3 - 19
English	20 - 35
Français	36 - 52
Italiano	53 - 69
Español	70 - 86

**For 3rd party systems**

**AMANNGIRRBACH**

- Original Gebrauchsanweisung -

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- Translation of the original Operating Instructions -

## **Table of Contents**

<b>Explanation of Symbols</b> .....	<b>21</b>
<b>General Safety Instructions</b> .....	<b>22</b>
<b>Suitable Personnel</b> .....	<b>22</b>
<b>Properties</b> .....	<b>23</b>
<b>Warranty/Exclusion of Liability</b> .....	<b>26</b>
<b>Testing the Blanks</b> .....	<b>26</b>
<b>Application</b> .....	<b>27</b>
<b>Fixation</b> .....	<b>34</b>
<b>Post-processing at the dentist's</b> .....	<b>34</b>
<b>Follow-up examination at the dentist's</b> .....	<b>35</b>
<b>Accessories</b> .....	<b>35</b>
<b>Download Infos</b> .....	<b>35</b>

## Explanation of Symbols

### Warning indications

Warning indications in the text are marked with a colour-backed triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- \_ **NOTE** means that property damage can occur.
- \_ **CAUTION** means that light to fairly serious personal injury can occur.
- \_ **WARNING** means that serious personal injury can occur.
- \_ **DANGER** means that life-threatening personal injury can occur.

### Important information

Important information that do not lead to hazards for humans or to property damage are marked with the icon aside and are boxed.

**Other symbols in the Manual**

Symbol	Meaning
▷	Item of an operation description
–	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

**General Safety Instructions****CAUTION:**

Possible health impairment from zirconium-oxide dust!

- ▷ When processing, wear personal protective equipment (dust protection mask, safety glasses/goggles, ...).

**Suitable Personnel****NOTE:**

The product may only be used by trained dental technicians.

## Properties

### Product description

Ceramill Zolid Preshades are pre-coloured blanks made of zirconium oxide (Y-TZP ZrO<sub>2</sub>) for type II, class 6 dental applications in accordance with DIN EN ISO 6872.

They are used for manufacturing permanent and removable prosthetic restorations (e.g., crowns and bridges, conical/ telescopic crowns, supra-constructions).

Ceramill Zolid Preshades are available in three shades and in three heights respectively. These blanks are not stained with the Ceramill Liquids, but monolithically stained upon successful final sintering with Ceramill Stain&Glaze or used as frame for veneering. The following Table lists the recommendations for achieving the respective tooth shade using Ceramill Zolid Preshades.

Blank Shade	Tooth Shade						
	A1	A2	A3	A3,5	B1	B2	B3
Ceramill Zolid Preshades Zolid 1	○	○	-	-	●	●	-
Ceramill Zolid Preshades Zolid 2	●	●	○	-	-	-	-
Ceramill Zolid Preshades Zolid 3	-	●	●	○	-	-	○

- ideal for this shade
- shade can be achieved
- not suitable for this shade

After completion of the specified final sintering, Ceramill Zolid Pre-shades meets the requirements of DIN EN ISO 6872; with appropriate further processing (see page 29), it also meets the requirements of ISO 13356.

**Safety data sheet/Declaration of conformity**

The data sheet on safety and the declaration of conformity are available on request under *www.amanngirrbach.com*.

**Technical Data**

	<b>Unit</b>	<b>Value</b>
Bending strength (4-point)	MPa	1200 ± 200
E-module	GPa	> 200
Grain size	µm	≤ 0.6
Density	g/cm <sup>3</sup>	≥ 6,05
Open porosity	%	0
Thermal expansion coefficient (CTE) (25 - 500°C)	1/K	11 ± 0,5 × 10 <sup>-6</sup>
Chemical solubility	µg/cm <sup>2</sup>	< 5
Radioactivity	Bq/g	< 0.2

### Chemical composition

Oxide	Mass percentage
ZrO <sub>2</sub> + HfO <sub>2</sub> + Y <sub>2</sub> O <sub>3</sub>	≥ 99.0
Y <sub>2</sub> O <sub>3</sub>	4.5 - 5.6
HfO <sub>2</sub>	< 5
Al <sub>2</sub> O <sub>3</sub>	< 0.5
Other oxides	< 1

### Shelf life of the blanks

When appropriately stored, Ceramill Zolid zirconium-oxide blanks Preshades can be used five years after the manufacturing date.

### Storage

Store Ceramill Zolid Preshades in the original packaging in a dry location.

### Packaging sizes

Art. No.	Designation	Description	Package
760261	Ceramill Zolid Preshades 1, 98x12 N	Zirconium-oxide blank, pre-coloured, 98 × 12 mm	Pack of 1
760263	Ceramill Zolid Preshades 1, 98x16 N	Zirconium-oxide blank, pre-coloured, 98 × 16 mm	Pack of 1
760265	Ceramill Zolid Preshades 1, 98x20 N	Zirconium-oxide blank, pre-coloured, 98 × 20 mm	Pack of 1
760267	Ceramill Zolid Preshades 2, 98x12 N	Zirconium-oxide blank, pre-coloured, 98 × 12 mm	Pack of 1

Art. No.	Designation	Description	Package
760269	Ceramill Zolid Preshades 2, 98x16 N	Zirconium-oxide blank, pre-coloured, 98 x 16 mm	Pack of 1
760271	Ceramill Zolid Preshades 2, 98x20 N	Zirconium-oxide blank, pre-coloured, 98 x 20 mm	Pack of 1
760273	Ceramill Zolid Preshades 3, 98x12 N	Zirconium-oxide blank, pre-coloured, 98 x 12 mm	Pack of 1
760275	Ceramill Zolid Preshades 3, 98x16 N	Zirconium-oxide blank, pre-coloured, 98 x 16 mm	Pack of 1
760277	Ceramill Zolid Preshades 3, 98x20 N	Zirconium-oxide blank, pre-coloured, 98 x 20 mm	Pack of 1

## Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

## Testing the Blanks

Upon receipt, it is imperative to visually check the proper condition of the Ceramill Zolid Preshades blanks. After using a (transport-)damaged blank, complaint claims are no longer possible.

## Application

### Indication ranges

- \_ Anatomically reduced crown and bridge frames in the anterior and posterior tooth range, and monolithic (fully anatomical) crowns and bridges.
- \_ Anatomically reduced bridge frames with a maximum of three connected intermediate units in the anterior and a maximum of two connected intermediate links in the posterior region, and a maximum anatomical length of 50 mm.
- \_ Monolithic bridges with a maximum of two connected intermediate units in the posterior region, and a maximum anatomical length of 50 mm.
- \_ Cantilever frames and bridges with a maximum of one bridge pontic (maximum one free-end pontic and no further than the second premolar).

Indication limitation for Canada:

- \_ Single crowns
- \_ Front-tooth bridges
- \_ Posterior bridges with up to four units

### Contraindications

- \_ Insufficient tooth-structure availability
- \_ Insufficient preparation results
- \_ Insufficient oral hygiene
- \_ More than two connected bridge units in the posterior region
- \_ Known incompatibilities with respect to the components
- \_ Heavily discoloured hard tooth structure

### Material-specific frame parameters

The following material-specific frame parameters must be complied with when fabricating Ceramill Zolid Preshades zirconium oxide frames in high-density sintered condition:

	Minimum frame thickness in mm	Connector cross-section in mm <sup>2</sup>		Maximum number of connected bridge units	
		Anterior	Posterior	Anterior	Posterior
Ceramill Zolid Preshades	0.5	≥ 7	≥ 9	3	2

The enlargement or shrinkage factor determined by Amann Girrbach for the Ceramill Zolid Preshades blanks is provided on the blanks with a batch-specific coding.

## Detailed minimum wall thicknesses and connector cross-sections

Indication	Pat- tern	Total number of units	Number of connected bridge units	Wall thickness in mm		Connector cross- section in mm <sup>2</sup>
				incisal/ occlusal	circular	
Primary compo- nents/dou- ble crowns	–	1	–	0.7	0.5	–
Single crown	–	1	–	0.5	0.5	–
Front-tooth bridge	OXO	3	1	0.5	0.5	>7
	OXXO	4	2	0.7	0.5	>9
	OXXXO	5	3			
Posterior bridge	OXO	3	1	0.7	0.5	>9
	OXXO	4	2	1.0	0.7	>12
Cantilever bridge	OOX	3	Free-end pontic	1.0	0.7	>12
	OOOX	4				
	OXOX	4	1 + free-end pontic	1.0	0.7	>12

**O** Abutment crown

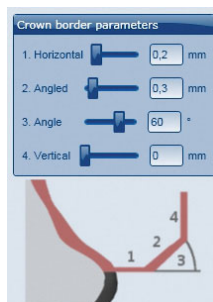
**X** Bridge unit

### Border parameters in the software

For Ceramill Zolid Preshades, it should be observed that the crown border thicknesses are sufficient. Recommended values:

- \_ 1. Crown border thickness (Horizontal): 0.2 mm
- \_ 2. Off-set (Angled): 0.3 mm
- \_ 3. Angle: 60°
- \_ 4. Height (Vertical): 0 mm

### Removal from the blank

Utmost care must be taken when removing the frames from the blank:

- ▷ Separate the frames by grinding them out using a turbine grinder (without water-cooling) and a pointed diamond bit (do not use cutting discs).

For trimming and dressing the connectors and smoothing out the frames, we recommend using the Amann Girrbach polishing set.

### Final sintering

The Ceramill Therm high-temperature furnace (178300/178350) or another high-quality and suitable furnace is recommended for the sintering (please also refer to the operating instructions of the respective sintering furnace).

For sintering, a separate sintering bowl (178370) and sintering pearls (178311) are used.

**NOTE:**

Loss of translucence!

- ▷ The sintering bowl with the pearls are to be used only for Ceramill Zolid and **not** for Ceramill ZI applications!
- ▷ Do not sinter Ceramill Zolid Preshades and Ceramill ZI frames together in one sintering bowl!

Ceramill Zolid Preshades frames and ZI frames may be sintered together in one sintering process in separate, stacked sintering bowls. Sintering bowls can only be stacked in the Ceramill Therm II (178350).

- ▷ Place the Ceramill Zolid Preshades frames for sintering into the sintering bowl (178370) filled with sintering pearls (178311).
- ▷ To ensure proper support, place the frames with slight pressure onto the sintering pearls.

**NOTE:**

- ▷ Pay attention that no pearls are jammed in the interdental spaces or located in the crown cavities!

- ▷ Perform the final sintering of the frames according to the following program:
  - Heat-up phase: Room temperature to final temperature 1450 °C; heating rate 5 - 10 K/min

- Dwell time at final temperature: 2 hrs
- Cooling phase: Final temperature 1450 °C to room temperature (at least < 200 °C); approx. 5 K/min (approx. 5 hours)

## Frames for Veneering / Ceramill Zolid Preshades

### Post-processing

If required, the zirconium oxide frames can be reworked after the final sintering without pressure, using a water-cooled lab turbine and suitable diamond grinding points (recommended grain approx. 40 µm).

### Frame preparation for veneering

After checking the fitting accuracy, contact points and occlusion:

- ▷ Clean the interior crown surfaces by means of corundum blasting (aluminium oxide, 110 µm, pressure ≤ 2 bar) and subsequent steaming off.

### Veneer ceramics

Ceramill Zolid Preshades is suitable for veneering with conventional zirconium-oxide veneer ceramics (e.g., Creation ZI-F). For information on the thermal-expansion coefficient of Ceramill Zolid Preshades blanks, please refer to the Technical data (see page 24).

For bridges with more than five units and bridges with massive intermediate units, step-by-step long-term cooling to 500 °C is recommended.

## Monolithic Dentures / Ceramill Zolid Preshades

### Post-processing

After the final sintering has taken place:

- ▷ If required, the zirconium oxide frames can be reworked after the final sintering without pressure, using a water-cooled lab turbine and suitable diamond grinding points (recommended grain approx. 40 µm).
- ▷ Polish the contact surfaces to the antagonist and to the neighbouring teeth to a high gloss when in high-density sintered condition. We recommend using the Amann Girrbach polishing set for this (please also refer to the processing instructions of the polishing kit).

### Frame preparation for staining and glazing

After checking the fitting accuracy, contact points and occlusion:

- ▷ Clean the interior crown surfaces by means of corundum blasting (aluminium oxide, 110 µm, pressure ≤ 2 bar) and subsequent ultrasonic treatment and/or steaming off.

The exterior crown surfaces are not corundum blasted.

For bridges with more than five units and bridges with massive intermediate units, step-by-step long-term cooling to 500 °C is recommended.

### Staining and glazing

- ▷ Stain and glaze the units afterwards. We recommend using the Amann Girrbach stain and glaze kit for this (please also refer to the processing instructions of the stain and glaze kit).

## Fixation

Conventional cementation:

Due to the high strength and stability of Ceramill Zolid Preshades frames, conventional fixation with zinc-oxide phosphate cement or glass ionomer cement (e.g., GC Fuji Plus) is possible in most cases.

► When applying conventional cementation, observe sufficient retention and a minimum abutment height of 3 mm!

Adhesive fixation:

For adhesive fixation, we recommend the bonding composite Panavia<sup>®</sup> 21 or Panavia<sup>®</sup> F 2.0, owing to the excellent bond strength with the zirconium-oxide frame material.

Provisional cementation is not recommended, as the frames can be subject to damage upon removal.

## Post-processing at the dentist's

When grinding is required by the dentist while installing the work into the patient's mouth, the units must be well re-polished to a high gloss. For grinding, we recommend using only diamond grinding points (recommended grain approx. 40 µm). For polishing, we recommend using the Amann Girrbach diamond polishing bits (please also refer to the processing instructions of the polishing kit).

### NOTE:

When the work is not sufficiently polished, the antagonist may become damaged due to abrasion!

## Follow-up examination at the dentist's

It is urgently recommended to check the monolithic dentures in the patient's mouth once a year. In this, the remaining dentition, the antagonists and the soft tissue are also to be inspected. Corrective measures are to be undertaken as required. Here, it is also important that the objects are polished afterwards to a high gloss.

## Accessories

### Art. No. Description

760349	Ceramill Stain & Glaze Kit update
178370	Ceramill Zolid sintering bowl, stackable
178311	Ceramill Therm sintering pearls, 200 g
760606	Ceramill Roto Motion 0.6
875510	Ceramill Polish - Dent Kit
875500	Ceramill Polish - Lab Kit

Plastic holding plates are available for swift fixation of the plastic modelling and secure positioning in the copy milling machine.

## Download Infos

Further instructions/manuals can be downloaded under [www.ceramill-m-center.com](http://www.ceramill-m-center.com) and/or [www.amanngirrbach.com](http://www.amanngirrbach.com).

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**Amann Girrbach AG**
**Amann Girrbach GmbH**