ceramill® a-cast

DE Gebrauchsanweisung	3 - 10
EN Instruction Manual	11 - 18
FR Instructions d'utilisation	19 - 26
IT Istruzioni per l'uso	27 - 34
ES Instrucciones de uso	35 - 42

- Original Gebrauchsanweisung -

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DE

- Translation of the original Operating Instructions -

Table of Contents

Explanation of Symbols	12
General Safety Instructions	14
Suitable Personnel	14
Properties	15
Warranty/Exclusion of Liability	16
Testing the Blanks	16
Mounting to the Worktable	16
Application	16
Environmental Protection	17
Accessories	18
Download information	18

EN

Explanation of Symbols

Warning indications

Warning indications in the text are marked with a triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- _ **NOTE** means that property damage can occur.
- _ **CAUTION** means that light to fairly serious personal injury can occur.
- _ **WARNING** means that serious personal injury can occur.
- _ **DANGER** means that life-threatening personal injury can occur.

Important information

Important information that do not lead to hazards for humans or to property damage are marked with the icon aside and are boxed.

Other symbols in the Manual

Symbol	Meaning
▷	Item of an operation description
–	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

Other symbols on the product

Symbol	Meaning
REF	Article number
LOT	Batch code
	Manufacturer
	Observe instructions for use
	Use by YYYY-MM-DD or YYYY-MM
SN	Serial number

General Safety Instructions

**CAUTION:**

Health impairment due to dust from PMMA!

When processing constructions made of Ceramill A-Cast, dusts can develop that may lead to mechanical irritation of the eyes and respiratory tract.

- ▷ Always make sure that the extraction system for the milling machine and the workplace for individual reworking operates properly.
- ▷ When processing, wear personal protective equipment (dust protection mask, safety glasses/goggles, ...).
- ▷ Further safety-relevant information can be found in the safety data sheet.

**NOTE:**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Suitable Personnel

**NOTE:**

The product may only be processed by trained dental technicians/dentists.

Properties

Intended use

Ceramill A-Cast are PMMA blanks for manufacturing CAD/CAM-fabricated crown and bridge frames in casting technology.

Product Description

Ceramill A-Cast are milling blanks for the production of crown and bridge constructions using CAD/CAM technology. The constructions are used for the production of lost moulds in casting technology. Ceramill A-Cast is not a medical product.

Safety data sheet/Declaration of conformity

The safety data sheet is available for download on the Amann Girrbach website under *Services > Downloads > Additional documents*. The product's declaration of conformity is available upon request from the manufacturer.

Chemical composition

_ Polymethylmethacrylate (PMMA)

Shelf life of the blanks

When appropriately stored, the blanks can be used 4 years after the manufacturing date.

Storage

- _ Store only in original packaging.
- _ Store separated from foodstuffs.
- _ Keep container tightly sealed.

Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

Testing the Blanks

Upon receipt, it is imperative to visually check the proper condition of the blanks. After using a (transport-) damaged blank, complaint claims are no longer possible.

Mounting to the Worktable

The mounting of the blanks is described in the Operating Instructions of the respective milling machine.

Application

Indications

- _ Production of lost moulds for casting technology.

Contraindications

- _ All indications that are not listed under "Indication".

Cutting Out and Processing

- ▷ Use cross-toothed tungsten carbide cutters or cutting discs suitable for plastic, for cutting out and processing constructions from the blank.
- ▷ To avoid fitting inaccuracies after the construction has been manufactured: Prevent intense heat generation during processing and polishing.

Environmental Protection

Packaging

In terms of packaging, the manufacturer participates in country-specific recycling systems, which ensure optimal recycling.

All packaging materials used are environmentally-friendly and recyclable.

Disposal of the blanks

Dispose of the contents/container in accordance with local/regional/international regulations.

Always dispose of according to official regulations.

Accessories

For more information on product-specific accessories, please refer to www.amangirrbach.com.

Download information

Further instructions/manuals can be downloaded under www.amangirrbach.com/instruction-manuals.

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Amann Girrbach AG
Amann Girrbach GmbH