- DE** Gebrauchsanweisung
- EN** Instructions for use
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- ES** Instrucciones de empleo

Manufacturer | Hersteller
Distribution | Vertrieb

Amann Gırrbach AG
Herrschaftswiesen 1
6842 Koblach | Austria
Fon +43 5523 62333-105
Fax +43 5523 62333-5119
austria@amanngırrbach.com

Amann Gırrbach GmbH
Dürrenweg 40
75177 Pforzheim | Germany
Fon +49 7231 957-100
Fax +49 7231 957-159
germany@amanngırrbach.com
www.amanngırrbach.com

DIN EN ISO 13485
ISO 9001

32995-FB 2016-08-30

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INSTRUCTIONS FOR USE CERAMILL D-WAX

Translation of the original instructions for use

PURPOSE

Milling wax for full denture bases for processing in the Ceramill Motion 2 (5x) CNC milling machine.

PRODUCT DESCRIPTION

Ceramill D WAX is a pink-coloured wax blank Class 1 medical product. It is used for the fabrication and try-in of full-denture bases and is processed using water cooling on both sides with the Ceramill Motion 2 (5x) CNC milling machine. The wax can be adjusted during and after intraoral try-in.

TECHNICAL DATA

Color:	pink	
Solidification point:	84 °C	ISO 2207
Viscosity at 120 °C:	< 50 mPas	DIN 53019
Solubility in water:	insoluble	
Flash point:	> 140 °C	DIN ISO 2592
Density at 20 °C:	ca. 0.93 g/cm ³	DIN 53217
Odour:	mild typical	

CHEMICAL COMPOSITION

Hydrocarbon waxes, natural waxes, dyes.

PACKAGE SIZES

760570	Ceramill D-WAX	ht = 30 mm	Dental arch shape
--------	----------------	------------	-------------------

STORAGE

Protect wax blanks against direct sunlight. Store in the original packaging and in a cool, well-ventilated location.

WARNING AND SAFETY INSTRUCTIONS

Ensure extraction of the fumes and adequate ventilation when processing thermally. Risk of skin burns due to hot molten wax. Use personal safety clothing (protective dust mask, safety glasses, protective gloves, protective coat) when processing.

INDICATIONS

- _ For milling of a denture base, which can be fitted in the patient's mouth for try-in.
- _ For processing in the flasking procedure for cold and heat-curing acrylics.

SHELF-LIFE

3 years from the date of manufacture.

CLEANING AFTER MILLING

After milling, the denture base should be cleaned of spots of coolant liquid using a cloth, which has been moistened with cold water. A toothbrush with soft bristles can be used for narrower areas that cannot be accessed with the cloth.

DISPOSAL

The material can generally be disposed of with domestic waste. It can be put into, e.g. a suitable incinerator taking into account the local authority regulations.

GUARANTEE

Application recommendations, no matter whether verbally, in writing, or by means of practical instructions are based on our experience and tests and are thus only a recommendation. Our products are subject to continuous further development. We reserve the right to alter the products in terms of design or composition.

SAFETY DATA SHEETS AND DECLARATION OF CONFORMITY

Safety data sheets and declaration of conformity are available on request and under www.amanngırrbach.com

	Batch code
	Catalog number
	Manufacturer
	Follow the instructions for use
Rx only	That may be used in place of statement "CAUTION". US Federal law restricts this device to sale by or on the order of a dentist.
	Used by YYYY-MM-DD
	Serial number

FR	
----	--

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