# **ceramill**<sup>®</sup> liquid cl

<b>DE</b> Gebrauchsanweisung	3 - 16
<b>EN</b> Instruction Manual	17 - 30
<b>FR</b> Instructions d'utilisation	31 - 44
<b>IT</b> Istruzioni per l'uso	45 - 58
<b>ES</b> Instrucciones de uso	59 - 72

- Original Gebrauchsanweisung -

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- Translation of the original Operating Instructions -

## Table of Contents

Explanation of Symbols .....	18
General Safety Instructions .....	20
Suitable Personnel .....	21
Properties .....	22
Warranty/Exclusion of Liability .....	23
Application .....	24
Pre-drying and Sintering .....	27
Environmental Protection .....	30
Accessories .....	30
Download Infos .....	30

EN

## Explanation of Symbols

### Warning indications

Warning indications in the text are marked with a triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- \_ **NOTE** means that property damage can occur.
- \_ **CAUTION** means that light to fairly serious personal injury can occur.
- \_ **WARNING** means that serious personal injury can occur.
- \_ **DANGER** means that life-threatening personal injury can occur.

### Important information

Important information that do not lead to hazards for humans or to property damage are marked with the icon aside and are boxed.

**Other symbols in the Manual**

Symbol	Meaning
▷	Item of an operation description
–	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

**Other symbols on the product**

Symbol	Meaning
	Article number
	Batch code
	Manufacturer
	Observe instructions for use
	Expiration date
	Caustic
<b>Rx only</b>	According to U.S. Federal Law, the product may only be sold through or on behalf of a dentist.

## General Safety Instructions

### CAUTION:

▷ If medical advice is needed, have product container or label at hand.

Keep out of reach of children.

▷ Read label before use.

▷ Do not breathe dust/fume/gas/mist/vapours/spray.

▷ Wear protective gloves/protective clothing/eye protection/face protection.

▷ IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing.

▷ Immediately call a POISON CENTER/doctor.

▷ Store locked up.

Dispose of the contents/container in accordance with local/regional/ international regulations.

For further information, such as e.g. the safety data sheets on Ceramill Liquid, please refer to [www.amanngirrbach.com](http://www.amanngirrbach.com) in the download area.

**NOTE:**

Damage to the environment!

The product contains heavy metals. Prevent emission into the environment. Specific pre-treatment is required.

Water-pollution class 1 (self-classification): slightly dangerous to water.

▷ Do not allow undiluted product or large quantities of it to be introduced into groundwater, bodies of water or the sewage system.

**NOTE:**

The Ceramill zirconium oxide blanks Preshade/Multilayer are not released for application with Ceramill Liquid.

- ▷ Use Ceramill Liquid only with white Ceramill zirconium oxide.
- ▷ Do not mix Ceramill Liquid with other Ceramill Liquid systems. The staining solutions may only be combined unmixed within a Ceramill Liquid system.

## Suitable Personnel

**NOTE:**

The product may only be used by trained dental technicians.

## Properties

### Intended use

Dye solutions for staining zirconium oxide blanks.

### Product description

Ceramill Liquids CL are water-based liquids with sugar for complete or partial staining of milled and pre-sintered frames of Ceramill ZI.

Ceramill Liquids CL are available in 4 base shades and 2 effect shades:

- \_ Base shades CL 1-4
- \_ Effect shade CL OR: Orange for the fissure area
- \_ Effect shade CL GR: Grey for the cusp area

### Safety data sheet/Declaration of conformity

The data sheet on safety and the declaration of conformity are available on request under [www.amanngirrbach.com](http://www.amanngirrbach.com).

**Shelf life**

Ceramill Liquids have a shelf life of 2 years.

The expiry date and batch number are indicated on the original packaging.

**Storage**

- ▷ Store in original container only.
- ▷ Store separated from foodstuffs.
- ▷ Keep container tightly sealed.

Within their shelf life, the staining solutions can change their natural colour during storage or usage. The colouration of the sintered restoration is not affected through this.

**Warranty/Exclusion of Liability**

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

## Application

### NOTE:

Before staining:

- ▷ Clean the frames from residual zirconia dust using a brush and compressed air.

### NOTE:

Ensure there are no metal traces on the objects.

- ▷ Use only plastic tweezers for immersing.

For recommendations on mixing the Ceramill Liquids, please refer to the ZI shade-mixing chart 1 on page 11. To achieve a less intense shading:

- ▷ Increase the amount of distilled water.

- ▷ Mix the base shade.
- ▷ Process the milled frames of Ceramill ZI as follows:
  - Immerse into the base shade for 2 minutes
  - Wipe dry with a paper tissue
  - Pre-dry in an enclosed ceramic furnace for **30** minutes at 80 °C (see page 15),
  - Sinter in the Ceramill Therm.

## Colour-mixing chart for Ceramill ZI

The information on shade allocation and mixing the dye solutions are recommendations by Amann Girrbach. Everyone has a different colour perception. Therefore, it is advisable to produce shade samples prior to staining the actual object.

Tooth shade	Base shade			Selection in the Ceramill Mind software
	Liquid	% Liquid	% Dist. Water	
<b>Bleach shades</b>	CL1	50	50	CL1 50
<b>A1</b>	CL1	100	0	CL1 100
<b>A2</b>	CL2	50	50	CL2 50
<b>A3</b>	CL2	100	0	CL2 100
<b>A3,5</b>	CL2	100	0	CL2 100
	CL3	100	0	CL3 100
<b>A4</b>	CL2	100	0	CL2 100
	CL4	100	0	CL4 100
<b>B1</b>	CL1	100	0	CL1 100
<b>B2</b>	CL2	50	50	CL2 50
	CL3	50	50	CL3 50
<b>B3</b>	CL2	100	0	CL2 100
	CL3	100	0	CL3 100
<b>B4</b>	CL2	100	0	CL2 100
	CL3	100	0	CL3 100

Tab. 1 Shade-mixing chart for Ceramill ZI

Tooth shade	Base shade			Selection in the Ceramill Mind software
	Liquid	% Liquid	% Dist. Water	
<b>C1</b>	CL3	50	50	CL3 50
<b>C2</b>	CL3	50	50	CL3 50
<b>C3</b>	CL3	100	0	CL3 100
	CL4	50	50	CL4 50
<b>C4</b>	CL4	100	0	CL4 100
<b>D2</b>	CL3	50	50	CL3 50
	CL4	75	25	CL4 75
<b>D3</b>	CL3	100	0	CL3 100
	CL4	100	0	CL4 100
<b>D4</b>	CL4	50	50	CL4 50

Tab. 1 Shade-mixing chart for Ceramill ZI

For further information and details on the immersing technique, such as shade-assignment tables, mixing information or immersing durations, please refer to the illustrated instructions on staining techniques under [www.amanngirrbach.com](http://www.amanngirrbach.com).

## Pre-drying and Sintering

### Pre-drying

After staining with Ceramill Liquid, the Ceramill zirconium oxide should be pre-dried. This is to avoid the formation of spots (homogenisation). For large jobs, the risk of cracks and fissures is reduced.

Material	Pre-drying temperature	Pre-drying duration
Ceramill zirconium oxide frames (without auxiliary sintering block)	80 °C	60 min

### Suitable equipment

After staining the white Ceramill zirconium oxide, the work can be dried using a ceramic furnace. The ceramic furnace and the further ceramic firing cycles are not influenced through the drying procedure. However, it is recommended to occasionally burn out the ceramic furnace at 1150 °C for 5 minutes. As an alternative, a drying cabinet or mini oven can also be used.

Suitable equipment for pre-drying:

- \_ Domestic ovens/mini ovens ✓✓✓
- \_ Drying cabinet ✓✓✓
- \_ Ceramic furnace ✓✓
  - In case the ceramic furnace cannot be programmed to 80 °C, enter the lowest possible temperature and leave furnace door open.
  - Use a thermometer to check, whether the 80 °C are reached on the object.
- \_ Ceramill Therm (from version 2008 on) ✓
- \_ Curing lamps ✓
  - Curing lamps generally do not achieve the recommended drying temperature. Therefore, when using curing lamps, the drying duration must be accordingly prolonged.

## Sintering

### Basic rule for sintering multi-unit restorations

All stained restorations **without** auxiliary sintering block (approx. to 9 units) can principally be sintered in standard program 1. After staining, all work **with** auxiliary sintering block require the slower sintering program 4 indicated below. In the Ceramill Therm 2, this must be programmed on program position 4. Smaller jobs can be integrated in the sintering process in this program.

### Adjustments on the sintering furnace

▷ Adjust the following parameters on the sintering furnace:

Material	Sintering furnace	Sintering program	Recommended pre-drying
Ceramill zirconium oxide without sintering block	Therm 1	Program 1	1 h at 80 °C (external, e.g. ceramic furnace)
Ceramill zirconium oxide with sintering block	From Therm 2 on	Program 1	1 h at 80 °C (external, e.g. ceramic furnace or programmable on program position 4)
Ceramill zirconium oxide with sintering block	Therm 1	not possible	-
Ceramill zirconium oxide with sintering block	From Therm 2 on	Program 4 <sup>1)</sup>	Pre-drying already included in the program's slow heating phase

1) 3 °C/min heating rate, 2 h dwell time at 1450 °C, long-term cooling as in program 3

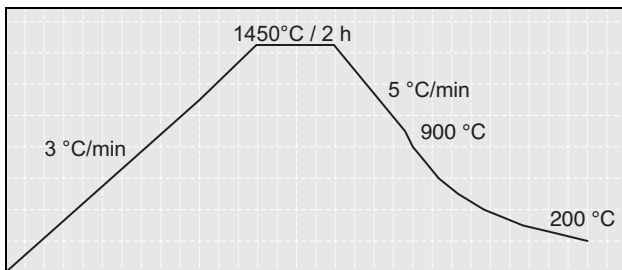
Fig. 1 Sintering program Ceramill ZI in the Ceramill Therm 2 for large bridges with auxiliary sintering block

▷ Sinter in a sintering bowl in the Ceramill Therm.

## Environmental Protection

### Packaging

All packaging materials used are environmentally-friendly and recyclable.

### Disposal

- \_ Ceramill Liquid may not be disposed of together with household waste.
- \_ Do not dispose of into sewer systems.
- \_ Dispose of the contents/container in accordance with local/ regional/ international regulations.

## Accessories

For more information on product-specific accessories, please refer to [www.amanngirrbach.com](http://www.amanngirrbach.com).

## Download Infos

Further instructions/manuals, safety data sheets and video tutorials can be downloaded under [www.amanngirrbach.com](http://www.amanngirrbach.com).

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**Amann Girrbach AG**
**Amann Girrbach GmbH**