- DE** Gebrauchsanweisung
- EN** Instructions for use
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- ES** Instrucciones de empleo

Manufacturer | Hersteller  
Distribution | Vertrieb

**Amann Girrnbach AG**  
Herrschaftswiesen 1  
6842 Koblach | Austria  
Fon +43 5523 62333-105  
Fax +43 5523 62333-5119  
austria@amanngirrnbach.com

Distribution | Vertrieb D/A

**Amann Girrnbach GmbH**  
Dürrenweg 40  
75177 Pforzheim | Germany  
Fon +49 7231 957-100  
Fax +49 7231 957-159  
germany@amanngirrnbach.com  
[www.amanngirrnbach.com](http://www.amanngirrnbach.com)

ISO 9001

DE

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Original-Gebrauchsanweisung

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EN

## INSTRUCTIONS FOR USE

Translation of the original instructions for use

### PURPOSE

Resin for producing dental models with a milling machine.

### PRODUCT DESCRIPTION

Ceramill M-Plast is an ivory-coloured resin blank, intended for producing dental models, for processing dry with a milling machine.

### TECHNICAL DATA

|                         |                                |
|-------------------------|--------------------------------|
| Colour:                 | ivory                          |
| Temperature-resistance: | Approx. 59 °C                  |
| Shore D hardness:       | Approx. 80                     |
| Compressive strength:   | Approx. 46 MPa                 |
| Flexural strength:      | Approx. 61 MPa                 |
| Density:                | Approx. 1,00 g/cm <sup>3</sup> |
| Odour:                  | odourless                      |

### CHEMICAL COMPOSITION (PERCENTAGE BY WEIGHT)

|       |              |
|-------|--------------|
| 100 % | polyurethane |
|-------|--------------|

### TRANSPORT AND STORAGE

Temperature during transport should not exceed 59 °C / 138 °F.  
Store flat and dry at a temperature between 18 °C / 64 °F and 25 °C / 77 °F.

### WARNING AND SAFETY INSTRUCTIONS

When processing, please use personal protective clothing (dust mask, protective goggles, ...).

### INDICATIONS

\_ For processing in the fabrication of dental models.  
\_ No clinical dental product, not suitable for use in the oral cavity.

### DISPOSAL

In accordance with official regulations waste can be incinerated in a suitable, authorised plant or disposed of at an authorised waste disposal site.

### GUARANTEE

Application recommendations, no matter whether verbally, in writing, or by means of practical instructions are based on our experience and tests and are thus only a recommendation. Our products are subject to continuous further development. We reserve the right to alter the products in terms of design or composition.

### SAFETY DATA SHEETS AND DECLARATION OF CONFORMITY

Safety data sheets and declaration of conformity are available on request and under [www.amanngirrnbach.com](http://www.amanngirrnbach.com)

|            |                                 |
|------------|---------------------------------|
| <b>LOT</b> | Batch code                      |
| <b>REF</b> | Catalog number                  |
|            | Manufacturer                    |
| <b>i</b>   | Follow the instructions for use |
|            | Used by YYYY-MM-DD              |
| <b>SN</b>  | Serial number                   |

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