- DE** Gebrauchsanweisung
- EN** Instructions for use
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- ES** Instrucciones de empleo

 Manufacturer | Hersteller  
Distribution | Vertrieb

**Amann Girschbach AG**  
Herrschaftswiesen 1  
6842 Koblach | Austria  
Fon +43 5523 62333-105  
Fax +43 5523 62333-5119  
austria@amanngirschbach.com  
[www.amanngirschbach.com](http://www.amanngirschbach.com)

**DIN EN ISO 13485**  
ISO 9001

31661FB 2016-08-30

**DE**

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**EN**

## INSTRUCTIONS FOR USE Ceramill PMMA

Translation of the original instructions for use

### PURPOSE

Transparent, fully combustible acrylic for processing in a milling machine.

### PRODUCT DESCRIPTION

Transparent acrylic which burns out without leaving a residue, for processing in a milling machine.

### TECHNICAL DATA

Colour	colourless or coloured
Odour	odourless
Softening temperature	approx. 102 °C
Ignition point	> 250 °C
Density	1.19 g/cm <sup>3</sup> at 20 °C
Water solubility	insoluble

### CHEMICAL COMPOSITION (PERCENTAGE BY WEIGHT)

100 % PMMA (Polymethylmethacrylat)

### STORAGE

No special measures required.

### WARNING AND SAFETY INSTRUCTIONS

When processing, please use personal protective clothing (dust mask, protective goggles, ...).

### INDICATIONS

- \_ Check of the fit and function of the laboratory restoration on the stone model and intraorally.
- \_ Not suitable for longer-term intraoral wear (max. 5 minutes).
- \_ Can be cast, pressed or over-pressed.

### CONTRAINDICATION

- \_ PMMA intolerance

### FRAMEWORK DESIGN AND FURTHER PROCESSING

Follow the manufacturer's instructions for materials in which the substructure is cast or pressed. To prevent the investment material from cracking, it is advisable to cover the framework with a thin layer of wax before investing. The PMMA framework must be cleaned and disinfected according to hygiene guidelines before placement in the mouth.

### SHELF LIFE

Shelf life of up to 5 years from the date of manufacture

### DISPOSAL

Contact the responsible certified waste disposal contractor.

### GUARANTEE

Application recommendations, no matter whether verbally, in writing, or by means of practical instructions are based on our experience and tests and are thus only a recommendation. Our products are subject to continuous further development. We reserve the right to alter the products in terms of design or composition.

### SAFETY DATA SHEETS AND DECLARATION OF CONFORMITY

Safety data sheets and declaration of conformity are available on request and under [www.amanngirschbach.com](http://www.amanngirschbach.com)

	Batch code
	Catalog number
	Manufacturer
	Follow the instructions for use
<b>Rx only</b>	That may be used in place of statement "CAUTION". US Federal law restricts this device to sale by or on the order of a dentist.
	Used by YYYY-MM-DD
	Serial number

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