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- Translation of the original Operating Instructions -

Contents

Explanation of Symbols	16
General Safety Instructions	17
Suitable Personnel	17
Properties	18
Warranty/Exclusion of Liability	21
Download Infos	21
Application	22
Cleaning, Disinfection and Sterilization	26

Explanation of Symbols

Warning indications

Warning indications in the text are marked with a triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- _ **NOTE** means that property damage can occur.
- _ **CAUTION** means that light to fairly serious personal injury can occur.
- _ **WARNING** means that serious personal injury can occur.
- _ **DANGER** means that life-threatening personal injury can occur.

Important information

Important information that do not lead to hazards for humans or property damage are marked with the icon aside and are also boxed.

Other symbols in the Manual

Symbol	Meaning
▷	Item of an operation description
–	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

General Safety Instructions**CAUTION:**

- ▷ When processing, wear personal protective equipment (gloves, safety glasses/goggles, ...).

Suitable Personnel**NOTE:****Lab kit:**

The product may only be used by trained dental technicians.

Dent Kit:

The product may only be used by dentists.

Properties

Product description

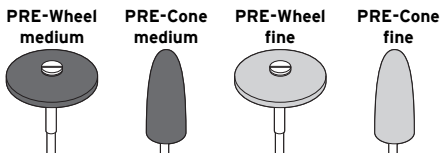
The Ceramill polish lab kit contains PRE and POST polishers:

- _ PRE-polishers are soft silicone polishers (without color pigments) for processing zirconium oxide in the pre-sintered state.
- _ POST polishers are diamond polishers for processing oxide ceramics in high-density sintered state.

The Ceramill Polish-Dent Kit contains diamond polishers for processing oxide ceramics in high-density sintered state within the patient's mouth.

Technical data

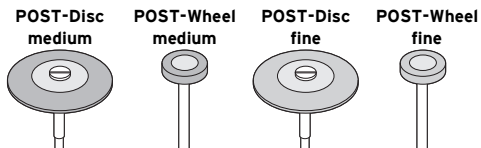
Lab kit

	PRE-Wheel medium	PRE-Cone medium	PRE-Wheel fine	PRE-Cone fine
Grain	Medium	Medium	Fine	Fine
Optimal speed	4000 - 7000 min ⁻¹			
Maximum speed	20000 min ⁻¹			
Shank diameter	2.35 mm			
ISO No.	658 900 372 523 220	658 104 275 523 100	658 900 372 513 220	658 104 275 513 100
Application	Thinning out the edges and smoothening the connectors in pre-sintered state			

Tab. 1 Technical data of the lab kit, PRE-polisher

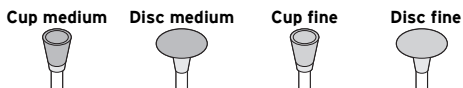
Grain	Medium	Medium	Fine	Fine
Optimal speed	7000 - 12000 min ⁻¹			
Maximum speed	20000 min ⁻¹			
Shank diameter	2.35 mm			
ISO No.	803 104 303 524 260	803 104 372 524 110	803 104 303 514 260	803 104 372 514 110
Application	Smoothing, pre-polishing in high-density sintered state		High-gloss polishing in high-density sintered state	

Tab. 2 Technical data of the lab kit, POST polisher

Dent Kit

Grain	Medium	Medium	Fine	Fine
Optimal speed	7000 - 10000 min ⁻¹			
Maximum speed	20000 min ⁻¹			
Shank diameter	2.35 mm			
ISO No.	803 204 030 524 060	803 204 304 524 100	803 204 030 524 060	803 204 304 524 100
Application	Smoothing, pre-polishing in high-density sintered state		High-gloss polishing in high-density sintered state	
Medical product	Class IIa			

Tab. 3 Technical Data, Dent Kit

Safety data sheet/Declaration of conformity

The data sheet on safety and the declaration of conformity are available on request under www.amanngirrbach.com.

Storage

Store in original packaging at room temperature, protected against dust, moisture, pressure and recontamination. The original packaging allows for clear product identification via the label.

Packaging sizes

Art. No.	Designation	Description
875500	Ceramill Polish - Lab Kit, includes one polisher each: _ PRE-...: (prior to the sintering) • PRE-Wheel medium • PRE-Cone medium • PRE-Wheel fine • PRE-Cone fine _ POST-...: (after the sintering) • POST-Disc medium • POST-Wheel medium • POST-Disc fine • POST-Wheel fine	Zirco nium oxide polisher set for dental technicians • Wheel medium* • Cone medium • Wheel fine* • Cone Fine • Disc medium • Wheel medium • Disc Fine • Wheel fine
875501	PRE-Wheel medium (10 pce.)	Wheel medium**
875502	PRE-Cone medium (10 pce.)	Cone medium
875503	PRE-Wheel fine (10 pce.)	Wheel fine**
875504	PRE-Cone fine (10 pce.)	Cone Fine

Art. No.	Designation	Description
875505	POST-Disc medium (2 pce.)	Disc medium
875506	POST-Wheel medium (4 pce.)	Wheel medium
875507	POST-Disc fine (2 pce.)	Disc Fine
875508	POST-Wheel fine (4 pce.)	Wheel fine
875509	Ceramill Polish - Dent Kit, includes one polisher each:	Zirconium oxide polisher set for dentists
	_ Cup medium	_ Cup medium
	_ Disc medium	_ Disc medium
	_ Cup fine	_ Cup Fine
	_ Disc fine	_ Disc Fine
875510	Cup medium (4 pce.)	Cup medium
875511	Disc medium (4 pce.)	Disc medium
875512	Cup fine (4 pce.)	Cup Fine
875513	Disc fine (4 pce.)	Disc Fine

* with reusable mandrel

** without mandrel

Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

Download Infos

Further instructions/manuals and video tutorials can be downloaded under www.amanngirrbach.com.

Application

PRE-polishers

NOTE:

Reduction of the translucency of Ceramill ZOLID on contamination with Ceramill ZI.

- ▷ Clean polishers with a whetstone between the processing of Ceramill ZI and Ceramill ZOLID.

The polishers are used for dressing the holding ridges and for thinning out the crown edges after grinding the frames out of the blank.

NOTE:

- ▷ Do **not** work the contact points to the antagonist and the neighbouring teeth with these polishers, as they are too abrasive.
- ▷ Prevent excessive heat generation.

POST-polishers

These polishers are used on monolithic crowns and bridges for:

- _ Polishing the contact points to the antagonist and neighbouring tooth
- _ Thinning out crown edges
- _ Polishing basal-positioned pontic surfaces.

- ▷ Pay particular attention as to high-gloss polish!

When not sufficiently polished, zirconium oxide acts very abrasive on the antagonist or neighbouring tooth.

**NOTE:**

- ▷ Prevent excessive heat generation.
- ▷ Carry out major material removal in high-density sintered state exclusively with the use of a water-cooled turbine and suitable dental diamond points (recommended grain: $\leq 40 \mu\text{m}$).

Dent Kit

When post grinding-in is required in the patient's mouth, these polishers are used for polishing the contact points to the antagonist and neighbouring tooth of monolithic crowns and bridges.

- ▷ Pay particular attention as to high-gloss polish!
When not sufficiently polished, zirconium oxide acts very abrasive on the antagonist or neighbouring tooth.

**NOTE:**

- ▷ Prevent excessive heat generation.
- ▷ Work with water-cooling.
- ▷ Carry out major material removal in high-density sintered state exclusively with the use of a water-cooled turbine and suitable dental diamond points (recommended grain: $\leq 40 \mu\text{m}$).

Correct application

WARNING:

In case of improper usage or material defects, carriers, shanks or the work being processed can break.

- ▷ Always wear safety glasses/goggles when processing Ceramill polish. Alternatively, it is also possible to work behind a protective glass pane.

CAUTION:

In order not to inhale dust from the polisher or the workpiece:

- ▷ Always wear a respirator/breathing protection when processing Ceramill polish. Using a dust extraction system is also recommended.

- ▷ Use only technical and hygienic-sound, properly maintained and cleaned turbines, as well as hand and angle pieces. Accurate true running of turbine and angle piece drive are necessary.
- ▷ For vibration-free working, center unmounted polishers after mounting their shank.
- ▷ Use only high-quality carriers. Low-grade carriers can break and cause injuries.
- ▷ Dispose of bent or out-of-true instruments immediately.
- ▷ Clamp instruments as deeply as possible.
- ▷ Run instruments up to speed before applying them to the object.

- ▷ Polish with a light circular motion in order to prevent the formation of grooves.
- ▷ Prevent wedging, jamming or levering, as this increases the danger of breakage.

Instructions on speed

CAUTION:

Polishers can become damaged when exceeding the maximum permissible speed, and thus result in personal injury.

- ▷ Never exceed the maximum permissible speed!

The recommended speeds and the maximum permissible speeds differ from product to product (see charts 1 to 3 on page 19 ff and the original packaging).

To achieve optimal working results:

- ▷ Observe the recommended speed range.

Pressure forces

CAUTION:

Excessive pressure forces cause overheating, which can lead to pulp damage in the mouth.

- ▷ Always prevent excessive pressure forces!

Water-cooling

CAUTION:

Insufficient water-cooling can lead to irreversible damage of the tooth and the surrounding tissue.

- ▷ To avoid unwanted heat-development at the tooth, provide for sufficient water-cooling (at least 50 ml/min)!

Storage

The instruments are to be stored in hygienically-maintained racks or other suitable containers.

Cleaning, Disinfection and Sterilization

NOTE:

- ▷ When disinfecting and sterilizing, make sure that the selected procedure is suitable for the respective instrument.

NOTE:

- ▷ Wear protective clothing and gloves when processing contaminated instruments.

**NOTE:**

Aggressive agents, too long exposure or incorrect dosing can destroy the polishers.

- ▷ Use only suitable disinfectants for polishers.
- ▷ Observe the manufacturer's instructions on suitability and exposure duration of disinfectants and cleaning agents.

Before using for the first time on patients:

- ▷ Disinfect and sterilize Ceramill POLISH polishers.

After oral use:

- ▷ Immediately disinfect polishers.
- ▷ Afterwards, perform cleaning, disinfection and sterilization again.

Sterilization

Sterilization shall take place in a vacuum autoclave at 134 °C for 20 minutes, or according to RKI guidelines.

**NOTE:**

Prevent damage to polishers.

- ▷ Do not exceed 150 °C. Do not sterilize the products in a Chemi-clave sterilizer or in a hot-air disinfector.

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EVE Ernst Vetter GmbH
Amann Girrbach AG