- DE** Gebrauchsanweisung
- EN** Instructions for use
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- ES** Instrucciones de uso
- PT** Instruções de utilização
- PL** Instrukcja użycia
- NL** Gebruiksaanwijzing
- SV** Bruksanvisning
- DA** Brugsanvisningen
- LT** Informacinis Lapelis
- LV** Lietošanas informācija
- HU** Használati tájékoztató
- CS** Informace k použití
- EL** Οδηγίες χρήσης
- RO** Instrucțiuni de utilizare
- USA** Instructions for use

Manufacturer | Hersteller

Distribution | Vertrieb

Merz Dental GmbH
Eetzweg 20
24321 Lütjenburg | Germany
Fon +49 4381 403-0
Fax +49 4381 403-403

www.merz-dental.de

90794/2018-10/04

Amann Girrbach AG
Herrschaftswiesen 1
6842 Koblach | Austria
Fon +43 5523 62333-105
Fax +43 5523 62333-5119
austria@amanngirrbach.com
www.amanngirrbach.com

Made in the European Union

CE 0482

Rx Only

EN ISO 13485

AMANNGIRRBACH

32572-FB 2018-10-01

Ceramill Splintec

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EN

Ceramill Splintec

PMMA milling blank without fillers and fibres

Instructions for use, please read carefully!

INTENDED USE

Ceramill Splintec, clear, used for removal of overburden or incorrect burden of hard tooth substance, the periodontal apparatus and the mandibular joints with splints (occlusal splints) / fabrication of patient-customised drilling templates for the correct positioning of implants.

INDICATION

- _ Physiological dysfunction leading to abrasion of hard tooth substance
- _ Complete loss of individual teeth

CONTRAINDICATION

- _ Hypersensitivity to acrylate compounds

TECHNICAL DATA

PHYSICAL VALUES FOR CERAMILL SPLINTEC

Test according to

Solubility	0.2 µg/mm ³	EN ISO 10477
Water uptake	26.5 µg/mm ³	EN ISO 10477
Modulus of elasticity	2773 MPa	EN ISO 20795-1:2013
Flexural strength	96.6 MPa	EN ISO 20795-1:2013

COMPOSITION

- _ Polymethylmethacrylate (PMMA) and cross-linked polymers based on methacrylic acid esters
- _ Residual peroxide as dibenzoyl peroxide
- _ Methylmethacrylate (MMA) may be contained as residual monomer up to max. 1%

SHADE

Clear (transparent)

WEAR PERIOD

Can be worn up to 3 years

STORAGE

No special measures required

WARNINGS

Processing Ceramill Splintec frameworks may generate dust which can lead to mechanical irritation of the eyes and airways. Therefore, always ensure that the extraction system on your milling machine and at the workplace is working properly for individual post-processing and that you are using personal protective equipment. Further safety information can be obtained from the safety data sheet.

GENERAL INFORMATION ON CERMILL SPLINTEC CONSTRUCTIONS, CLEAR

_ Minimum wall thickness after polishing ≥ 1 mm

TRIMMING

_ Cross-cut carbide burs suitable for resin or suitable cutting discs should be used to rework and separate frameworks, crowns, bridges, denture bases, bite splints and drilling templates from the blank.

_ To prevent inaccurate fitting after fabricating the construction, avoid excessive heat development during trimming and polishing.

RESTORATION

Frameworks, crowns, bridges, denture bases, bite splints and drilling templates, which have been fabricated from Ceramill Splintec blanks can be restored and supplemented at any time with all standard MMA-based cold-curing polymers, such as Combipress N/LM, Weropress® or artConcept®. Prior conditioning of the surface is necessary, preferably with artConnect or a similar MMA-based bonding agent.

NOTES

Changes to the construction through use, e.g. due to fissures, cracks, clouding or significant signs of abrasion, necessitate a repair or refabrication. Monitoring should be carried out with regular assessment of the denture at the follow-up presentation in the dental surgery. Enter the LOT No. with each procedure that requires identification of the material. To ensure traceability and assignment of the product data as well as the LOT numbers for the patient at any time, it is recommended to store the milling blank in the original packaging during two milling operations.

DISPOSAL

Residues from processing the Ceramill Splintec have to be disposed of as residual waste.

SAFETY DATA SHEETS

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from www.merz-dental.de.

Date of information 2018-07

The product properties are based on complying with and observing these instructions for use.

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CAUTION

Federal (USA) law restricts this device by or on the order of a licensed dental practitioner.

NOTE

Ceramill Splintec is designed for milling equipment from Amann Girrbach or Equivalent.

INTENDED USE

Ceramill Splintec, clear, indicated for the fabrication of removable bite splints such as mouthguards and nightguards, and of drill guides for placement of endosseous implants.

CONTRAINDICATION

_ Hypersensitivity to acrylate compounds

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NOTES FOR THE USER, TRIMMING

- _ Cross-cut carbide burs suitable for resin or suitable cutting discs should be used to rework and separate frameworks, crowns, bridges, denture bases, bite splints and drilling templates from the blank.
- _ To prevent inaccurate fitting after fabricating the construction, avoid excessive heat development during trimming and polishing.

RESTORATION

Crowns, bridges, denture bases, bite splints and drilling templates, which have been fabricated from Ceramill Splintec can be restored and supplemented at any time with all standard MMA-based cold-curing polymers, such as Weropress®. Prior conditioning of the surface is necessary, preferably with a standard MMA-based bonding agent.

Wear period

Can be worn up to 3 years

STORAGE

No special measures required. See label for expiration date

NOTES

Changes to the construction through use, e.g. due to fissures, cracks, clouding or significant signs of abrasion, necessitate a repair or refabrication. Monitoring should be carried out with regular assessment of the denture at the follow-up presentation in the dental surgery. Record the LOT number with every process that requires identification of the material. To ensure traceability and assignment of the product data as well as the LOT numbers for the patient at any time, it is recommended to store the milling blank in the original packaging during two milling operations.

RISK PHRASES

Processing Ceramill Splintec may generate dust which can lead to mechanical irritation of the eyes and airways. Therefore, always ensure that the extraction system on your milling machine and at the workplace is working properly for individual post-processing and that you are using personal protective equipment.

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Date of information 2018-07

The product properties are based on compliance with and observation of these instructions for use.

LOT

Batch code

REF

Catalog number

Manufacturer

Follow the instructions for use

Usable up to