# **ceramill**<sup>®</sup> stain & glaze

<b>DE</b> Gebrauchsanweisung	3 - 6
<b>EN</b> Instruction Manual	7 - 10
<b>FR</b> Instructions d'utilisation	11 - 14
<b>IT</b> Istruzioni per l'uso	15 - 18
<b>ES</b> Instrucciones de uso	19 - 22

- Original Gebrauchsanweisung -

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- Translation of the original Operating Instructions -

## Table of Contents

<b>1</b>	<b>Explanation of Symbols</b> .....	<b>8</b>
<b>2</b>	<b>Suitable Personnel</b> .....	<b>8</b>
<b>3</b>	<b>Properties</b> .....	<b>9</b>
3.1	Product description .....	9
3.2	Indication .....	9
3.3	Safety data sheet/Declaration of conformity .....	9
3.4	Shelf life .....	9
3.5	Storage .....	9
3.6	Technical data .....	9
<b>4</b>	<b>Warranty/Exclusion of Liability</b> .....	<b>9</b>
<b>5</b>	<b>Application</b> .....	<b>10</b>
5.1	Staining and Glazing .....	10
5.2	Stain and Glaze Firing .....	10
<b>6</b>	<b>Environmental Protection</b> .....	<b>10</b>
<b>7</b>	<b>Download Infos</b> .....	<b>10</b>

EN

## 1 Explanation of Symbols

### Warning indications

Warning indications in the text are marked with a triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- \_ **NOTE** means that property damage can occur.
- \_ **CAUTION** means that light to fairly serious personal injury can occur.
- \_ **WARNING** means that serious personal injury can occur.
- \_ **DANGER** means that life-threatening personal injury can occur.

### Important information

Important information that do not lead to hazards for humans or property damage are marked with the icon aside and are also boxed.

### Other symbols in the Manual

Symbol	Meaning
▷	Item of an operation description
—	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

### Other symbols on the product

Symbol	Meaning
	Catalog number
	Batch code
	Manufacturer
	Consult instructions for use
	Used by YYYY-MM-DD or YYYY-MM

## 2 Suitable Personnel

### NOTE:

The product may only be used by trained dental technicians.

### 3 Properties

#### 3.1 Product description

Ceramill Stain are stains for direct application onto zirconium oxide. They are intended for shade corrections and slight color adaptations.

Ceramill Glaze is a glaze for direct application onto zirconium oxide.

Ceramill Stain & Glaze Working Liquid and Ceramill Stain & Glaze Reflow Liquid are liquids for diluting Ceramill Stain and Ceramill Glaze.

#### 3.2 Indication

Monolithic crowns and bridges of translucent zirconium oxide.

#### 3.3 Safety data sheet/Declaration of conformity

The data sheet on safety and the declaration of conformity are available on request under [www.amannjirrbach.com](http://www.amannjirrbach.com).

#### 3.4 Shelf life

- \_ Stain and glaze: Five years (minimum)
- \_ Liquids: Four years (minimum).

#### 3.5 Storage

Store in the original packaging in a dry, cool and dark location.

#### 3.6 Technical data

	Unit	Value
Thermal expansion coefficient (25 - 500 °C)	1/K	$8 \times 10^{-6}$
Glass transition point	°C	530 ± 10
Bending strength	MPa	85
Grain size D90/10	µm	20
Chemical solubility	µg/cm <sup>2</sup>	16

Tab. 1

#### Composition (in percentage by mass)

	Unit	Value
SiO <sub>2</sub>	%	42 - 45
Al <sub>2</sub> O <sub>3</sub>	%	5,5 - 7
K <sub>2</sub> O	%	4 - 5
Na <sub>2</sub> O	%	5 - 7
Other oxides	%	37 - 43,5

Tab. 2

### 4 Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

## 5 Application

### 5.1 Staining and Glazing

The stains can be mixed among each other.

The stains contain glaze compound. Therefore, mixing stains with glaze is not required.

The stains and glaze compound can be applied together in a single workstep.

The crown/bridge must be smooth or polished. Not recommended for sand-blasted surfaces.

- ▷ Clean frames intensively using ultrasonic bath and/or dental steam cleaner.
- ▷ Before use, mix stain and glaze compound with a glass spatula.
- ▷ Remove the required quantity of stain or glaze compound with a glass spatula.
- ▷ Apply the stain and glaze compounds onto the partly smooth, partly polished zirconium oxide surfaces.

The applied stain and glaze compounds should not exceed a plating thickness of 1 mm..

A too thick application or consistency can cause the formation of stains.

- ▷ Adjust the consistency and stain intensity using the Working Liquid. The more Liquid, the more viscous and less color-intensive the stain.
- ▷ In areas where no stain is applied, completely coat the object with glaze compound. The basal surface of the pontics can also be covered or coated.

When the stain/glaze has dried:

- ▷ Bring the stain/glaze back to its original condition using the Reflow Liquid (shake well before using).

### 5.2 Stain and Glaze Firing

The firing does not have to take place in a vacuum.

The Stain and Glaze firing can take place in a single process.

- \_ Starting temperature: 450 °C
- \_ Drying period: 5 min
- \_ Temperature increase: 40 °C/min
- \_ Final temperature: 850 °C
- \_ Dwell time: 1 min
- \_ For bridges with more than five units and/or massive intermediate units, we recommend step-by-step long-term cooling to 500°C .

The resulting appearance is glossy.

When the desired shade was not achieved, corrective adjustment is possible by applying another firing run.

## 6 Environmental Protection

### Disposal

- \_ May not be disposed of together with household waste.
- \_ Do not dispose of into sewer systems.
- \_ Dispose of the contents/container in accordance with local/regional/ international regulations.

## 7 Download Infos

Further instructions/manuals can be downloaded under

[www.amanngirrbach.com/instruction-manuals](http://www.amanngirrbach.com/instruction-manuals).

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**Amann Girrbach AG**
**Amann Girrbach GmbH**