Achtung  
Warning  
Attenzione  
Atención

ISO 13485  
Medizinprodukt  
Medical device  
Dispositif médical  
Dispositivo medico  
Producto sanitario

## SYMBOLERKLÄRUNG

### Warnhinweise

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## EXPLANATION OF SYMBOLS

### Warning indications

- Warning indications in the text are marked with a triangle and boxed.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- **NOTE** means that property damage can occur.
- **CAUTION** means that light to fairly serious personal injury can occur.

### Important information

Important information that do not lead to hazards for humans or to property damage are marked with the icon aside and are boxed.

## Other symbols in the Manual

### Symbol Meaning

>	Item of an operation description
-	Item of a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

## GENERAL SAFETY INSTRUCTIONS

### CAUTION:

May cause an allergic skin reaction. Harmful to aquatic life with long lasting effects.  
 > Avoid release to the environment.  
 > Wear protective gloves/protective clothing/eye protection.  
 > If skin irritation or rash occurs: Get medical advice/attention.  
 > Observe safety data sheet

### CAUTION:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## SUITABLE PERSONNEL

### CAUTION:

The product may only be processed by trained dental technicians/dentists.

## PROPERTIES

### Product Description

Ceramill modelling materials are light-curing modelling resins, which are thin-flowing and kneadable in their consistencies. Ceramill Gel (760514) is thin-flowing and is used for modelling the copings/crowns. It is slightly transparent and easy to apply with the dosing syringe.

### NOTE:

Ceramill Gel can harden when not being used and clog the metal tip. To avoid this:  
 > Close the syringe opening with a cured remainder of silicone.

To insulate plaster and stump lacquer against synthetic modelling material, Ceramill Sep insulation is available and has been formulated for use with these materials.

### Safety data sheet/Declaration of conformity

The safety data sheet is available for download on the Amann Gırrbach website under *Services > Downloads > Additional documents*. The product's declaration of conformity is available upon request from the manufacturer.

### Composition

- Acrylates
- Pigments
- Phosphine oxide

### Storage

- Do not store above 25 °C.
- Store only in original packaging.
- Keep container tightly sealed.
- Protect against the incidence of light.

### Shelf life

Ceramill Gel can be used until the expiry date, if stored appropriately.  
 > Do not use after expiration date Ceramill Gel has elapsed.

### Side effects

- Patient and user: Ceramill modelling materials can cause allergic reactions.
- User: Frequent and prolonged skin contact with unpolymerised material may cause skin irritation.

### Interactions

There are no known interactions.

### Contraindications

Patient and user: Ceramill modelling materials should not be used if there is a known allergy to any of the ingredients.

## DELIVERY UNITS

### Art. No.

760514 Ceramill Gel - Pckg. à 4 x 3 g

## HANDLING

When processing at room temperature and using a Dentacolor XS UV lamp, the following applies:

- Polymerisation wavelength: 400 - 500 nm
- Polymerisation duration: 180 s

Light-curing plastics always shrink during polymerisation. In order to minimise this effect, the modelling resin must be applied step by step. The modelling resin polymerises very well with a layer thickness of up to 1 mm.

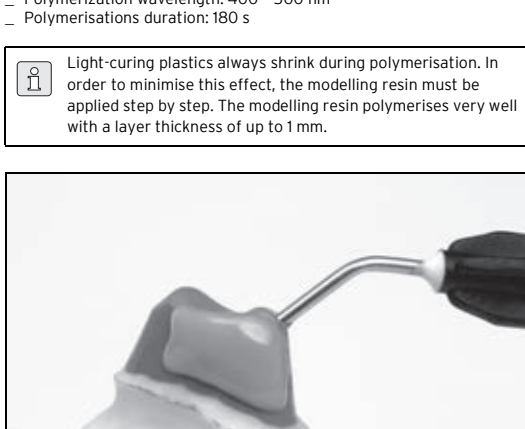
Fig. 1

- > Step by step from the cusp tip/incisal to approx. 1 mm above the preparation margin, apply a strip of maximum 4 mm width in layers of maximum 1 mm.
- > Completely polymerise applied material under a UV lamp (e.g. Dentacolor XS) for 180 seconds.
- > Then apply the next layer to this surface and polymerise.
- > Model the entire coping in this manner.

Recommendation:  
 > Design the coping in a reduced anatomical shape to achieve a statically supportive frame shape for subsequent ceramic veneering.  
 > The more angular the abutment, the smaller the individual surfaces to be polymerised.  
 > Do not model the crown margin in the first step.  
 > When designing the wall thicknesses and connector cross-sections, observe the material-specific specifications of the final frame material (also see the Ceramill ZI processing instructions).

- > Carefully remove the cured coping (still without crown margin) from the abutment, polymerise it from the inside and place it back on the abutment without pressure.

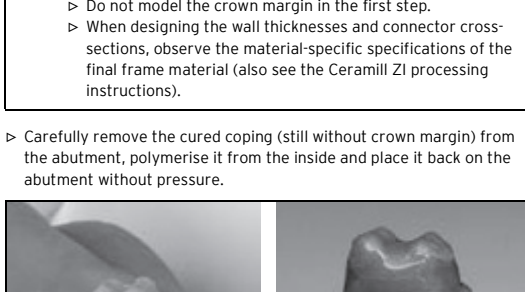
Fig. 2

- > Apply crown margin step by step and polymerise. This increases the hardness and strength of the resin and minimises the possibility of frame distortion.
- > If necessary, rework the frame with rotating tools.
- > Firstly, connect the Ceramill Pontic bridge ponnics with each other [1].

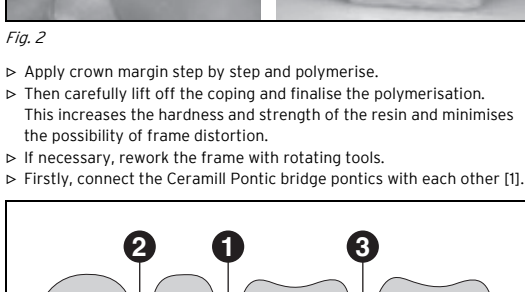
Fig. 3

- > Completely polymerise the compound under a UV lamp (e.g. Dentacolor XS) for 180 seconds.
- > Fix the bridge ponnics to the terminal abutments ([2] and [3]).
- > Completely polymerise the compound under a UV lamp (e.g. Dentacolor XS) for 180 seconds.
- > If the fit of the crown margin is not satisfactory: remove the entire crown margin and renew. If only part of the crown margin is removed and new resin is applied, this can lead to inaccuracies.
- > If the frame is not milled on the same day, leave the model on the plaster model. This avoids possible distortion of the frame.

The plastic frame can be temporarily inserted into the patient's oral cavity to check the fit during complex work or if the frame design is unclear. This try-in, i.e. the time remaining in the patient's oral cavity, is not always carried out at every treatment and takes only a few minutes.

## WARRANTY/EXCLUSION OF LIABILITY

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

## ENVIRONMENTAL PROTECTION

### Disposal

Dispose of the contents/container in accordance with local/regional/international regulations.

## DOWNLOAD INFORMATION

Further instructions/manuals can be downloaded under [www.amanngırrbach.com/instruction-manuals](http://www.amanngırrbach.com/instruction-manuals).

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