## SYMBOLERKLÄRUNG

### Warnhinweise

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## EXPLANATION OF SYMBOLS

### Warning indications

Warning indications in the text are marked with a triangle and boxed.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- **NOTE** means that property damage can occur.
- **CAUTION** means that light to fairly serious personal injury can occur.

### Important information

Important information that do not lead to hazards for humans or to property damage are marked with the icon aside and are boxed.

### Other symbols in the Manual

Symbol	Meaning
▷	Item of an operation description
–	Item of a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

## GENERAL SAFETY INSTRUCTIONS

### CAUTION:

- May cause an allergic skin reaction.
- Toxic to aquatic life with long lasting effects.
- Avoid breathing dust/fume/gas/mist/vapours/spray.
- Wear protective gloves/protective clothing/eye protection.
- If skin irritation or rash occurs: Get medical advice/attention.
- IF ON SKIN: Wash with plenty of water.
- Avoid release to the environment.
- Observe safety data sheet

### CAUTION:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## SUITABLE PERSONNEL

### CAUTION:

The product may only be processed by trained dental technicians/dentists.

## PROPERTIES

### Product Description

Ceramill modelling materials are light-curing modelling resins, which are thin-flowing and kneadable in their consistencies. Ceramill Pontic (760522) has a kneadable consistency and is used for modelling pontics.

To insulate plaster and stump lacquer against synthetic modelling material, Ceramill Sep insulation is available and has been formulated for use with these materials.

### Safety data sheet/Declaration of conformity

The safety data sheet is available for download on the Amann Girrnbach website under *Services > Downloads > Additional documents*. The product's declaration of conformity is available upon request from the manufacturer.

### Composition

- Acrylates
- Pigments
- Phosphine oxide

### Storage

- Do not store above 25°C.
- Store only in original packaging.
- Keep container tightly sealed.
- Protect against the incidence of light.

### Shelf life

Ceramill Pontic can be used until the expiry date, if stored appropriately.

- Do not use after expiration date Ceramill Pontic has elapsed.

### Side effects

- Patient and user: Ceramill modelling materials can cause allergic reactions.
- User: Frequent and prolonged skin contact with unpolymerised material may cause skin irritation.

### Interactions

There are no known interactions.

### Contraindications

Patient and user: Ceramill modelling materials should not be used if there is a known allergy to any of the ingredients.

## DELIVERY UNITS

### Art. No.

760522 Ceramill Pontic - Pckg. à 2 x 3 g

## HANDLING

When processing at room temperature and using a Dentacolor XS UV lamp, the following applies:

- Polymerization wavelength: 400 - 500 nm
- Polymerisations duration: 180 s

Light-curing plastics always shrink during polymerisation.

- Connect the Ceramill Pontic bridge pontics first with each other [1].

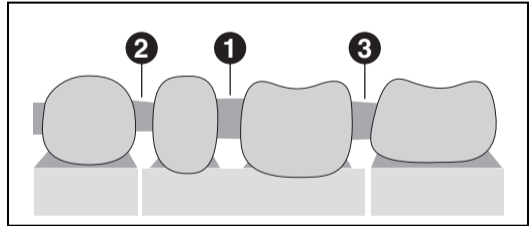
Fig. 2

- Completely polymerise the compound under a UV lamp (e.g. Dentacolor XS) for 180 seconds.
- Fix the bridge pontics to the terminal abutments ([2] and [3]).
- Completely polymerise the compound under a UV lamp (e.g. Dentacolor XS) for 180 seconds.

The plastic frame can be temporarily inserted into the patient's oral cavity to check the fit during complex work or if the frame design is unclear. This try-in, i.e. the time remaining in the patient's oral cavity, is not always carried out at every treatment and takes only a few minutes.

## WARRANTY/EXCLUSION OF LIABILITY

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

## ENVIRONMENTAL PROTECTION

### Disposal

Dispose of the contents/container in accordance with local/regional/international regulations.

## DOWNLOAD INFORMATION

Further instructions/manuals can be downloaded under [www.amanngirrbach.com/instruction-manuals](http://www.amanngirrbach.com/instruction-manuals).

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