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USER MANUAL GIROFORM DIE LINK

782100ff

Giroform Die Link is an assortment of air-drying die spacers containing innovative chemical primers and nano-materials. Giroform Die Link adheres well to dental plasters and gives the die a distinctive, perfectly smooth and scratch-resistant surface.

Giroform Die Link is super thixotropic and can be applied in even and homogeneous layers.

Handling:

Dental plasters used for casting models should always be vacuum-mixed. Surface irregularities, such as pores and scratches, may lead to surface defects. The plaster die should be clean and dry.

- 1) Apply the hardener to increase the bond strength and condition the plaster die. Apply a single generous coat of hardener (also covering the margin). After a short drying time of approx. 30 to 60 seconds, you may apply the appropriate die spacer.

– Giroform Die Link hardener is coloured light blue, thus making it easy to recognise on areas already coated, even on light-coloured plasters. This provides for confident and controlled hardener application.

- 2) Before using the pigmented die spacers, shake the bottles thoroughly to disperse the pigments evenly. Correct mixing ensures an even coating and consistent drying times.

- 3) Pick up the die spacer with the brush. Wipe off the brush on the brim of the bottle and apply rapidly. To do this, place the brush approx. 1 mm above the preparation margin and rotate the die. First, coat one half of the die, working from mesial to distal, then coat the other half. Finally, apply a coat to the occlusal/incisal area.

– Several layers may be applied, according to the desired thickness of the cement gap. Allow one coat to dry for approximately 60 to 90 seconds before applying another.

– Coat only two to three dies in succession, then wet the brush with die spacer once more and wipe off on the rim of the bottle.

– To prevent drying out, replace the brush and close the bottle immediately after use.

- 4) The die spacer reaches final hardness after around 20 minutes, depending on layer thickness, environmental temperature and die material. You may then continue with further work stages, such as fabricating the coping.

In General:

Thinner:

To keep the die spacer at consistent quality, you may need to add thinner from time to time. Add only one or two drops of thinner until the required consistency is reached.

Separating agent:

All die spacers are highly chemically compatible with water and alcohol-based wax separating agents. **Caution:** Solvent-based wax separating agents may dissolve die spacer surface slightly.

Packaging:

Order No.

782100 Starter set with hardener, blue, grey, thinner, tray

782110 Hardener (0 µm) 15ml

782111 Hardener clear (0 µm) 15ml

782120 Die spacer blue (20 µm) 15ml

782130 Die spacer grey (20 µm) 15ml

782140 Die spacer gold (15 µm) 15ml

782150 Die spacer silver (13 µm) 15ml

782160 Die spacer red (8 µm) 15ml

782170 Thinner 15ml

782180 Lube (0 µm) 15ml

Precautions:

 Please pay attention to the hazard pictogram on the particular product.

H319 Causes serious eye irritation. H225 Highly flammable liquid and vapour. H336 May cause drowsiness or dizziness. H314 Causes severe skin burns and eye damage. P280 Wear protective gloves / protective clothing / eye protection / face protection. P233 Keep container tightly closed. P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P337+P313 If eye irritation persists: Get medical advice / attention. P302+P352 IF ON SKIN: Wash with plenty of soap and water. P403+P233 Store in a well-ventilated place. Keep container tightly closed. P210 Keep away from heat / sparks / open flames / hot surfaces. - No smoking. P501 Dispose of contents / container in accordance with local / regional / national regulations. P404 Store in a closed container.

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