Gebrauchsanweisung  
Instructions for use  
Mode d'emploi  
Istruzioni per l'uso  
Instrucciones de empleo

Manufacturer | Hersteller  
Distribution | Vertrieb  
**Amann Girschbach AG**  
Herrschaftswiesen 1  
6842 Koblach | Austria  
Fon: +43 5523 62333-105  
Fax: +43 5523 62333-519  
austria@amanngirschbach.com  
www.amanngirschbach.com

Achtung  
Warning  
Attenzione  
Atención

AMANNGIRSBACH

**Steamer X3**

FIG. A

FIG. B

**Electronic Steamer II**

**Steamer Plus**

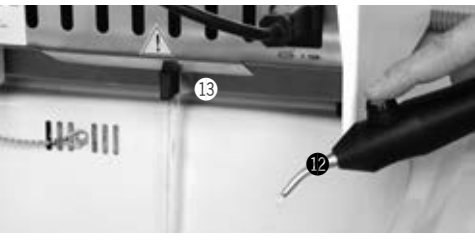
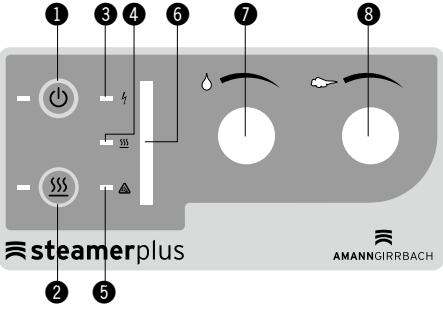
FIG. C

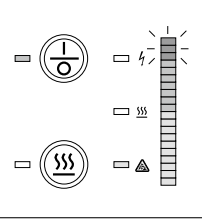
FIG. D

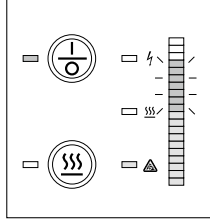
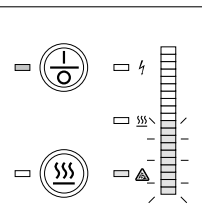
FIG. E

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Original-Gebrauchsanweisung

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**INSTRUCTIONS FOR USE**

Translation of the original instructions for use

**Descaling liquid for Steamer X3, Electronic Steamer II, Steamer Plus**

Order no. 116811 Kalk-X descaling agent, 4 x 250 ml

**DESCALING PROCEDURE FOR THE STEAMER X3**

The descaling procedure must be done when the instrument is cool and not pressurized.

- Do not turn the instrument on during the descaling operation! Only Kalk-X Liquid (art. 116811) must be used for this procedure, which is a descaling agent that is specifically designed for aluminum.
- The unit:  
**FIG. B** / Place the Steamer X3 over a sink and open the drain screw (9) located at the bottom of the instrument. Note: Always empty the instrument by means of the drainage screw (9) and not through the filling neck.  
**WARNING! Never open the drainage screw when the Steamer X3 is pressurized!**
- Once the boiler tank has been completely emptied, close the drainage screw (9).
- Pour the contents of two containers of Kalk-X liquid descaling agent (2 x 250 ml) into the boiler and then fill the tank with warm water.
- Allow the descaling solution to sit for at least 1 hour. After this time, reopen the drainage screw and empty the instrument.
- FIG. A** / Accurately rinse the boiler tank twice using clean water to remove any calcium residues. The gasket on the safety cap (6) should also be checked at this time, replace it if necessary.
- Ensure that the draining screw is properly closed after descaling.
- The Steamer X3 can now be refilled with water and the safety cap screwed back on. Turn the instrument on.
- After the warm-up period, the instrument will be ready for use.

**DESCALING PROCEDURE FOR THE ELECTRONIC STEAMER II**

Descaling should only be carried out when the unit is cold and non-pressurised. Do not switch on the unit during descaling!

- Place the Electronic Steamer II over a washbasin and open the drain plug on the bottom of the unit.  
**CAUTION! Never open the drain plug when the unit is pressurized!**
- Close the drain plug when the unit is completely empty.
- Pour two full bottles of Kalk-X descaling liquid (2 x 250 ml) into the tank and then fill it with lukewarm water.
- Leave the descaling solution in the tank for 1 h.
- At the end of the reaction time open the drain plug again and empty the unit.
- Rinse the tank thoroughly with clean water to flush away any residual lime scale.
- Always change the two seals on the drain plug as well as the seal on the tank cap after descaling. (Use the Order no. 1168432, 116382, 116383)
- Ensure that the drain plug is properly closed after descaling.
- Refill the Electronic Steamer II with water and replace the tank cap. Switch on the unit and then press the reset button on the bottom of the unit to exit the descaling programme.
- After the unit has been heated up, it is ready again for operation.

**ELECTRONIC DESCALING PROGRAMME STEAMER PLUS**

Always ensure the unit is cold and non-pressurised before starting the descaling programme!

- The unit should be connected to the mains during the entire descaling procedure.
- Fully open the saturated steam control (7) and steam control (8).
- Switch on the unit (1) and deactivate the heating (2).
- Hold the unit over a washbasin.  
**Ensure that the drain valve (13) is over the washbasin. Leave the steam handpiece on the edge of the washbasin, as water or steam escapes from the nozzle during descaling.**
- FIG. C** / Start the automatic descaling programme: press the ON/OFF key (1) for 5 s with the unit switched on. All the LEDs on the pressure indicator (6) light up and the red bars start to flash.
- Empty the unit:  
\_ Remove the water tank (9).  
\_ Hold the steam handpiece in the washbasin and press the steam button (12) until no more steam or water comes out.  
\_ Open the drain valve (13).  
\_ Activate the pump to empty the unit by pressing the steam key (12) until no more water comes out of the boiler.  
\_ Then close the drain valve (13) again and press the heating key (2) 1x briefly to ensure the tank is empty. The flashing LEDs go out when the tank is empty.
- Empty any residual water from the watertank (9) and add Kalk-X descaler concentrate in a ratio of 1:5 (1 bottle of descaler: 5 bottles of lukewarm water) to the tank. Replace the water tank. The unit automatically pumps the descaling solution into the boiler. Water may come out of the steam handpiece during this procedure.
- FIG. D** / Descaling requires approx. 1 h and the process can be followed on the green LEDs of the pressure indicator (6). Descaling is complete as soon as all the green bars go out and the orange bars flash.
- FIG. E** / After descaling is complete, empty the unit -> see section 6.
- Rinsing: empty any residual descaling solution from the water tank, rinse the water tank and fill with clean water. Replace the tank on the unit. The unit now pumps the clean water into the boiler.
- After filling it with clean water, empty the unit again to rinse it -> see section 6.
- The descaling programme is complete when all the LEDs on the pressure indicator (6) go out. To end the descaling programme press the ON/OFF key (1) for 5 s.

Three acoustic signals confirm that the descaling has ended. Switch on the Steamer Plus and fill the water tank in the usual way. As soon as the unit has filled the boiler to maximum, press the steam button (12) briefly to reduce the excess pressure built up during filling. After the unit has been heated up, it is ready again for operation.

**Caution! After the descaling programme has started, all other functions of the unit are shut off. Do not disconnect the unit from the mains supply or the descaling procedure will not be properly completed or stored.**

**Safety Aspects**

Keep the descaling agent out of the reach of children! Causes serious eye irritation. Causes skin irritation. Wear protective gloves / protective clothing / eye protection / face protection. Wash thoroughly after handling.

**IF ON SKIN (or hair):** Remove / Take off immediately all contaminated clothing. Rinse skin with water / shower.

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Wash contaminated clothing before reuse. If eye irritation persists: Get medical advice / attention.

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