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
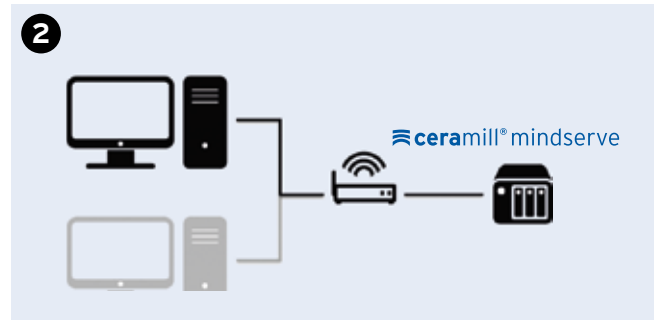
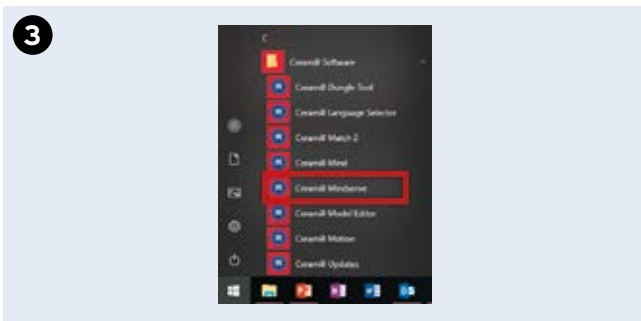
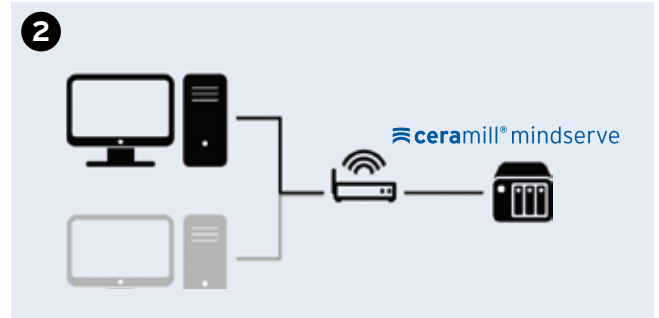
_ Download Ceramill Mindserve Installer from the C3 Portal.

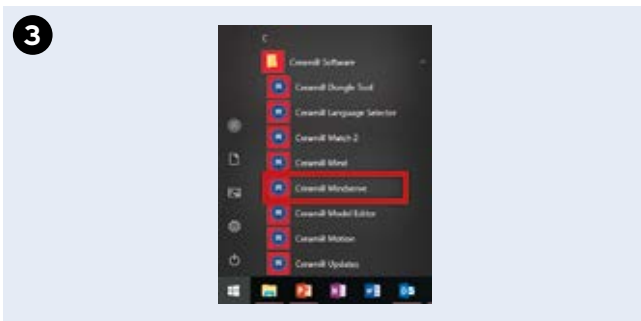_ Connect Ceramill Mindserve to the internal network.
_ Install Ceramill Mindserve Connect Tool.

_ Click on "Ceramill Mindserve" in the start menu under "Ceramill Software".
_ The Ceramill Mindserve Tool starts.

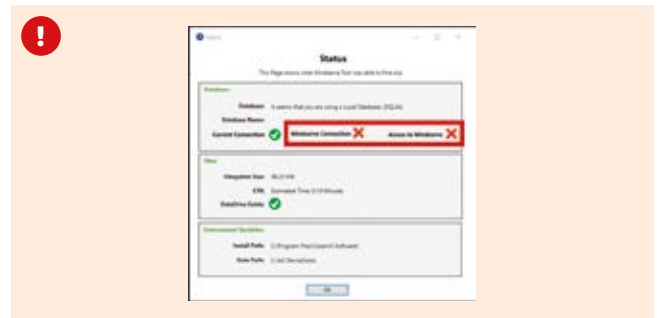
_ The automatic connection control window opens.
_ When all items light up green, close with OK.
_ The Mindserve Tool becomes visible.

_ Click on "Mindserve Setup".
_ Both the files and the database are transferred from the PC to the Ceramill Mindserve.

INFO: synchronization may take some time. Allow at least half an hour. The data are not moved, but only copied.

Ceramill Mindserve is now set up and can be used.

Troubleshooting 1:

_ This error message indicates that the Ceramill Mindserve database cannot be accessed from this PC.

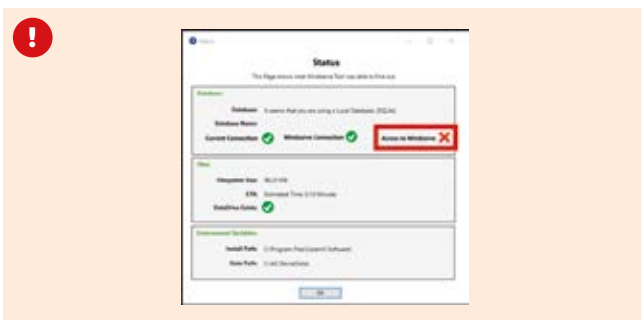
Potential solutions:

1. Switch on Mindserve.
2. Check (cable) connections between PC, router & Mindserve.
3. Send ping to "mindserve" via command prompt (ping mindserve).

```

Eingabeaufforderung
Microsoft Windows [Version 10.0.17134.885]
(c) 2018 Microsoft Corporation. Alle Rechte vorbehalten.
H:\>ping mindserve
    
```

4. Check if port 3306 is blocked by firewall (contact responsible IT employee of the company).
5. Contact Helpdesk.

Troubleshooting 2:

_ Connection to the Ceramill Mindserve is established, but the registered user does not have access to the Ceramill Mindserve database.

Reason: extremely outdated Ceramill Mindserve setup

Potential solution: contact Helpdesk